

INTOXILYZER® 8000 INITIAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006500

A. GENERAL Setup and Checks:

- Power On
 5 min. countdown during initial warm-up
 1 min. countdown to begin test after standby mode
- Display Intensity (Ctrl + ↑ or ↓)
3. Password Security:
 Level 1 (Operator) Level 2 (Field Inspector) Level 3 (Lab Personnel)
- Printer Setup (Level 3,S,P)
- Forms Load (CMI, Inc., CoBrA Software)

B. LEVEL 1 Setup and Checks:

- Internal Printer Test and Paper Feed (Level 1,P). Attach record.
- External Printer Test (Level 1,P). Attach record.
3. Barcode Reader (Level 1,B):
 Operator ID ND License R key: Refusal. Attach record.

C. LEVEL 2 Setup and Checks:

- Continuous Air Blank (Level 2,A)
- Diagnostic (Level 2,D). Attach record.
- Prelim Data Entry (Level 2,E)
4. Ser Number/Version Info (Level 2,V):
Inst Ser#: 80-006500 DSP Ver: 1362.14
Core Ver: 8164.13 CPLD Ver: 7

D. LEVEL 3 Setup and Checks:

- DVM Monitor (Level 3,D,D): 3: 12538 s: 1 C: 47.00
9: 13077 s: 2 B: 45 0
(RFI indicator with keyed radio): 1
2. Function (F) Keys: : F5 – Tone F9 – Fan
 F6 – Green LED F10 – Pump
 F7 – Red LED F11 – Solenoid
3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 964 CF: 1.05
4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 232 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 49.5 I: 0.0
B: 45.1 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 570 psi P (gauge): ~570 psi
7. General Setup (Level 3,S,G)
8. Configure Start Test (Level 3,S,T)
9. Network (Level 3,S,N)

E. TESTS (attach all test records):

Configure wet bath simulator standards before running tests (Level 1,S):

1. Zero AC (Level 1,C):
Sim ser #: DR3375 Lot #: N/A Mill-Q Exp date: N/A
Known value: 0.000 AC
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):
Sim ser #: DR3374 Lot #: 14240 Exp date: 10/20/16
Known value: 0.020 AC
Results: 0.020 AC 0.020 AC 0.020 AC
3. High AC (Level 1,C):
Sim ser #: DR7111 Lot #: 201408C Exp date: 08/12/16
Known value: 0.300 AC
Results: 0.303 AC 0.303 AC 0.303 AC

Configure dry gas standard before running tests (Level 1,S):

4. Ethanol + 0.05% Acetone (Level 1,B):
Sim ser #: DR5144 Lot #: ICS4 Exp date: 09/09/17
Known value: 0.10AC + 0.05% acetone
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):
Sim ser #: DR3455 Lot #: Acc 1 Exp date: 09/29/17
Known value: 0.05% acetone
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):
Sim ser #: DR5189 Lot #: Met 2 Exp date: 09/29/17
Known value: 0.10% methanol
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):
Sim ser #: DR3454 Lot #: Ipa 2 Exp date: 09/29/17
Known value: 0.10% isopropanol
 Display reads "Interferent Detect"

- 8. Ambient Fail (Level 1,B): Display and test record read correct.
- 9. Invalid Sample (Level 1,B): Display and test record read correct.
- 10. Sequence Aborted (Level 1,B): Display and test record read correct.
- 11. ABA (Level 1,B): Test record reads correct.
- 12. Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13. RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14. Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15. Deficient Samples (CMS mode): Display and test record read correct.
- 16. Improper Sample (CMS mode): Display and test record read correct.
- 17. Difference Too Great (CMS mode): Display and test record read correct.
- 18. CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0818 Std Dev: 0.00016 Rel Std Dev: 0.7990

Maintenance/Repairs/Notes: _____

Instrument is acceptable to be used in the field.

Deb Shanaver
Field Inspector's Signature

22 Jan 16
Date

Charles E. Eder
Reviewed by

22 JAN 16
Date

Intoxilyzer Test Record and Checklist
NDDAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 08:54

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMN

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMN

24 characters per line
abcdefghijklmnopqrstuwx
ABCDEFGHIJKLMN

32 characters per line
abcdefghijklmnopqrstuwx123456
ABCDEFGHIJKLMN

42 characters per line
abcdefghijklmnopqrstuwx1234567890abcdef
ABCDEFGHIJKLMN

Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On MemFull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 135
Standard Cyl #? 12458
Standard Expiration? 11/11/2020
Oper No? 777777

Flow Cal. Date: 08/18/2015
Slope 681
Intercept -680985

IR Calibration Date: 08/18/2015

	Sum	Sum
0th Coef(*100):	-3148	-17259
1st Coef(*100):	254475	138520
2nd Coef(*100):	1954	1329
H2O adj(mg/l*10K):	193	414

***** Test End *****

Deb Shanauer
Operator Signature
DEB SHANAUER

Remarks:

Print test
Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 08:58

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 135
Standard Cyl #? 12458
Standard Expiration? 11/11/2020
Oper No? 777777

Flow Cal. Date: 08/18/2015
Slope 681
Intercept -680985

IR Calibration Date: 08/18/2015
 3um 9um

0th Coef(*100): -3148 -17259
1st Coef(*100): 254475 138520
2nd Coef(*100): 1954 1329
H2O adj(mg/l*10k): 193 414

***** Printer Test End *****

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Print Test


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 09:00

Test	AC	Time
01 Room Air	0.000	09:00
02 *Subject Test	REF*	09:00
03 Room Air	0.000	09:01

*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 135
Cyl No = 12458
Expiration Date = 11/11/2020
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Refusal

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 09:01

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Diagnostic test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 09:07

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:08
02 Std. Sol.	0.000	09:08
03 Room Air	0.000	09:09
04 Std. Sol.	0.000	09:10
05 Room Air	0.000	09:10
06 Std. Sol.	0.000	09:11
07 Room Air	0.000	09:11

08 Sim Temp = 34.0°C

Simul Ser No = DR3375

Std Sol No = MILLIQ

County = 08

Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

zero alcohol 0.000 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 09:12

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:13
02 Std. Sol.	0.020	09:13
03 Room Air	0.000	09:14
04 Std. Sol.	0.020	09:15
05 Room Air	0.000	09:15
06 Std. Sol.	0.020	09:16
07 Room Air	0.000	09:16

08 Sim Temp = 34.0°C

Simul Ser No = DR3374
Std Sol No = 14240
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:
low alcohol 0.020AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 09:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:20
02 Std. Sol.	0.303	09:21
03 Room Air	0.000	09:22
04 Std. Sol.	0.303	09:23
05 Room Air	0.000	09:23
06 Std. Sol.	0.303	09:24
07 Room Air	0.000	09:25

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 201408C
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:
High alcohol 0.300AC

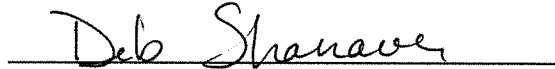
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 09:26

Test	AC	Time
01 Room Air	0.000	09:27
02 *Subject Test	INT*	09:28
03 Room Air	0.000	09:28

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777


Operator Signature
DEB SHANAVER

Remarks:

Interferent detected 0.10AC + 0.0590 acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 09:30

Test	AC	Time
01 Room Air	0.000	09:30
02 *Subject Test	INT*	09:31
03 Room Air	0.000	09:31

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Interferent detected 0.05% acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 09:32

Test	AC	Time
01 Room Air	0.000	09:33
02 *Subject Test	INT*	09:33
03 Room Air	0.000	09:34

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Interferent detected

. Mefhenol
0-10% ~~isopropanol~~ DS

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 09:35

Test	AC	Time
01 Room Air	0.000	09:36
02 *Subject Test	INT*	09:37
03 Room Air	0.000	09:37

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Interferent detected 0.10% isopropanol

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 09:46

Test	AC	Time
01 Room Air	AMB*	09:47
02 Room Air	0.000	09:48

*Invalid Test
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Ambient fail

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 10:15

Test	AC	Time
01 Room Air	0.000	10:15
02 Invalid Sample	X.XXX	10:16
03 Room Air	0.000	10:16

*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Invalid sample

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 10:18

Test	AC	Time
01 Room Air	0.000	10:18
02 *Subject Test	ABT*	10:19
03 Room Air	0.000	10:19

*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Sequence aborted

Form 106-I8000

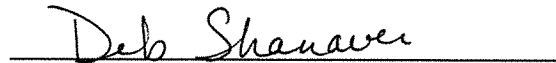
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 10:19

Test	AC	Time
01 Room Air	0.000	10:20
02 Subject Test 1	0.009	10:20
03 Room Air	0.000	10:21
04 Reported AC	0.009	10:20

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777


Operator Signature
DEB SHANAVER

Remarks:

ABA test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 10:22

Test	AC	Time
01 Diagnostic	OK	10:23
02 Room Air	0.000	10:23
03 Subject Test 1	0.000	10:24
04 Room Air	0.000	10:26
05 Std. Gas	0.000*	10:27
06 Room Air	0.000	10:27

*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB SHANAVER

Remarks:

Calibration check out of tolerance

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 10:28

Test	AC	Time
01 Diagnostic	OK	10:29
02 Room Air	RFI*	10:29
03 Room Air	0.000	10:29

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

RFI test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 10:30

Test	AC	Time
01 Diagnostic	OK	10:31
02 Room Air	0.000	10:31
03 *Subject Test	0.000*	10:34
04 Room Air	0.000	10:36
05 Std. Gas	0.082	10:38
06 Room Air	0.000	10:39
07 *Subject Test	0.000*	10:42
08 Room Air	0.000	10:43

No RFI Detected

*Deficient Sample - Value Printed was
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Deficient samples

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 10:43

Test	AC	Time
01 Diagnostic	OK	10:44
02 Room Air	IPS*	10:44
03 Room Air	0.000	10:44

*Improper Sample

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB SHANAVER

Remarks:

Improper sample

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 10:44

Test	AC	Time
01 Diagnostic	OK	10:45
02 Room Air	0.000	10:46
03 Subject Test 1	0.043*	10:46
04 Room Air	0.000	10:48
05 Std. Gas	0.082	10:50
06 Room Air	0.000	10:51
07 Subject Test 2	0.000*	10:51
08 Room Air	0.000	10:52

*Difference Too Great

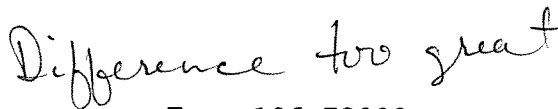
Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB SHANAVER

Remarks:



Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 10:52

Test	AC	Time
01 Diagnostic	OK	10:53
02 Room Air	0.000	10:53
03 Subject Test 1	0.000	10:54
04 Room Air	0.000	10:56
05 Std. Gas	0.082	10:57
06 Room Air	0.000	10:58
07 Subject Test 2	0.000	10:59
08 Room Air	0.000	11:00
09 Reported AC	0.000	10:54

Difference OK
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 25
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

CMS test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = TOXL 8164.13.00 06/09
01/22/2016 09:49

Test	AC	Time
01 Room Air	0.000	09:50
02 Std. Gas	0.081	09:50
03 Room Air	0.000	09:51
04 Std. Gas	0.081	09:51
05 Room Air	0.000	09:52
06 Std. Gas	0.081	09:52
07 Room Air	0.000	09:53
08 Std. Gas	0.082	09:53
09 Room Air	0.000	09:54
10 Std. Gas	0.082	09:54
11 Room Air	0.000	09:54
12 Std. Gas	0.082	09:55
13 Room Air	0.000	09:55
14 Std. Gas	0.082	09:56
15 Room Air	0.000	09:56
16 Std. Gas	0.082	09:57
17 Room Air	0.000	09:57
18 Std. Gas	0.082	09:57
19 Room Air	0.000	09:58
20 Std. Gas	0.081	09:58
21 Room Air	0.000	09:59
22 Std. Gas	0.082	09:59
23 Room Air	0.000	10:00
24 Std. Gas	0.083	10:00
25 Room Air	0.000	10:01
26 Std. Gas	0.083	10:01
27 Room Air	0.000	10:02
28 Std. Gas	0.081	10:02
29 Room Air	0.000	10:03
30 Std. Gas	0.082	10:03
31 Room Air	0.000	10:03
32 Std. Gas	0.082	10:04
33 Room Air	0.000	10:04
34 Std. Gas	0.081	10:05
35 Room Air	0.000	10:05
36 Std. Gas	0.082	10:06
37 Room Air	0.000	10:06
38 Std. Gas	0.081	10:06
39 Room Air	0.000	10:07
40 Std. Gas	0.082	10:07
41 Room Air	0.000	10:08
42 Cal Check Stats:		
43 Average	0.0818	
44 Std Dev	0.0006	
45 Rel Std Dev	0.79%	

*Stability test
Deb Shanaver*

Operator Signature
DEB SHANAVER

Remarks:

lot: 29614080A2 cyl: 25 exp: 12/05/16

Form 106-I8000

Stability test
Deb Shanaver