



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (1-2010)

Serial Number: 80-004936 Instrument Location: DICKINSON LEC

Reason for Install/Repair:
 Install After Lab Repair/Inspection
 Other (Specify) _____

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level 1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: Install @ Dickinson LEC

Ward _____ Date 2-6-11
 Field Inspector Signature

Debs Kashner _____ Date 15 Feb 2011
 Reviewed By

80-004936
 P. 66 of 69
 FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004936
Location = DKPD 8164.13.00 06/09
02/06/2011 16:35

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 15410080A1
Standard Cyl #? 20
Standard Expiration? 07/01/2012
Oper No? 019620

Flow Cal. Date: 01/18/2010
Slope 732
Intercept -767769

IR Calibration Date: 02/04/2011
 3um 9um

0th Coef(*100): -20847 -25572
1st Coef(*100): 253809 130277
2nd Coef(*100): 3314 1256
H2O adj(mg/l*10k): 614 629

***** Printer Test End *****



Operator Signature
WILLIAM D. VANCE

Remarks: *Print Test*

80-004936
P. 67 of 69
EM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004936
Location = DKPD 8164.13.00 06/09
02/06/2011 16:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:36
02 Std. Gas	0.078	16:37
03 Room Air	0.000	16:37
04 Std. Gas	0.079	16:37
05 Room Air	0.000	16:38
06 Std. Gas	0.079	16:38
07 Room Air	0.000	16:39

Lot No = 15410080A1
Cyl No = 20
Exp Date = 07/01/2012
County = 45 Oper No. = 019620

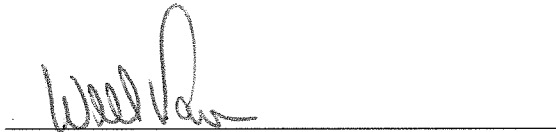

Operator Signature
WILLIAM D. VANCE

Remarks: **ACA**

Form 106-I8000

80-004936
P. 68 of 69
SAM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004936
Location = DKPD 8164.13.00 06/09
02/06/2011 16:55

Test	AC	Time
01 Diagnostic	OK	16:56
02 Room Air	RFI*	16:56
03 Room Air	0.000	16:57

*Invalid Test
Inhibited - RFI

Sub Name = TEST, RFI NA
Sub DOB = 11/11/1970
Sub Sex = Male Weight = NA
Test = DUI Cit = 111111
Dr. Lic. = ND/11111
Lot No = 15410080A1
Cyl No = 20
Expiration Date = 07/01/2012
County = 45 Oper No. = 019620

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
WILLIAM D. VANCE

Remarks:

RFI Test

Form 106-I8000

80-004936
P. 69 of 69
JMV