

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004935

Location: TOX LAB

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5190 Lot No. — Rep AC 0.021 AC ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5113 Lot No. — Rep AC 0.285 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. — Rep AC 0.05% Acetone + 0.10 AC ✓
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓

K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. <u>659 358</u>	Cyl No. <u>38</u>	Exp. Date <u>9/03/2011</u>
Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.078</u>	Test 5 <u>0.080</u>	Test 8 <u>0.079</u>
Test 3 <u>0.080</u>	Test 6 <u>0.078</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

L. Remarks/Maintenance record: Annual Inspection

10 Feb 2010
Date

Mary Pearson
Field Inspector's Signature

DK
~~10 Feb 2010~~ 12 Feb 2010
Date

Deb Kashner
Reviewed by

80-004935
24/32 PK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = TOXL 8164.13.00 06/09
02/10/2010 10:11

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer (Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659358
Standard Cyl #? 38
Standard Expiration? 09/03/2011
Oper No? 107501

Flow Cal. Date: 01/20/2010
Slope 696
Intercept -664812

IR Calibration Date: 02/10/2010

	3um	9um
0th Coef(*100):	-4573	-2516
1st Coef(*100):	251841	123171
2nd Coef(*100):	2261	1381
H2O adj(mg/l*10k):	529	468

***** Printer Test End *****

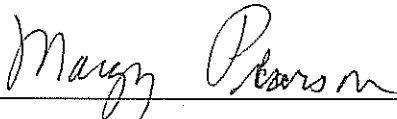
Operator Signature
MARGY PEARSON

Remarks:

Print

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = TOXL 8164.13.00 06/09
02/10/2010 09:32

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:32
02 Std. Sol.	0.020	09:33
03 Room Air	0.000	09:34
04 Std. Sol.	0.020	09:35
05 Room Air	0.000	09:35
06 Std. Sol.	0.019	09:36
07 Room Air	0.000	09:36

08 Sim Temp = 34.0°C

Simul Ser No = 0000000

Std Sol No = 000000

County = 08

Oper No. = 107501

Margy Pearson

Operator Signature

MARGY PEARSON

Remarks:

LOW 0.021AC

Form 106-I8000

80-004935
24/32 KLR

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = TOXL 8164.13.00 06/09
02/10/2010 09:47

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:48
02 Std. Sol.	0.285	09:49
03 Room Air	0.000	09:49
04 Std. Sol.	0.286	09:50
05 Room Air	0.000	09:51
06 Std. Sol.	0.286	09:51
07 Room Air	0.000	09:52


08 Sim Temp = 34.0°C

Simul Ser No = 0000000

Std Sol No = 000000

County = 08

Oper No. = 107501

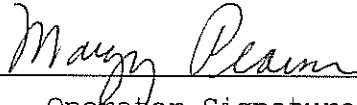

Operator Signature

MARGY PEARSON

Remarks:

Linearity 0.285 AC

Form 106-18000

80-004935
27/32 YK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = TOXL 8164.13.00 06/09
02/10/2010 09:59

Test	AC	Time
01 Room Air	0.000	10:00
02 *Subject Test	INT*	10:01
03 Room Air	0.000	10:01

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 0000000
Std Sol No = 000000
County = 08 Oper No. = 107501

Margy Pearson

Operator Signature
MARGY PEARSON

Remarks:

Interfer-Alc + Acetone

Form 106-I8000

80-004935
24/32 1712

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = TOXL 8164.13.00 06/09
02/10/2010 10:03

Test	AC	Time
01 Diagnostic	OK	10:04
02 Room Air	RFI*	10:04
03 Room Air	0.000	10:05

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 0000000
Std Sol No = 000000
County = 08 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

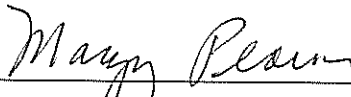

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004935
29/32 JFR


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = TOXL 8164.13.00 06/09
02/10/2010 10:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:14
02 Std. Gas	0.079	10:14
03 Room Air	0.000	10:14
04 Std. Gas	0.078	10:15
05 Room Air	0.000	10:15
06 Std. Gas	0.080	10:16
07 Room Air	0.000	10:16

Lot No = 659358
Cyl No = 38
Exp Date = 09/03/2011
County = 08 Oper No. = 107501

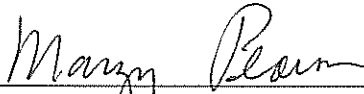

Operator Signature
MARGY PEARSON

Remarks:

MAP
ACA ~~0.106 AC~~ 0.080 AC

Form 106-I8000

80-004935
30/32 KH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = TOXL 8164.13.00 06/09
02/10/2010 10:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:17
02 Std. Gas	0.079	10:18
03 Room Air	0.000	10:18
04 Std. Gas	0.080	10:18
05 Room Air	0.000	10:19
06 Std. Gas	0.078	10:19
07 Room Air	0.000	10:20

Lot No = 659358

Cyl No = 38

Exp Date = 09/03/2011

County = 08

Oper No. = 107501

Margy Pearson

Operator Signature
MARGY PEARSON

Remarks:

MP
ACA 0.106 AC 0.080 AC

Form 106-I8000

80-004935
3/32 FK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = TOXL 8164.13.00 06/09
02/10/2010 10:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:21
02 Std. Gas	0.079	10:21
03 Room Air	0.000	10:22
04 Std. Gas	0.079	10:22
05 Room Air	0.000	10:23
06 Std. Gas	0.079	10:23
07 Room Air	0.000	10:23

Lot No = 659358
Cyl No = 38
Exp Date = 09/03/2011
County = 08 Oper No. = 107501

Margy Pearson

Operator Signature
MARGY PEARSON

Remarks:

MAP
ACA 0.106 AC 0.080 AC

Form 106-18000

80-004935
32/32 KRL