



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (5-2011)

Serial Number 80-004201	Instrument Location GFAF
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>Upload/Download Info ; Installed new cylinder</u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>John W. [Signature]</i>	Date 1/31/2013 ^{DK} 02/11/2013
Reviewed By (Crime Laboratory Use Only) <i>Deb Kasher</i>	Date 25 Feb 2013

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = GFAF 8164.13.00 06/09
01/31/2013 20:42

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 05911080A1
Standard Cyl #? 10
Standard Expiration? 02/01/2013
Oper No? 031121

Flow Cal. Date: 06/03/2009
Slope 696
Intercept -646527

IR Calibration Date: 04/18/2012
 3um 9um

0th Coef(*100): -12334 -9834
1st Coef(*100): 257748 124056
2nd Coef(*100): 7079 2216
H2O adj(mg/l*10k): 716 632

***** Printer Test End *****


Operator Signature
JOHN STOTLER

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = GFAP 8164.13.00 06/09
01/31/2013 20:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	20:53
02 Std. Gas	0.081	20:53
03 Room Air	0.000	20:54
04 Std. Gas	0.081	20:54
05 Room Air	0.000	20:55
06 Std. Gas	0.081	20:55
07 Room Air	0.000	20:55

Lot No = 16512080A2
Cyl No = 29
Exp Date = 08/01/2014
County = 65

Oper No. = 031121



Operator Signature
JOHN STOTLER

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = GFAP 8164.13.00 06/09
01/31/2013 21:04


DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	21:04
02 Room Air	RFI*	21:04

*Invalid Test
Inhibited - RFI

Lot No = 16512080A2
Cyl No = 29
Exp Date = 08/01/2014
County = 65

Oper No. = 031121


Operator Signature
JOHN STOTLER

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = GFAF 8164.13.00 06/09
02/11/2013 15:04

Test	AC	Time
01 Diagnostic	OK	15:06
02 Room Air	RFI*	15:06
03 Room Air	RFI*	15:06

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST TEST
Sub DOB = 10/10/1965
Sub Sex = Male Weight = 120
Test = DUI Cit = NA
Dr. Lic. = ND/TEST
Lot No = 16512080A2
Cyl No = 29
Expiration Date = 08/01/2014
County = 65 Oper No. = 031121

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
JOHN STOTLER

Remarks: