



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (03-2014)

Serial Number 80-004947	Instrument Location NAPO
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- ~~2~~ 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ~~9~~ 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Tom Maly</i>	Date 2/16/2016
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date 11 Feb 16

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004947
Location = NAPO 8164.13.00 06/09
02/11/2016 12:23

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 29614080A2
Standard Cyl #? 31
Standard Expiration? 12/05/2016
Oper No? 131150


Flow Cal. Date: 01/19/2010
Slope 679
Intercept -681178

IR Calibration Date: 05/12/2015

	3um	9um
--	-----	-----

0th Coef(*100):	-14993	-20474
1st Coef(*100):	257346	132154
2nd Coef(*100):	2870	1330
H2O adj(mg/l*10k):	492	531

***** Printer Test End *****

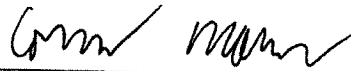

Operator Signature
CONNER MONK

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004947
Location = NAPO 8164.13.00 06/09
02/11/2016 12:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:07
02 Std. Gas	0.079	12:08
03 Room Air	0.000	12:08
04 Std. Gas	0.079	12:08
05 Room Air	0.000	12:09
06 Std. Gas	0.080	12:09
07 Room Air	0.000	12:10


Lot No = 29614080A2

Cyl No = 31

Exp Date = 12/05/2016

County = 24

Oper No. = 131150

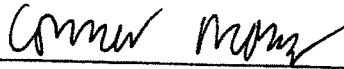

Operator Signature
CONNER MONK

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004947
Location = NAPO 8164.13.00 06/09
02/11/2016 12:30

Test	AC	Time
01 Diagnostic	OK	12:32
02 Room Air	RFI*	12:32
03 Room Air	0.000	12:33

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST TEST
Sub DOB = 01/01/2001
Sub Sex = Male Weight = 123
Test = DUI Cit = 12367890
Dr. Lic. = ND/TEST
Lot No = 29614080A2
Cyl No = 31
Expiration Date = 12/05/2016
County = 24 Oper No. = 131150

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CONNER MONK

Remarks: