

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004956

Location: TOX LAB

- | | Check When Done |
|---|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5190</u> Lot No. <u>—</u> Rep AC <u>0.021 AC</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5113</u> Lot No. <u>—</u> Rep AC <u>0.285 AC</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR5135</u> Lot No. <u>—</u> Rep AC <u>0.10 AC + 0.05% Acetone</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>668180</u>	Cyl No. <u>94</u>	Exp. Date <u>9/21/2011</u>
Test 1 <u>0.079</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.079</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080 AC</u>		

L. Remarks/Maintenance record: Annual Inspec

<u>12 February 2010</u> Date	<u>Mary Plean</u> Field Inspector's Signature
16 Feb 2010 ^{DK} <u>12 Feb 2010</u> Date	<u>Deb Kashur</u> Reviewed by

80-004956
25/33 JH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.13.00 06/09
02/12/2010 14:36

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:37
02 Std. Sol.	0.020	14:38
03 Room Air	0.000	14:38
04 Std. Sol.	0.020	14:39
05 Room Air	0.000	14:40
06 Std. Sol.	0.020	14:40
07 Room Air	0.000	14:41


08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 0000

County = 08

Oper No. = 107501

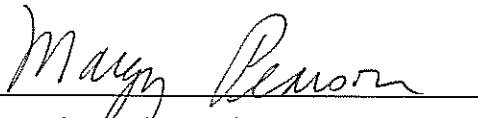

Operator Signature

MARGY PEARSON

Remarks:

Low AC 0.021

Form 106-I8000

80-004956
27/33 KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.13.00 06/09
02/12/2010 14:49

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:50
02 Std. Sol.	0.287	14:51
03 Room Air	0.000	14:52
04 Std. Sol.	0.289	14:52
05 Room Air	0.000	14:53
06 Std. Sol.	0.289	14:54
07 Room Air	0.000	14:54

08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 0000

County = 08

Oper No. = 107501

Margy Pearson

Operator Signature
MARGY PEARSON

Remarks:

Linearity 0.285

Form 106-I8000

80-004956
28/33 FM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.13.00 06/09
02/12/2010 14:56

Test	AC	Time
01 Room Air	0.000	14:57
02 *Subject Test	INT*	14:57
03 Room Air	0.000	14:58

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 000000
Std Sol No = 0000
County = 08 Oper No. = 107501

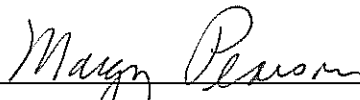

Operator Signature
MARGY PEARSON

Remarks:

Alcohol + Acetone

Form 106-I8000

80-004956
29/33 KAK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.13.00 06/09
02/12/2010 15:00

Test	AC	Time
01 Diagnostic	OK	15:00
02 Room Air	RFI*	15:01
03 Room Air	0.000	15:01

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 000000
Std Sol No = 0000
County = 08 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

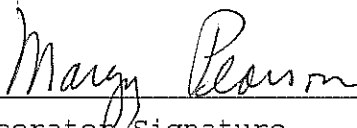

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004956
30/33 1A4


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.13.00 06/09
02/12/2010 10:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:25
02 Std. Gas	0.079	10:25
03 Room Air	0.000	10:25
04 Std. Gas	0.080	10:26
05 Room Air	0.000	10:26
06 Std. Gas	0.080	10:27
07 Room Air	0.000	10:27

Lot No = 668180
Cyl No = 94
Exp Date = 09/21/2011
County = 08 Oper No. = 107501

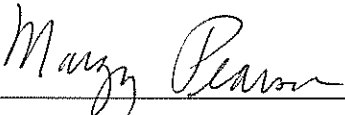

Operator Signature
MARGY PEARSON

Remarks:

ACA Cal Check
Form 106-I8000

*80-004956
3/33 YH*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.13.00 06/09
02/12/2010 10:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:30
02 Std. Gas	0.080	10:30
03 Room Air	0.000	10:30
04 Std. Gas	0.079	10:31
05 Room Air	0.000	10:31
06 Std. Gas	0.080	10:32
07 Room Air	0.000	10:32

Lot No = 668180
Cyl No = 94
Exp Date = 09/21/2011
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

ACA Cal Check

Form 106-I8000

*80-004956
32/33 KM*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.13.00 06/09
02/12/2010 10:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:36
02 Std. Gas	0.080	10:37
03 Room Air	0.000	10:37
04 Std. Gas	0.080	10:37
05 Room Air	0.000	10:38
06 Std. Gas	0.080	10:38
07 Room Air	0.000	10:39

Lot No = 668180
Cyl No = 94
Exp Date = 09/21/2011
County = 08 Oper No. = 107501

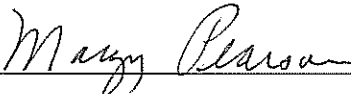

Operator Signature
MARGY PEARSON

Remarks:

ACA Cal check

Form 106-I8000

80-004956
33/33 *FA*