



ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Jonathan Kaatz</i>		
Location <i>ASHL</i>		Intoxilyzer® Serial Number <i>80-005360</i>
Gas Lot Number <i>29614080A2</i>	Gas Cylinder Number <i>007</i>	Gas Expiration Date <i>12/15/2016</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.083* AC
 - B. *0.085* AC
 - C. *0.084* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Jonathan Kaatz MCSO#4401</i>	Date <i>2/13/16</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Dub Shanaver</i>	Date <i>16 Feb 2016</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005360
Location = ASHL 8164.13.00 06/09
02/13/2016 00:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	00:23
02 Std. Gas	0.083	00:23
03 Room Air	0.000	00:24
04 Std. Gas	0.085	00:24
05 Room Air	0.000	00:25
06 Std. Gas	0.084	00:25
07 Room Air	0.000	00:26

Lot No = 29614080A2

Cyl No = 7

Exp Date = 12/05/2016

County = 26

Oper No. = 129953



Operator Signature
JONATHAN KAATZ

Remarks:

Form 106-I8000