



ETHANOL BREATH STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (5-2011)

| | | |
|--|--|--|
| Chemical Test Operator Name (Print) <i>Mark W. McDonald</i> | | |
| Location <i>Wolley City Police Dept</i> | Intoxilyzer® Serial Number <i>80-002667</i> | |
| Gas Lot Number <i>11611080 A2</i> | Gas Cylinder Number <i>133</i> | Gas Expiration Date <i>06/01/2013</i> |

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0. 081 AC
 - B. 0. 081 AC
 - C. 0. 081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Breath Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

| | |
|---|--|
| Chemical Test Operator Signature <i>Mark W. McDonald</i> | Date <i>09/27/2013</i> ^{2011 DK} |
| Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i> | Date <i>23 Feb 2012</i> |

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = VALL 8164.13.00 06/09
09/27/2011 09:52

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 09:53 |
| 02 Std. Gas | 0.081 | 09:54 |
| 03 Room Air | 0.000 | 09:54 |
| 04 Std. Gas | 0.081 | 09:55 |
| 05 Room Air | 0.000 | 09:55 |
| 06 Std. Gas | 0.081 | 09:55 |
| 07 Room Air | 0.000 | 09:56 |

Lot No = 11611080A2
Cyl No = 133
Exp Date = 06/01/2013
County = 02 Oper No. = 119502



Operator Signature
MARK W. MCDONALD

Remarks: