

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004203

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5144</u> Lot No. <u> </u> Rep AC <u>0.021</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3847</u> Lot No. <u> </u> Rep AC <u>0.294</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3452</u> Lot No. <u> </u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>15410080A1</u>	Cyl No. <u>12</u>	Exp. Date <u>07/01/2012</u>
Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.080</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

L. Tank pressure, Level 3 Func D Sub G P: 701 psi Reg 700 psi

M. Remarks/Maintenance record: Annual inspection w/software 8164.13

28 Feb 2011
Date

Deb Kashur
Field Inspector's Signature

28 Feb 2011
Date

Maryn Plann
Reviewed by

80-004203
p. 87 of 95
AM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
02/28/2011 14:33

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNORSTUVWXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNORSTUVWXYZ!@#%&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 15410080A1
Standard Cyl #? 12
Standard Expiration? 07/01/2012
Oper No? 777777

Flow Cal. Date: 01/28/2010
Slope 719
Intercept -711903

IR Calibration Date: 01/28/2010
 3um 9um

0th Coef(*100): -17469 -28710
1st Coef(*100): 264216 135022
2nd Coef(*100): 2778 1081
H2O adj(mg/l*10k): 430 605

***** Printer Test End *****

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Print test

80-004203
p. 88 of 95
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
02/28/2011 13:39

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:40
02 Std. Sol.	0.021	13:41
03 Room Air	0.000	13:41
04 Std. Sol.	0.021	13:42
05 Room Air	0.000	13:43
06 Std. Sol.	0.020	13:43
07 Room Air	0.000	13:44


08 Sim Temp = 34.0°C

Simul Ser No = 0000

Std Sol No = 000

County = 08

Oper No. = 777777

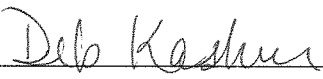

Operator Signature
DEB KASHUR

Remarks:

low AC 0.021AE

Form 106-I8000

80-004203
P. 89 of 95
JKM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
02/28/2011 13:45

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:45
02 Std. Sol.	0.294	13:46
03 Room Air	0.000	13:47
04 Std. Sol.	0.295	13:48
05 Room Air	0.000	13:48
06 Std. Sol.	0.295	13:49
07 Room Air	0.000	13:49

08 Sim Temp = 34.0°C

Simul Ser No = 0000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.294 AC

Form 106-I8000

80-004203
P. 90 of 95
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
02/28/2011 14:35

Test	AC	Time
01 Room Air	0.000	14:36
02 *Subject Test	INT*	14:36
03 Room Air	0.000	14:37

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 15410080A1
Cyl No = 12
Expiration Date = 07/01/2012
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100AC ^{acetone} ~~factor~~

Form 106-I8000

80-004203
P. 9/14/95
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
02/28/2011 14:37

Test	AC	Time
01 Diagnostic	OK	14:38
02 Room Air	RFI*	14:38
03 Room Air	0.000	14:39

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 15410080A1
Cyl No = 12
Expiration Date = 07/01/2012
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:



Form 106-I8000

80-004203
P. 92 of 95
YH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
02/28/2011 14:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:40
02 Std. Gas	0.079	14:40
03 Room Air	0.000	14:40
04 Std. Gas	0.079	14:41
05 Room Air	0.000	14:41
06 Std. Gas	0.079	14:42
07 Room Air	0.000	14:42

Lot No = 15410080A1
Cyl No = 12
Exp Date = 07/01/2012
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004203
p. 93 of 95
RM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
02/28/2011 14:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:43
02 Std. Gas	0.079	14:43
03 Room Air	0.000	14:44
04 Std. Gas	0.080	14:44
05 Room Air	0.000	14:45
06 Std. Gas	0.079	14:45
07 Room Air	0.000	14:46

Lot No = 15410080A1
Cyl No = 12
Exp Date = 07/01/2012
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC

Form 106-I8000

80-004203
p. 94 of 95
JAM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
02/28/2011 14:46

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:47
02 Std. Gas	0.080	14:47
03 Room Air	0.000	14:48
04 Std. Gas	0.079	14:48
05 Room Air	0.000	14:48
06 Std. Gas	0.079	14:49
07 Room Air	0.000	14:49

Lot No = 15410080A1
Cyl No = 12
Exp Date = 07/01/2012
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004203
P. 95 of 95
KTM