



ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) HERZIG THOMAS		
Location CARRINGTON P.D.	Intoxilyzer® Serial Number 80-003060	
Gas Lot Number 19105080A5	Gas Cylinder Number 28	Gas Expiration Date 09/05/17

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0. 080 AC
 - B. 0. 081 AC
 - C. 0. 080 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature 	Date 02/29/16 ✓
Reviewed By (Crime Laboratory Use Only) Deb Shanaver	Date 02 Mar 16

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = CARR 8164.13.00 06/09
02/29/2016 11:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:39
02 Std. Gas	0.080	11:39
03 Room Air	0.000	11:39
04 Std. Gas	0.081	11:40
05 Room Air	0.000	11:40
06 Std. Gas	0.080	11:41
07 Room Air	0.000	11:41

Lot No = 19615080A5
Cyl No = 28
Exp Date = 09/05/2017
County = 16 Oper No. = 039706


Operator Signature
THOMAS HERZIG 276 NDHP

Remarks:

Form 106-I8000

CYLINDER INSTAL 02/29/16