



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (5-2011)

Serial Number <i>80-005943</i>	Instrument Location <i>BSPD</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): *Install*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Pat Reuz</i>	Date <i>3-6-14</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashur</i>	Date <i>19 Mar 14</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Eden* Certified Date: *15 Nov. 2016*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = BSPD 8164.13.00 06/09
03/06/2014 16:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:42
02 Std. Gas	0.080	16:42
03 Room Air	0.000	16:43
04 Std. Gas	0.081	16:43
05 Room Air	0.000	16:44
06 Std. Gas	0.082	16:44
07 Room Air	0.000	16:45

Lot No = 16512080A2
Cyl No = 40
Exp Date = 08/01/2014
County = 08 Oper No. = 029710

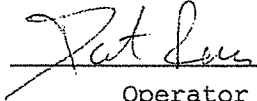

Operator Signature
PATRICK RENZ

Remarks: *New Intoxilyzer install SN 80-005943*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = BSPD 8164.13.00 06/09
03/06/2014 16:49

Test	AC	Time
01 Room Air	RFI*	16:54
02 Room Air	0.000	16:54

*Invalid Test
Inhibited - RFI

Sub Name = OTHER, TEST M
Sub DOB = 01/29/1987
Sub Sex = Male Weight = 180
Test = OTH Cit = 00900
Dr. Lic. = ND/09000
Lot No = 16512080A2
Cyl No = 40
Expiration Date = 08/01/2014
County = 08 Oper No. = 029710

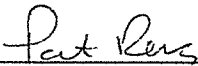

Operator Signature
PATRICK RENZ

Remarks: *Test install new Intoxilyzer SN 80-005943*

19 March 2014

Officer Pat Renz did not run the RFI test in CMS mode during his 3-06-2014 Intoxilyzer 8000 installation. However, this does not make the installation ineffective or incomplete. The RFI test Officer Renz performed shows that the RFI is working properly on this instrument.

Deb Kashur,
Forensic Scientist,
ND Crime Laboratory

Deb Kashur
19 Mar 14