

**INTOXILYZER 5000 G and 8000 INSPECTION**

Serial No.: 80-004942

Location: TOX LAB

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5190 Lot No. — Rep AC 0.019 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5113 Lot No. — Rep AC 0.285 AC ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. — Rep AC 0.10 AC + 0.05% Acetone ✓
- J. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 709352 Cyl No. 26 Exp. Date 23 Dec 2011

Test 1 0.081 Test 4 0.081 Test 7 0.080

Test 2 0.081 Test 5 0.081 Test 8 0.081

Test 3 0.081 Test 6 0.080 Test 9 0.081

Average 0.081

L. Remarks/Maintenance record: Annual Inspection

10 Mar 10  
Date

Margy Pearson  
Field Inspector's Signature

15 Mar '10  
Date

Deb Karkun  
Reviewed by

80-004942  
23/31 KR

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer              Alcohol Analyzer  
North Dakota Model 8000              SN 80-004942  
Location = TOXL                      8164.13.00 06/09  
03/10/2010                              10:03

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNOPQRSTUvwxyz!@#%\$^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNOPQRSTUvwxyz!@#%\$^&\*()\_+?

Current Instrument Setup


Data Entry Mode:              Enabled  
Start Test Sequence:          DABACABA  
Display Prelim Rslt?         Yes  
Display Third Digit?         Yes  
Inhib Printer(Y/N)?         No  
Display Volume?               No  
Disable On Memfull?         No  
# of Print Copies?             1  
Select Std (D/W/I)?          Dry  
Standard Value?               0.080  
Standard Lot #?                709352  
Standard Cyl #?                26  
Standard Expiration?         12/23/2011  
Oper No?                        107501

Flow Cal. Date:                01/20/2010  
Slope                            699  
Intercept                       -690752

IR Calibration Date:         01/20/2010  
                                            3um                      9um

-----  
0th Coef(\*100):              -21936                      -23747  
1st Coef(\*100):              262605                      132062  
2nd Coef(\*100):              1446                        1060  
H2O adj(mg/l\*10k):          551                         461

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
MARGY PEARSON

Remarks:

Print

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

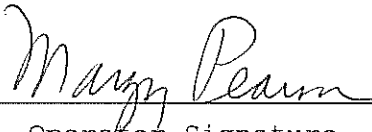
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004942  
Location = TOXL      8164.13.00 06/09  
03/10/2010      10:27

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:28
02 Std. Sol.	0.019	10:29
03 Room Air	0.000	10:30
04 Std. Sol.	0.019	10:31
05 Room Air	0.000	10:31
06 Std. Sol.	0.019	10:32
07 Room Air	0.000	10:32

08 Sim Temp = 34.0°C

Simul Ser No = 0000000  
Std Sol No = 00000  
County = 08      Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

*Low AC 0.019 AC*

Form 106-I8000

*80-004942  
25/31 KHL*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004942  
Location = TOXL      8164.13.00 06/09  
03/10/2010      10:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:45
02 Std. Sol.	0.288	10:45
03 Room Air	0.000	10:46
04 Std. Sol.	0.289	10:47
05 Room Air	0.000	10:47
06 Std. Sol.	0.290	10:48
07 Room Air	0.000	10:48

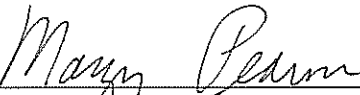
08 Sim Temp = 34.0°C

Simul Ser No = 0000000

Std Sol No = 00000

County = 08

Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

*Linearity 0.285 AC*

Form 106-18000

80-004942  
26/31 YAL

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004942  
Location = TOXL      8164.13.00 06/09  
03/10/2010      11:03

Test	AC	Time
01 Room Air	0.000	11:04
02 *Subject Test	INT*	11:04
03 Room Air	0.000	11:05

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS821456  
Simul Ser No = 0000000  
Std Sol No = 00000  
County = 08      Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

ALCOHOL + ACETONE

Form 106-I8000

80-004942  
27/31 HLL

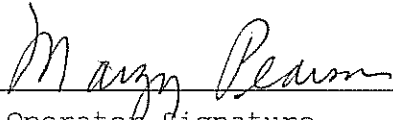
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004942  
Location = TOXL      8164.13.00 06/09  
03/10/2010      10:16

Test	AC	Time
01 Room Air	RFI*	10:16
02 Room Air	0.000	10:17

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 11/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 709352  
Cyl No = 26  
Expiration Date = 12/23/2011  
County = 08      Oper No. = 107501



Operator Signature  
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004942  
28/31 KAL

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004942  
Location = TOXL      8164.13.00 06/09  
03/10/2010      12:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:05
02 Std. Gas	0.081	12:05
03 Room Air	0.000	12:06
04 Std. Gas	0.081	12:06
05 Room Air	0.000	12:07
06 Std. Gas	0.081	12:07
07 Room Air	0.000	12:08

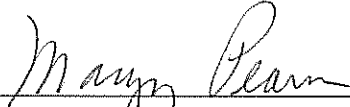
Lot No = 709352

Cyl No = 26

Exp Date = 12/23/2011

County = 08

Oper No. = 107501



Operator Signature  
MARGY PEARSON

Remarks:

*Cal check A C A*

Form 106-I8000

80-004942  
2/3/11 KHL


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004942  
Location = TOXL      8164.13.00 06/09  
03/10/2010      12:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:09
02 Std. Gas	0.081	12:09
03 Room Air	0.000	12:10
04 Std. Gas	0.081	12:10
05 Room Air	0.000	12:11
06 Std. Gas	0.080	12:11
07 Room Air	0.000	12:11

Lot No = 709352  
Cyl No = 26  
Exp Date = 12/23/2011  
County = 08      Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

*Cal Check ACA*

Form 106-I8000

*80-004942  
30/31 RM*



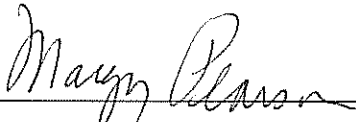
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004942  
Location = TOXL      8164.13.00 06/09  
03/10/2010      12:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:13
02 Std. Gas	0.080	12:14
03 Room Air	0.000	12:14
04 Std. Gas	0.081	12:14
05 Room Air	0.000	12:15
06 Std. Gas	0.081	12:15
07 Room Air	0.000	12:16

Lot No = 709352  
Cyl No = 26  
Exp Date = 12/23/2011  
County = 08      Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

*Cal Check ACA*

Form 106-I8000

80-004942  
3/31 KM