

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004944

Location: Tox Lab

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5190 Lot No. — Rep AC 0.019 AC ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5113 Lot No. — Rep AC 0.285 AC ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. — Rep AC 0.10^{MFP} AC + 0.05% Acetone ✓
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. 709352 Cyl No. 12 Exp. Date 23 Dec 2011

Test 1 0.080 Test 4 0.080 Test 7 0.081

Test 2 0.080 Test 5 0.080 Test 8 0.081

Test 3 0.080 Test 6 0.081 Test 9 0.081

Average 0.080 AC

L. Remarks/Maintenance record: Annual Inspection

12 March 2010
Date

Mary Ryan
Field Inspector's Signature

15 Mar 2010
Date

Deb Kashur
Reviewed by

80-004944
23/31 JFM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = TOXL 8164.13.00 06/09
03/12/2010 14:12

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:12
02 Std. Sol.	0.017	14:13
03 Room Air	0.000	14:13
04 Std. Sol.	0.018	14:14
05 Room Air	0.000	14:15
06 Std. Sol.	0.018	14:15
07 Room Air	0.000	14:16


08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 000000

County = 08

Oper No. = 107501

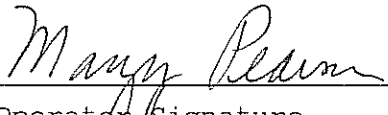
Operator Signature
MARGY PEARSON

Remarks:

Low 0.019 AC

Form 106-I8000

80-004944
25/31 HLL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = TOXL 8164.13.00 06/09
03/12/2010 14:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:26
02 Std. Sol.	0.286	14:27
03 Room Air	0.000	14:28
04 Std. Sol.	0.287	14:29
05 Room Air	0.000	14:29
06 Std. Sol.	0.289	14:30
07 Room Air	0.000	14:30


08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 000000

County = 08

Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Linearity 0.285 AC

Form 106-I8000

80-004944
26/31 AK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = TOXL 8164.13.00 06/09
03/12/2010 15:45

Test	AC	Time
01 Room Air	0.000	15:45
02 *Subject Test	INT*	15:46
03 Room Air	0.000	15:46

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 709352
Cyl No = 12
Expiration Date = 12/23/2011
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Interfer-Alc + Acetone

Form 106-I8000

80-004944
27/31 JSM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = TOXL 8164.13.00 06/09
03/12/2010 14:47

Test	AC	Time
01 Room Air	RFI*	14:47
02 Room Air	0.000	14:48

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 709352
Cyl No = 12
Expiration Date = 12/23/2011
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004944
28/31 KTL


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = TOXL 8164.13.00 06/09
03/12/2010 14:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:40
02 Std. Gas	0.080	14:41
03 Room Air	0.000	14:41
04 Std. Gas	0.080	14:42
05 Room Air	0.000	14:42
06 Std. Gas	0.080	14:43
07 Room Air	0.000	14:43

Lot No = 709352
Cyl No = 12
Exp Date = 12/23/2011
County = 08 Oper No. = 107501

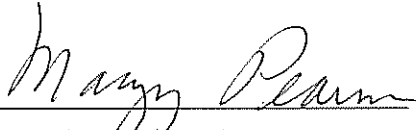

Operator Signature
MARGY PEARSON

Remarks:

ACA Calo Check

Form 106-I8000

80-004944
29/31 YAL


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = TOXL 8164.13.00 06/09
03/12/2010 14:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:49
02 Std. Gas	0.080	14:49
03 Room Air	0.000	14:50
04 Std. Gas	0.080	14:50
05 Room Air	0.000	14:51
06 Std. Gas	0.081	14:51
07 Room Air	0.000	14:52

Lot No = 709352
Cyl No = 12
Exp Date = 12/23/2011
County = 08 Oper No. = 107501

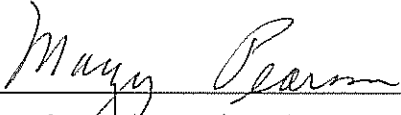

Operator Signature
MARGY PEARSON

Remarks:

ACA Cal Check

Form 106-I8000

80-004944
3/31/12


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = TOXL 8164.13.00 06/09
03/12/2010 14:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:52
02 Std. Gas	0.081	14:53
03 Room Air	0.000	14:53
04 Std. Gas	0.081	14:54
05 Room Air	0.000	14:54
06 Std. Gas	0.081	14:54
07 Room Air	0.000	14:55

Lot No = 709352
Cyl No = 12
Exp Date = 12/23/2011
County = 08 Oper No. = 107501

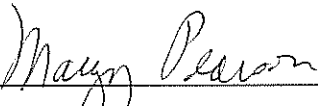

Operator Signature
MARGY PEARSON

Remarks:

ACA Cal Check

Form 106-I8000

*80-004944
3/31/11*