



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <b>80-005950</b>	Instrument Location <b>SNER</b>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <b>03-16-2015</b>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <b>23 Mar 15</b>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *[Signature]* Certified Date: **18 Nov. 2016**

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005950  
Location = SHER                    8164.13.00 06/09  
03/16/2015                            14:42

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            14214080A6  
Standard Cyl #?            60  
Standard Expiration?       07/05/2016  
Oper No?                    048909

Flow Cal. Date:            06/20/2013  
Slope                      713  
Intercept                  -723515

IR Calibration Date:       06/20/2013  
                                  3um                    9um

-----  
0th Coef(\*100):            -37081                -23816  
1st Coef(\*100):            290549                133411  
2nd Coef(\*100):            552                    1090  
H2O adj(mg/l\*10k):        683                    502

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Operator Signature  
DELBERT HOFFMANN

Remarks:    PRINT TEST

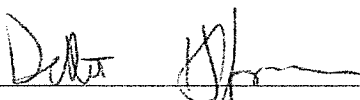
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005950  
Location = SHER      8164.13.00 06/09  
03/16/2015      14:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:44
02 Std. Gas	0.082	14:44
03 Room Air	0.000	14:45
04 Std. Gas	0.082	14:45
05 Room Air	0.000	14:46
06 Std. Gas	0.082	14:46
07 Room Air	0.000	14:47

Lot No = 14214080A6  
Cyl No = 60  
Exp Date = 07/05/2016  
County = 42      Oper No. = 048909

  
Operator Signature  
DELBERT HOFFMANN

Remarks:      ACA

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

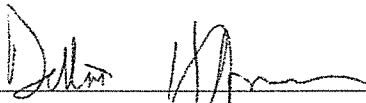
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005950  
Location = SHER      8164.13.00 06/09  
03/16/2015      15:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	15:05
02 Room Air	0.000	15:05

\*Invalid Test  
Inhibited - RFI

Lot No = 14214080A6  
Cyl No = 60  
Exp Date = 07/05/2016  
County = 42      Oper No. = 048909

  
Operator Signature  
DELBERT HOFFMANN

Remarks:

RFI TEST

Form 106-I8000