

**INTOXILYZER® 8000 INITIAL INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-006493

**A. GENERAL Setup and Checks:**

1.  Power On
  - 5 min. countdown during initial warm-up
  - 1 min. countdown to begin test after standby mode
2.  Display Intensity (Ctrl + ↑ or ↓)
3. Password Security:
  - Level 1 (Operator)     Level 2 (Field Inspector)     Level 3 (Lab Personnel)
4.  Printer Setup (Level 3,S,P)
5.  Forms Load (CMI, Inc., CoBrA Software)

**B. LEVEL 1 Setup and Checks:**

1.  Internal Printer Test and Paper Feed (Level 1,P). Attach record.
2.  External Printer Test (Level 1,P). Attach record.
3. Barcode Reader (Level 1,B):
  - Operator ID     ND License     R key: Refusal. Attach record.

**C. LEVEL 2 Setup and Checks:**

1.  Continuous Air Blank (Level 2,A)
2.  Diagnostic (Level 2,D). Attach record.
3.  Prelim Data Entry (Level 2,E)
4. Ser Number/Version Info (Level 2,V):
  - Inst Ser#: 80-006493    DSP Ver: 1362.14
  - Core Ver: 8164.13    CPLD Ver: 7

**D. LEVEL 3 Setup and Checks:**

1. DVM Monitor (Level 3,D,D): 3: 12592 s: 2 C: <sup>DS</sup>2 47.00  
9: 13079 s: 3 B: <sup>DS</sup>3 45 0  
(RFI indicator with keyed radio): 1
2. Function (F) Keys: :  F5 – Tone                             F9 – Fan  
 F6 – Green LED                             F10 – Pump  
 F7 – Red LED                                 F11 – Solenoid
3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 962 CF: 1.05
4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 206 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 49.0 I: 0.0  
B: 45.0 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 105 psi P (gauge): ~100 psi
7.  General Setup (Level 3,S,G)
8.  Configure Start Test (Level 3,S,T)
9.  Network (Level 3,S,N)

**E. TESTS (attach all test records):**

**Configure wet bath simulator standards before running tests (Level 1,S):**

1. Zero AC (Level 1,C):  
Sim ser #: DR3375 Lot #: N/A <sup>Milli- $\phi$</sup>  <sub>H<sub>2</sub>O</sub> Exp date: N/A  
Known value: 0.000 AC  
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):  
Sim ser #: DR3374 Lot #: 14240 Exp date: 10/20/16  
Known value: 0.020  
Results: 0.018 AC 0.018 AC 0.018 AC
3. High AC (Level 1,C):  
Sim ser #: DR 7111 Lot #: 201408C Exp date: 08/12/16  
Known value: 0.300  
Results: 0.305 AC 0.303 AC 0.303 AC

**Configure dry gas standard before running tests (Level 1,S):**

4. Ethanol + 0.05% Acetone (Level 1,B):  
Sim ser #: DR5144 Lot #: ICS 4 Exp date: 09/09/17  
Known value: 0.10 AC + 0.05% acetone  
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):  
Sim ser #: DR3455 Lot #: Acet 1 Exp date: 09/29/17  
Known value: 0.05% acetone  
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):  
Sim ser #: DR5189 Lot #: Met 2 Exp date: 09/29/17  
Known value: 0.10% Methanol  
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):  
Sim ser #: DR3454 Lot #: IPA 2 Exp date: 09/29/17  
Known value: 0.10% isopropanol  
 Display reads "Interferent Detect"

- 8.  Ambient Fail (Level 1,B): Display and test record read correct.
- 9.  Invalid Sample (Level 1,B): Display and test record read correct.
- 10.  Sequence Aborted (Level 1,B): Display and test record read correct.
- 11.  ABA (Level 1,B): Test record reads correct.
- 12.  Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13.  RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14.  Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15.  Deficient Samples (CMS mode): Display and test record read correct.
- 16.  Improper Sample (CMS mode): Display and test record read correct.
- 17.  Difference Too Great (CMS mode): Display and test record read correct.
- 18.  CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0802 Std Dev: 0.0006 Rel Std Dev: 0.77%

Maintenance/Repairs/Notes: Initial inspection  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

Instrument is acceptable to be used in the field.

Deb Shanover  
Field Inspector's Signature

18 Mar 2016  
Date

Charles [Signature]  
Reviewed by

18 MAR 2016  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer  
North Dakota Model 8000 SN 80-006493  
Location = TOXL 8164.13.00 06/09  
03/18/2016 09:23

\*\*\*\*\* Printer Test \*\*\*\*\*

12 char/line  
abcdefghijkl  
ABCDEFGHIJKL

16 char per line  
abcdefghijklmnop  
ABCDEFGHIJKLMN

21 chars per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMNQRSTU

24 characters per line  
abcdefghijklmnopqrstuwx  
ABCDEFGHIJKLMNQRSTUWX

32 characters per line  
abcdefghijklmnopqrstuwx123456  
ABCDEFGHIJKLMNQRSTUWXYZ123456

42 characters per line  
abcdefghijklmnopqrstuwx1234567890abcdef  
ABCDEFGHIJKLMNQRSTUWXYZ1234567890ABCDEF

Current Instrument Setup

Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On Menfull? No  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.000  
Standard Lot #? 80  
Standard Cyl #? 1234578  
Standard Expiration? 11/11/2020  
Oper No? 777777

Flow Cal. Date: 02/12/2016  
Slope: 691  
Intercept: -663384

IR Calibration Date: 02/12/2016

	3um	9um
0th Coef(*100):	-36676	-26473
1st Coef(*100):	258504	130468
2nd Coef(*100):	6151	2344
H2O adj(ng/l*10K):	821	589

\*\*\*\*\* Test End \*\*\*\*\*

Deb Shanauer  
Operator Signature  
DEB SHANAUER

Remarks:

Form 106-18000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            13:24

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        No  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            29614080A2  
Standard Cyl #?            25  
Standard Expiration?        12/05/2016  
Oper No?                    777777

Flow Cal. Date:            02/12/2016  
Slope                      691  
Intercept                  -663384

IR Calibration Date:        02/12/2016

	3um	9um
0th Coef(*100):	-36676	-26473
1st Coef(*100):	258504	130468
2nd Coef(*100):	6151	2344
H2O adj(mg/l*10k):	821	589

\*\*\*\*\* Printer Test End \*\*\*\*\*

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006493  
Location = TOXL                8164.13.00 06/09  
03/18/2016                        13:27

Test	AC	Time
01 Room Air	0.000	13:28
02 *Subject Test	REF*	13:28
03 Room Air	0.000	13:28


\*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                Weight = 150  
Test = DUI                        Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                        Oper No. = 777777



Operator Signature  
DEB SHANAVER

Remarks:

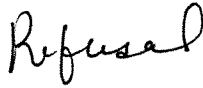

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006493  
Location = TOXL      8164.13.00 06/09  
03/18/2016      13:31

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 777777

  
\_\_\_\_\_

Operator Signature  
DEB SHANAVER

Remarks:

*Diagnostic*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006493  
Location = TOXL      8164.13.00 06/09  
03/18/2016      13:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:44
02 Std. Sol.	0.000	13:45
03 Room Air	0.000	13:46
04 Std. Sol.	0.000	13:46
05 Room Air	0.000	13:47
06 Std. Sol.	0.000	13:47
07 Room Air	0.000	13:48

08 Sim Temp = 34.0°C

Simul Ser No = DR3375  
Std Sol No = MILLIQ  
County = 08      Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB SHANAVER

Remarks:  
*Zero alcohol      0.000 AC*



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            13:50

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:50
02 Std. Sol.	0.018	13:51
03 Room Air	0.000	13:52
04 Std. Sol.	0.018	13:52
05 Room Air	0.000	13:53
06 Std. Sol.	0.018	13:54
07 Room Air	0.000	13:54

08 Sim Temp = 34.0°C

Simul Ser No = DR3374  
Std Sol No = 14240  
County = 08                            Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB SHANAVER

Remarks:

Low AC                    0.020 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006493  
Location = TOXL      8164.13.00 06/09  
03/18/2016      13:56

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:57
02 Std. Sol.	0.305	13:57
03 Room Air	0.000	13:58
04 Std. Sol.	0.303	13:59
05 Room Air	0.000	13:59
06 Std. Sol.	0.303	14:00
07 Room Air	0.000	14:01

08 Sim Temp = 34.0°C

Simul Ser No = DR7111  
Std Sol No = 201408C  
County = 08      Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB SHANAVER

Remarks:

*High AC      0.300 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            14:26

Test	AC	Time
01 Room Air	0.000	14:27
02 *Subject Test	INT*	14:27
03 Room Air	0.000	14:28

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                            Oper No. = 777777

*Deb Shanaver*

Operator Signature  
DEB SHANAVER

Remarks:

*Interferent Detected*

*0.10 AC + 0.05% acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006493  
Location = TOXL                8164.13.00 06/09  
03/18/2016                            14:33

Test	AC	Time
01 Room Air	0.000	14:34
02 *Subject Test	INT*	14:34
03 Room Air	0.000	14:35

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                            Oper No. = 777777

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:  
*Interferent detected      005 proactone*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            14:36

Test	AC	Time
01 Room Air	0.000	14:37
02 *Subject Test	INT*	14:37
03 Room Air	0.000	14:38

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                            Oper No. = 777777

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:  
*Interferent detected 0.10% methanol*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            14:39

Test	AC	Time
01 Room Air	0.000	14:39
02 *Subject Test	INT*	14:40
03 Room Air	0.000	14:40

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                            Oper No. = 777777

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:

*Interferent detected 0.10% isopropanal*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006493  
Location = TOXL      8164.13.00 06/09  
03/18/2016      14:42

Test	AC	Time
01 Room Air	AMB*	14:42
02 Room Air	0.000	14:43

\*Invalid Test  
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08      Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB SHANAVER

Remarks:

*Ambient fail*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006493  
Location = TOXL                8164.13.00 06/09  
03/18/2016                        14:51

Test	AC	Time
01 Room Air	0.000	14:51
02 Invalid Sample	X.XXX	14:52
03 Room Air	0.000	14:53

\*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                Weight = 150  
Test = DUI                        Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                        Oper No. = 777777

*Deb Shanaver*

Operator Signature  
DEB SHANAVER

Remarks:

*Invalid sample*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                      8164.13.00 06/09  
03/18/2016    14:54

Test	AC	Time
01 Room Air	0.000	14:55
02 *Subject Test	ABT*	14:55
03 Room Air	0.000	14:55

\*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                      Weight = 150  
Test = DUI    Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                      Oper No. = 777777

Deb Shanaver

Operator Signature  
DEB SHANAVER

Remarks:

*Sequence aborted*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            14:59

Test	AC	Time
01 Room Air	0.000	15:00
02 Subject Test 1	0.000	15:00
03 Room Air	0.000	15:01
04 Reported AC	0.000	15:00

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                            Oper No. = 777777



Operator Signature  
DEB SHANAVER

Remarks:

ABA

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            15:37

Test	AC	Time
01 Diagnostic	OK	15:37
02 Room Air	0.000	15:38
03 Subject Test 1	0.000	15:38
04 Room Air	0.000	15:40
05 Std. Gas	0.100*	15:41
06 Room Air	0.000	15:42

\*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:

Calibration check out of tolerance

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            15:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	15:01
02 Room Air	0.000	15:02

\*Invalid Test  
Inhibited - RFI

Lot No = 29614080A2  
Cyl No = 25  
Exp Date = 12/05/2016  
County = 08                            Oper No. = 777777

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:

RFI test

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            15:03

Test	AC	Time
01 Diagnostic	OK	15:04
02 Room Air	0.000	15:04
03 *Subject Test	0.000*	15:07
04 Room Air	0.000	15:11
05 Std. Gas	0.080	15:12
06 Room Air	0.000	15:13
07 *Subject Test	0.000*	15:16
08 Room Air	0.000	15:18


No RFI Detected

\*Deficient Sample - Value Printed was  
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                          Oper No. = 777777


20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the  
instructions displayed by the Intoxilyzer  
in conducting this test.



Operator Signature  
DEB SHANAVER

Remarks:

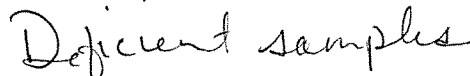

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            15:18

Test	AC	Time
01 Diagnostic	OK	15:19
02 Room Air	IPS*	15:19
03 Room Air	0.000	15:20

\*Improper Sample

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Shanaver

Operator Signature  
DEB SHANAVER

Remarks:

*Improper sample*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            15:20

Test	AC	Time
01 Diagnostic	OK	15:21
02 Room Air	0.000	15:21
03 Subject Test 1	0.037*	15:22
04 Room Air	0.000	15:24
05 Std. Gas	0.080	15:25
06 Room Air	0.000	15:27
07 Subject Test 2	0.000*	15:27
08 Room Air	0.000	15:28

\*Difference Too Great


Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                          Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
DEB SHANAVER

Remarks:

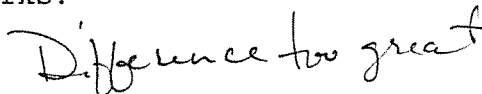

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            15:28

Test	AC	Time
01 Diagnostic	OK	15:29
02 Room Air	0.000	15:29
03 Subject Test 1	0.000	15:30
04 Room Air	0.000	15:32
05 Std. Gas	0.080	15:33
06 Room Air	0.000	15:34
07 Subject Test 2	0.000	15:35
08 Room Air	0.000	15:35
09 Reported AC	0.000	15:30

Difference OK  
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 25  
Expiration Date = 12/05/2016  
County = 08                          Oper No. = 777777

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the  
instructions displayed by the Intoxilyzer  
in conducting this test.



Operator Signature  
DEB SHANAVER

Remarks:

CMS



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006493  
Location = TOXL                    8164.13.00 06/09  
03/18/2016                            14:05

Test	AC	Time
01 Room Air	0.000	14:05
02 Std. Gas	0.080	14:06
03 Room Air	0.000	14:06
04 Std. Gas	0.080	14:07
05 Room Air	0.000	14:07
06 Std. Gas	0.080	14:08
07 Room Air	0.000	14:08
08 Std. Gas	0.080	14:09
09 Room Air	0.000	14:09
10 Std. Gas	0.079	14:09
11 Room Air	0.000	14:10
12 Std. Gas	0.080	14:10
13 Room Air	0.000	14:11
14 Std. Gas	0.079	14:11
15 Room Air	0.000	14:12
16 Std. Gas	0.080	14:12
17 Room Air	0.000	14:13
18 Std. Gas	0.080	14:13
19 Room Air	0.000	14:14
20 Std. Gas	0.081	14:14
21 Room Air	0.000	14:15
22 Std. Gas	0.081	14:15
23 Room Air	0.000	14:15
24 Std. Gas	0.080	14:16
25 Room Air	0.000	14:16
26 Std. Gas	0.081	14:17
27 Room Air	0.000	14:17
28 Std. Gas	0.081	14:18
29 Room Air	0.000	14:18
30 Std. Gas	0.080	14:19
31 Room Air	0.000	14:19
32 Std. Gas	0.080	14:19
33 Room Air	0.000	14:20
34 Std. Gas	0.080	14:20
35 Room Air	0.000	14:21
36 Std. Gas	0.081	14:21
37 Room Air	0.000	14:22
38 Std. Gas	0.080	14:22
39 Room Air	0.000	14:23
40 Std. Gas	0.081	14:23
41 Room Air	0.000	14:24
42 Cal Check Stats:		
43 Average	0.0802	
44 Std Dev	0.0006	
45 Rel Std Dev	0.77%	

Stability test  
Deb Shanaver  
p. 1 of 2

Operator Signature  
DEB SHANAVER

Remarks:

Lot # 29614080A2 cyl # 25 exp 12/5/2016

Form 106-I8000

Stability test  
Deb Shanaver  
P. 2 of 2