



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (1-2010)

Serial Number: <u>80-004935</u>	Instrument Location: <u>Harv</u>
Reason for Install/Repair:	
<input type="checkbox"/> Install After Lab Repair/Inspection <input checked="" type="checkbox"/> Other (Specify) <u>initial installation (new)</u>	

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: _____

N/A

Tyler Wolfe
 Field Inspector Signature

3/19/10
 Date

Deb Kashner
 Reviewed By

29 Mar 2010
 Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = HARV 8164.13.00 06/09
03/19/2010 10:54

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer (Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 709352
Standard Cyl #? 45
Standard Expiration? 12/23/2011
Oper No? 060901

Flow Cal. Date: 01/20/2010
Slope 696
Intercept -664812

IR Calibration Date: 02/10/2010
3um 9um

	3um	9um
0th Coef(*100):	-4573	-2516
1st Coef(*100):	251841	123171
2nd Coef(*100):	2261	1381
H2O adj(mg/l*10k):	529	468

***** Printer Test End *****

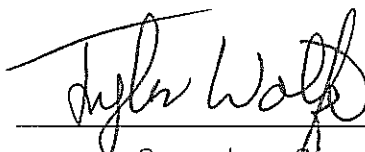

Operator Signature
TYLER A. WOLFE

Remarks:

Print test


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = HARV 8164.13.00 06/09
03/19/2010 10:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:51
02 Std. Gas	0.077	10:51
03 Room Air	0.000	10:52
04 Std. Gas	0.077	10:52
05 Room Air	0.000	10:53
06 Std. Gas	0.078	10:53
07 Room Air	0.000	10:53

Lot No = 709352
Cyl No = 45
Exp Date = 12/23/2011
County = 52 Oper No. = 060901



Operator Signature
TYLER A. WOLFE

Remarks:

ACA Form 104-G

Form 106-I8000

80-004935
37/38 AL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = HARV 8164.13.00 06/09
03/19/2010 11:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:03
02 Std. Gas	0.077	11:03
03 Room Air	RFI*	11:04
04 Room Air	0.000	11:04

*Invalid Test
Inhibited - RFI

Lot No = 709352
Cyl No = 45
Exp Date = 12/23/2011
County = 52 Oper No. = 060901



Operator Signature
TYLER A. WOLFE

Remarks:

RFI

Form 106-I8000

80-004935
3/18/10 JAH