



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (1-2010)

Serial Number: <u>80-002667</u>	Instrument Location: <u>6 FAF</u>
Reason for Install/Repair:	
<input checked="" type="checkbox"/> Install After Lab Repair/Inspection <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level 1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: _____

Will Pitt
 Field Inspector Signature

21 Mar 11
 Date

Deb Kashur
 Reviewed By

14 Apr 2011
 Date

80-002667
 P. 92 of 95
 YFM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = GFAF 8164.13.00 06/09
03/21/2011 11:00

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659358
Standard Cyl #? 39
Standard Expiration? 09/03/2011
Oper No? 120011

Flow Cal. Date: 09/10/2007
Slope 693
Intercept -570251

IR Calibration Date: 09/10/2007

	3um	9um
0th Coef(*100):	-21400	-13991
1st Coef(*100):	268177	135590
2nd Coef(*100):	1202	658
H2O adj(mg/l*10k):	369	165

***** Printer Test End *****

William R. Petty

Operator Signature
WILLIAM R. PETTY

Remarks: Re-install


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = GFAP 8164.13.00 06/09
03/21/2011 13:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:21
02 Std. Gas	0.077	13:21
03 Room Air	0.000	13:22
04 Std. Gas	0.077	13:22
05 Room Air	0.000	13:23
06 Std. Gas	0.078	13:23
07 Room Air	0.000	13:24

Lot No = 05911080A1
Cyl No = 10
Exp Date = 02/01/2013
County = 18 Oper No. = 120011

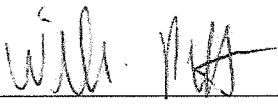

Operator Signature
WILLIAM R. PETTY

Remarks: *Re-install*

Form 106-I8000

*80-002667
P. 94 of 95
RM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = GFAP 8164.13.00 06/09
03/21/2011 14:32

Test	AC	Time
01 Room Air	RFI*	14:33
02 Room Air	RFI*	14:33

*Invalid Test
Inhibited - RFI

Sub Name = RFI TEST, N/A NA
Sub DOB = 10/09/1959
Sub Sex = Male Weight = 175
Test = DUI Cit = N/A
Dr. Lic. = ND/NA
Lot No = 05911080A1
Cyl No = 10
Expiration Date = 02/01/2013
County = 18 Oper No. = 120011

Will Petty

Operator Signature
WILLIAM R. PETTY

Remarks:

Re-install RFI Test

Form 106-I8000

80-002667
P. 95 of 95
JAM