

**INTOXILYZER® 8000 INITIAL INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-006506

**A. GENERAL Setup and Checks:**

1.  Power On
  - 5 min. countdown during initial warm-up
  - 1 min. countdown to begin test after standby mode
2.  Display Intensity (Ctrl + ↑ or ↓)
3. Password Security:
  - Level 1 (Operator)    Level 2 (Field Inspector)    Level 3 (Lab Personnel)
4.  Printer Setup (Level 3,S,P)
5.  Forms Load (CMI, Inc., CoBrA Software)

**B. LEVEL 1 Setup and Checks:**

1.  Internal Printer Test and Paper Feed (Level 1,P). Attach record.
2.  External Printer Test (Level 1,P). Attach record.
3. Barcode Reader (Level 1,B):
  - Operator ID    ND License    R key: Refusal. Attach record.

**C. LEVEL 2 Setup and Checks:**

1.  Continuous Air Blank (Level 2,A)
2.  Diagnostic (Level 2,D). Attach record.
3.  Prelim Data Entry (Level 2,E)
4. Ser Number/Version Info (Level 2,V):
  - Inst Ser#: 80-006506   DSP Ver: 1362.14
  - Core Ver: 8164.13   CPLD Ver: 7

**D. LEVEL 3 Setup and Checks:**

1. DVM Monitor (Level 3,D,D): 3: 12680   s: 1   C: 47.00  
9: 13348   s: 1   B: 45   0  
(RFI indicator with keyed radio): 1
2. Function (F) Keys: :  F5 – Tone                       F9 - Fan  
 F6 – Green LED                       F10 – Pump  
 F7 – Red LED                          F11 – Solenoid
3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 949   CF: 1.06
4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 190   F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 49.5 I: 0.0  
B: 45.0 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 994 psi P (gauge): ≈ 1000 psi
7.  General Setup (Level 3,S,G)
8.  Configure Start Test (Level 3,S,T)
9.  Network (Level 3,S,N)

**E. TESTS (attach all test records):**

**Configure wet bath simulator standards before running tests (Level 1,S):**

1. Zero AC (Level 1,C):  
Sim ser #: DR3375 Lot #: Milli-Q H<sub>2</sub>O Exp date: N/A  
Known value: 0.000 AC  
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):  
Sim ser #: DR3374 Lot #: 14240 Exp date: 10/20/16  
Known value: 0.020  
Results: 0.019 AC 0.020 AC 0.020 AC
3. High AC (Level 1,C):  
Sim ser #: DR7111 Lot #: 201408c Exp date: 08/12/16  
Known value: 0.300  
Results: 0.294 AC 0.294 AC 0.295 AC

**Configure dry gas standard before running tests (Level 1,S):**

4. Ethanol + 0.05% Acetone (Level 1,B):  
Sim ser #: DR5144 Lot #: ICS4 Exp date: 09/09/17  
Known value: 0.10 AC + 0.05% acetone  
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):  
Sim ser #: DR3455 Lot #: Acet Exp date: 09/29/17  
Known value: 0.05% acetone  
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):  
Sim ser #: DR5189 Lot #: Met 2 Exp date: 09/29/17  
Known value: 0.10% Methanol  
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):  
Sim ser #: DR3454 Lot #: IPA 2 Exp date: 09/29/17  
Known value: 0.10% isopropanol  
 Display reads "Interferent Detect"

- 8.  Ambient Fail (Level 1,B): Display and test record read correct.
- 9.  Invalid Sample (Level 1,B): Display and test record read correct.
- 10.  Sequence Aborted (Level 1,B): Display and test record read correct.
- 11.  ABA (Level 1,B): Test record reads correct.
- 12.  Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13.  RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14.  Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15.  Deficient Samples (CMS mode): Display and test record read correct.
- 16.  Improper Sample (CMS mode): Display and test record read correct.
- 17.  Difference Too Great (CMS mode): Display and test record read correct.
- 18.  CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0808 Std Dev: 0.0004 Rel Std Dev: 0.56%

Maintenance/Repairs/Notes: Initial inspection  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

Instrument is acceptable to be used in the field.

Deb Shanaver  
Field Inspector's Signature

22 Mar 2016  
Date

Charles E. Elder  
Reviewed by

22 MAR 2016  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer  
North Dakota Model 8000 SN 80-006506  
Location = TOXL 8164.13.00 06/09  
03/21/2016 11:30

\*\*\*\*\* Printer Test \*\*\*\*\*

12 char/line  
abcdefghijkl  
ABCDEFGHIJKL

16 char per line  
abcdefghijklmnop  
ABCDEFGHIJKLMN

21 chars per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMNQRSTU

24 characters per line  
abcdefghijklmnopqrstuvw  
ABCDEFGHIJKLMNQRSTUW

32 characters per line  
abcdefghijklmnopqrstuvwxy123456  
ABCDEFGHIJKLMNQRSTUWXY123456

42 characters per line  
abcdefghijklmnopqrstuvwxy1234567890abcdef  
ABCDEFGHIJKLMNQRSTUWXY1234567890ABCDEF

Current Instrument Setup  
Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On Menfull? No  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 80  
Standard Cyl #? 123467  
Standard Expiration? 11/11/2020  
Oper No? 777777

Flow Cal. Date: 08/20/2015  
Slope: 657  
Intercept: -627881

IR Calibration Date: 08/20/2015  
3um 9um

0th Coef(\*100): -22470 -18919  
1st Coef(\*100): 266505 136109  
2nd Coef(\*100): 3702 1644  
H2O adj(mg/l)\*10K): 709 510

\*\*\*\*\* Test End \*\*\*\*\*



Operator Signature  
DEB SHANAVER

Remarks:

Print test

Form 106-18000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006506  
Location = TOXL                    8164.13.00 06/09  
03/21/2016                            11:45

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-|=|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

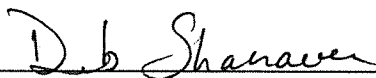
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        No  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            80  
Standard Cyl #?            123467  
Standard Expiration?        11/11/2020  
Oper No?                    777777

Flow Cal. Date:            08/20/2015  
Slope                        657  
Intercept                    -627881

IR Calibration Date:        08/20/2015

	3um	9um
0th Coef(*100):	-22470	-18919
1st Coef(*100):	266505	136109
2nd Coef(*100):	3702	1644
H2O adj(mg/l*10k):	709	510

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Operator Signature  
DEB SHANAVER

Remarks:

Print test


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006506  
Location = TOXL                8164.13.00 06/09  
03/21/2016                        11:45

Test	AC	Time
01 Room Air	0.000	11:46
02 *Subject Test	REF*	11:46
03 Room Air	0.000	11:46

\*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                Weight = 150  
Test = DUI                        Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 80  
Cyl No = 123467  
Expiration Date = 11/11/2020  
County = 08                        Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB SHANAVER

Remarks:

*Refusal*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006506  
Location = TOXL      8164.13.00 06/09  
03/21/2016      11:47

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 777777

*Deb Shanaver*

---

Operator Signature  
DEB SHANAVER

Remarks:

*Diagnostic*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006506  
Location = TOXL                    8164.13.00 06/09  
03/22/2016                            08:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:44
02 Std. Sol.	0.000	08:44
03 Room Air	0.000	08:45
04 Std. Sol.	0.000	08:46
05 Room Air	0.000	08:46
06 Std. Sol.	0.000	08:47
07 Room Air	0.000	08:47

08 Sim Temp = 34.0°C

Simul Ser No = DR3375  
Std Sol No = MILLIQ  
County = 08                            Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB SHANAVER

Remarks:

Zero alcohol            0.000 AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006506  
Location = TOXL      8164.13.00 06/09  
03/22/2016      08:49

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:50
02 Std. Sol.	0.019	08:50
03 Room Air	0.000	08:51
04 Std. Sol.	0.020	08:52
05 Room Air	0.000	08:52
06 Std. Sol.	0.020	08:53
07 Room Air	0.000	08:53

08 Sim Temp = 34.0°C

Simul Ser No = DR3374  
Std Sol No = 14240  
County = 08      Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB SHANAVER

Remarks:

*low Ac      0.020 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006506  
Location = TOXL      8164.13.00 06/09  
03/22/2016      09:03

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:08
02 Std. Sol.	0.294	09:11
03 Room Air	0.000	09:11
04 Std. Sol.	0.294	09:12
05 Room Air	0.000	09:13
06 Std. Sol.	0.295	09:13
07 Room Air	0.000	09:14

08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 201408C

County = 08

Oper No. = 777777

Deb Shanaver

Operator Signature  
DEB SHANAVER

Remarks:

High AC

0.300AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006506  
Location = TOXL                    8164.13.00 06/09  
03/22/2016                            09:28

Test	AC	Time
01 Diagnostic	OK	09:29
02 Room Air	0.000	09:29
03 *Subject Test	INT*	09:30
04 Room Air	0.000	09:30

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:

*Interferent detected      0.10 AC + 0.05% acetone*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006506  
Location = TOXL                8164.13.00 06/09  
03/22/2016                      09:31

Test	AC	Time
01 Diagnostic	OK	09:32
02 Room Air	0.000	09:33
03 *Subject Test	INT*	09:33
04 Room Air	0.000	09:34

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                          Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
DEB SHANAVER

Remarks:

*Interferent detected 0.05% acetone*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006506  
Location = TOXL                    8164.13.00 06/09  
03/22/2016                            09:37

Test	AC	Time
01 Diagnostic	OK	09:38
02 Room Air	0.000	09:38
03 *Subject Test	INT*	09:40
04 Room Air	0.000	09:40

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

*Deb Shanaver*

Operator Signature  
DEB SHANAVER

Remarks:

*Interferent detected            0.10% methanol*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000         SN 80-006506  
Location = TOXL                 8164.13.00 06/09  
03/22/2016                         09:41

Test	AC	Time
01 Diagnostic	OK	09:43
02 Room Air	0.000	09:43
03 *Subject Test	INT*	09:44
04 Room Air	0.000	09:44

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                     Weight = 150  
Test = DUI                             Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                           Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
DEB SHANAVER

Remarks:

*Interferent detected 0.10% IPA*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006506  
Location = TOXL                      8164.13.00 06/09  
03/22/2016                                      13:02

Test	AC	Time
01 Room Air	AMB*	13:02
02 Room Air	PUR**	13:03

\*Invalid Test  
  Check Ambient Conditions  
\*\*Purge Fail

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                      Weight = 150  
Test = DUI                              Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                              Oper No. = 777777

*Deb Shanaver*

Operator Signature  
DEB SHANAVER

Remarks:

*Ambient fail (plus Purge fail)*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006506  
Location = TOXL                    8164.13.00 06/09  
03/22/2016                            13:06

Test	AC	Time
01 Room Air	0.000	13:07
02 Invalid Sample	X.XXX	13:07
03 Room Air	0.000	13:08

\*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                            Oper No. = 777777

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:

*Invalid sample - mouth alcohol*



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006506  
Location = TOXL                    8164.13.00 06/09  
03/22/2016                            13:23

Test	AC	Time
01 Room Air	0.000	13:24
02 *Subject Test	ABT*	13:24
03 Room Air	0.000	13:25

\*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                            Oper No. = 777777

*Deb Shanaver*

Operator Signature  
DEB SHANAVER

Remarks:

*Sequence aborted*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006506  
Location = TOXL                    8164.13.00 06/09  
03/22/2016                            13:21

Test	AC	Time
01 Room Air	0.000	13:22
02 Subject Test 1	0.000	13:22
03 Room Air	0.000	13:22
04 Reported AC	0.000	13:22

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                          Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB SHANAVER

Remarks:

ABA fest

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-006506  
Location = TOXL                              8164.13.00 06/09  
03/22/2016    09:19

Test	AC	Time
01 Diagnostic	OK	09:20
02 Room Air	0.000	09:21
03 Subject Test 1	0.000	09:21
04 Room Air	0.000	09:23
05 Std. Gas	0.100*	09:24
06 Room Air	0.000	09:25

\*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                              Weight = 150  
Test = DUI    Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                                      Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
Operator Signature  
DEB SHANAVER

Remarks:

*Calibration check out of tolerance*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006506  
Location = TOXL      8164.13.00 06/09  
03/22/2016      13:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:27
02 Std. Gas	RFI*	13:27
03 Room Air	0.000	13:28

\*Invalid Test  
Inhibited - RFI

Lot No = 19615080A5  
Cyl No = 26  
Exp Date = 09/05/2017  
County = 08      Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB SHANAVER

Remarks:

RFI test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006506  
Location = TOXL                    8164.13.00 06/09  
03/22/2016                            13:28

Test	AC	Time
01 Diagnostic	OK	13:29
02 Room Air	0.000	13:29
03 *Subject Test	0.000*	13:32
04 Room Air	0.000	13:34
05 Std. Gas	0.080	13:36
06 Room Air	0.000	13:37
07 *Subject Test	0.000*	13:40
08 Room Air	0.000	13:42

No RFI Detected

\*Deficient Sample - Value Printed was  
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                          Oper No. = 777777

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the  
instructions displayed by the Intoxilyzer  
in conducting this test.

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:

Deficient samples

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006506  
Location = TOXL                    8164.13.00 06/09  
03/22/2016                            13:43

Test	AC	Time
01 Diagnostic	OK	13:44
02 Room Air	IPS*	13:44
03 Room Air	0.000	13:44

\*Improper Sample

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
\_\_\_\_\_

Operator Signature  
DEB SHANAVER

Remarks:

*Improper sample*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006506  
Location = TOXL                    8164.13.00 06/09  
03/22/2016                            13:46

Test	AC	Time
01 Diagnostic	OK	13:46
02 Room Air	0.000	13:47
03 Subject Test 1	0.039*	13:47
04 Room Air	0.000	13:50
05 Std. Gas	0.080	13:51
06 Room Air	0.000	13:52
07 Subject Test 2	0.000*	13:52
08 Room Air	0.000	13:53

\*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
\_\_\_\_\_  
Operator Signature  
DEB SHANAVER

Remarks:

*Difference too great*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006506  
Location = TOXL                      8164.13.00 06/09  
03/22/2016                              13:54

Test	AC	Time
01 Diagnostic	OK	13:55
02 Room Air	0.000	13:56
03 Subject Test 1	0.000	13:56
04 Room Air	0.000	13:58
05 Std. Gas	0.080	13:59
06 Room Air	0.000	14:01
07 Subject Test 2	0.000	14:01
08 Room Air	0.000	14:02
09 Reported AC	0.000	13:56

Difference OK  
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                      Weight = 150  
Test = DUI                              Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 19615080A5  
Cyl No = 26  
Expiration Date = 09/05/2017  
County = 08                              Oper No. = 777777

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
DEB SHANAVER

Remarks:

CMS



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006506  
Location = TOXL                    8164.13.00 06/09  
03/21/2016                            11:54

Test	AC	Time
01 Room Air	0.000	11:55
02 Std. Gas	0.080	11:55
03 Room Air	0.000	11:56
04 Std. Gas	0.080	11:56
05 Room Air	0.000	11:57
06 Std. Gas	0.081	11:57
07 Room Air	0.000	11:58
08 Std. Gas	0.081	11:58
09 Room Air	0.000	11:58
10 Std. Gas	0.081	11:59
11 Room Air	0.000	11:59
12 Std. Gas	0.080	12:00
13 Room Air	0.000	12:00
14 Std. Gas	0.081	12:01
15 Room Air	0.000	12:01
16 Std. Gas	0.080	12:02
17 Room Air	0.000	12:02
18 Std. Gas	0.081	12:03
19 Room Air	0.000	12:03
20 Std. Gas	0.081	12:03
21 Room Air	0.000	12:04
22 Std. Gas	0.081	12:04
23 Room Air	0.000	12:05
24 Std. Gas	0.080	12:05
25 Room Air	0.000	12:06
26 Std. Gas	0.081	12:06
27 Room Air	0.000	12:07
28 Std. Gas	0.081	12:07
29 Room Air	0.000	12:08
30 Std. Gas	0.081	12:08
31 Room Air	0.000	12:09
32 Std. Gas	0.081	12:09
33 Room Air	0.000	12:10
34 Std. Gas	0.081	12:10
35 Room Air	0.000	12:11
36 Std. Gas	0.081	12:11
37 Room Air	0.000	12:12
38 Std. Gas	0.081	12:12
39 Room Air	0.000	12:12
40 Std. Gas	0.081	12:13
41 Room Air	0.000	12:13
42 Cal Check Stats:		
43 Average	0.0808	
44 Std Dev	0.0004	
45 Rel Std Dev	0.56%	

Stability Test  
Deb Shanaver  
p.1 of 2

Operator Signature  
DEB SHANAVER

Remarks:

Lot #: 19615080A5

Cyl #: 26

Exp: 09/05/2017

Form 106-I8000

Stability test  
Db Shanaver  
p. 2 of 2