

INTOXILYZER® 8000 INITIAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006494

A. GENERAL Setup and Checks:

1. Power On
 - 5 min. countdown during initial warm-up
 - 1 min. countdown to begin test after standby mode
2. Display Intensity (Ctrl + ↑ or ↓)
3. Password Security:
 - Level 1 (Operator) Level 2 (Field Inspector) Level 3 (Lab Personnel)
4. Printer Setup (Level 3,S,P)
5. Forms Load (CMI, Inc., CoBrA Software)

B. LEVEL 1 Setup and Checks:

1. Internal Printer Test and Paper Feed (Level 1,P). Attach record.
2. External Printer Test (Level 1,P). Attach record.
3. Barcode Reader (Level 1,B):
 - Operator ID AND License R key: Refusal. Attach record.

C. LEVEL 2 Setup and Checks:

1. Continuous Air Blank (Level 2,A)
2. Diagnostic (Level 2,D). Attach record.
3. Prelim Data Entry (Level 2,E)
4. Ser Number/Version Info (Level 2,V):
 - Inst Ser#: 80-006494 DSP Ver: 1362.14
 - Core Ver: 8164.13 CPLD Ver: 7

D. LEVEL 3 Setup and Checks:

1. DVM Monitor (Level 3,D,D): 3: 12479 s: 0 C: 47.00
9: 13242 s: 1 B: 44 0
(RFI indicator with keyed radio): 1
2. Function (F) Keys : F5 – Tone F9 - Fan
 F6 – Green LED F10 – Pump
 F7 – Red LED F11 – Solenoid
3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 952 CF: 1.06
4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 248 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 47.5 I: 0.0
B: 44.9 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 999 psi P (gauge): ~1000 psi
7. General Setup (Level 3,S,G)
8. Configure Start Test (Level 3,S,T)
9. Network (Level 3,S,N)

E. TESTS (attach all test records):

Configure wet bath simulator standards before running tests (Level 1,S):

1. Zero AC (Level 1,C):
Sim ser #: MP3067 Lot #: MILLI Q H₂O Exp date: NA
Known value: 0.000 AC
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):
Sim ser #: MP3066 Lot #: 15160 Exp date: 7.14.17
Known value: 0.020 AC
Results: 0.018 AC 0.018 AC 0.019 AC
3. High AC (Level 1,C):
Sim ser #: MP3068 Lot #: 14102 Exp date: 4.23.16
Known value: 0.400 AC
Results: 0.410 AC 0.410 AC 0.410 AC

Configure dry gas standard before running tests (Level 1,S):

4. Ethanol + 0.05% Acetone (Level 1,B):
Sim ser #: DR5135 Lot #: ICS 4 Exp date: 9.09.17
Known value: 0.10 AC + 0.05% ACETONE
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):
Sim ser #: DR5143 Lot #: ACE 2 Exp date: 9.29.17
Known value: 0.05% ACETONE
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):
Sim ser #: DR3377 Lot #: MET 2 Exp date: 9.29.17
Known value: 0.10% METHANOL
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):
Sim ser #: DR3453 Lot #: IPA 2 Exp date: 9.29.17
Known value: 0.10% ISOPROPANOL
 Display reads "Interferent Detect"

- 8. Ambient Fail (Level 1,B): Display and test record read correct.
- 9. Invalid Sample (Level 1,B): Display and test record read correct.
- 10. Sequence Aborted (Level 1,B): Display and test record read correct.
- 11. ABA (Level 1,B): Test record reads correct.
- 12. Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13. RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14. Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15. Deficient Samples (CMS mode): Display and test record read correct.
- 16. Improper Sample (CMS mode): Display and test record read correct.
- 17. Difference Too Great (CMS mode): Display and test record read correct.
- 18. CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0803 Std Dev: 0.0006 Rel Std Dev: 0.80%

Maintenance/Repairs/Notes: _____

Instrument is acceptable to be used in the field.

Charles E. Edr
Field Inspector's Signature

24 MAR. 2016
Date

Deb Shanaver
Reviewed by

24 Mar 2016
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 00-006494
Location = TOXL 8164.13.00 05/09
03/24/2016 10:35

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMNQP

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNQPQRSTU

24 characters per line
abcdefghijklmnopqrstuwx
ABCDEFGHIJKLMNQPQRSTUWX

32 characters per line
abcdefghijklmnopqrstuwx123456
ABCDEFGHIJKLMNQPQRSTUWXYZ123456

42 characters per line
abcdefghijklmnopqrstuwx1234567890abcdef
ABCDEFGHIJKLMNQPQRSTUWXYZ1234567890abcdef

Printer Configuration

Auto Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On MemFull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 2346
Standard Cyl #? 12356
Standard Expiration? 11/11/2020
Oper No? 666666

Flow Cal. Date: 08/18/2015
Slope 655
Intercept -624238

IR Calibration Date: 08/18/2015

	3um	9um
0th Coef(*100):	-24880	-16926
1st Coef(*100):	256511	140602
2nd Coef(*100):	4127	1722
H2O adj(mg/l*10K):	783	426

***** Test End *****



Operator Signature
CHARLES EDER

Remarks: **INTERNAL
PRINT
TEST**

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 10:38

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 29614080A2
Standard Cyl #? 6
Standard Expiration? 12/05/2016
Oper No? 666666

Flow Cal. Date: 08/18/2015
Slope 655
Intercept -624238

IR Calibration Date: 08/18/2015
 3um 9um

0th Coef(*100): -24880 -16926
1st Coef(*100): 256511 140602
2nd Coef(*100): 4127 1722
H2O adj(mg/l*10k): 783 426

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

EXTERNAL PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 10:43

Test	AC	Time
01 Room Air	0.000	10:44
02 *Subject Test	REF*	10:44
03 Room Air	0.000	10:45

*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = REFUSAL
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: **REFUSAL**

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 10:46

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

DIAGNOSTIC
TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 11:59

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:59
02 Std. Sol.	0.000	12:00
03 Room Air	0.000	12:01
04 Std. Sol.	0.000	12:01
05 Room Air	0.000	12:02
06 Std. Sol.	0.000	12:03
07 Room Air	0.000	12:03

08 Sim Temp = 34.0°C

Simul Ser No = MP3067
Std Sol No = MILLIQ H2O
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

ZERO AC
0.000 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 12:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:05
02 Std. Sol.	0.018	12:06
03 Room Air	0.000	12:06
04 Std. Sol.	0.018	12:07
05 Room Air	0.000	12:07
06 Std. Sol.	0.019	12:08
07 Room Air	0.000	12:09

08 Sim Temp = 34.0°C

Simul Ser No = MP3066

Std Sol No = 15160

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 12:10

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:11
02 Std. Sol.	0.410	12:12
03 Room Air	0.000	12:12
04 Std. Sol.	0.410	12:13
05 Room Air	0.000	12:14
06 Std. Sol.	0.410	12:14
07 Room Air	0.000	12:15

08 Sim Temp = 34.0°C

Simul Ser No = MP3068
Std Sol No = 14102
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: HIGH AC
 0.400 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 12:17

Test	AC	Time
01 Room Air	0.000	12:18
02 *Subject Test	INT*	12:18
03 Room Air	0.000	12:19

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *INTERFERENT*
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 12:20

Test	AC	Time
01 Room Air	0.000	12:21
02 *Subject Test	INT*	12:22
03 Room Air	0.000	12:22

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *INTERFERENT*
0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 12:23

Test	AC	Time
01 Room Air	0.000	12:24
02 *Subject Test	INT*	12:24
03 Room Air	0.000	12:25

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: 0.10% METHANOL
 INTERFERENT

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 12:26

Test	AC	Time
01 Room Air	0.000	12:27
02 *Subject Test	INT*	12:27
03 Room Air	0.000	12:28

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

INTERFERENT
0.10% ISOPROPANOL

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 12:29

Test	AC	Time
01 Room Air	AMB*	12:30
02 Room Air	0.000	12:31

*Invalid Test
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = AMBIENT FAIL
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *AMBIENT FAIL*


Intoxilyzer Test Record and Checklist
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
 North Dakota Model 8000 SN 80-006494
 Location = TOXL 8164.13.00 06/09
 03/24/2016 12:36

Test	AC	Time
01 Room Air	0.000	12:36
02 Invalid Sample	X.XXX	12:37
03 Room Air	0.000	12:38

*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT
 Sub DOB = 01/01/1982
 Sub Sex = Female Weight = 150
 Test = DUI Cit = INVALID SAMPLE
 Dr. Lic. = ND/DIS821456
 Lot No = 29614080A2
 Cyl No = 6
 Expiration Date = 12/05/2016
 County = 08 Oper No. = 666666

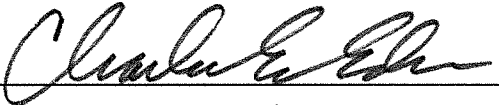

Operator Signature
 CHARLES EDER

Remarks:

INVALID SAMPLE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 12:38

Test	AC	Time
01 Room Air	0.000	12:39
02 *Subject Test	ABT*	12:39
03 Room Air	0.000	12:40

*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = SEQ ABORTED
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: SEQUENCE ABORTED

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 12:41

Test	AC	Time
01 Room Air	0.000	12:42
02 Subject Test 1	0.016	12:43
03 Room Air	0.000	12:43
04 Reported AC	0.016	12:43

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = ABA TEST
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

ABA TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 12:44

Test	AC	Time
01 Diagnostic	OK	12:45
02 Room Air	0.000	12:45
03 Subject Test 1	0.017	12:46
04 Room Air	0.000	12:48
05 Std. Gas	0.000*	12:49
06 Room Air	0.000	12:50

*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = CAL OUT OF TOL
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
OUT OF TOLERANCE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 12:54

Test	AC	Time
01 Diagnostic	OK	12:55
02 Room Air	0.000	12:56
03 Subject Test 1	0.205	12:57
04 Room Air	0.000	12:59
05 Std. Gas	0.080	13:00
06 Room Air	0.000	13:02
07 *Subject Test	RFI*	13:02
08 Room Air	0.000	13:02

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI DETECTED
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI DETECTED

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 14:00

Test	AC	Time
01 Diagnostic	OK	14:01
02 Room Air	0.000	14:02
03 *Subject Test	0.000*	14:05
04 Room Air	0.000	14:09
05 Std. Gas	0.081	14:10
06 Room Air	0.000	14:11
07 *Subject Test	0.000*	14:14
08 Room Air	0.000	14:15

No RFI Detected

*Deficient Sample - Value Printed was
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = DEFICIENT SAMP
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
CHARLES EDER

Remarks: DEFICIENT
SAMPLES

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 14:18

Test	AC	Time
01 Diagnostic	OK	14:19
02 Room Air	IPS*	14:19
03 Room Air	0.000	14:20

*Improper Sample

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = IMPROPER SAMP
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

*IMPROPER
SAMPLE*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 14:21

Test	AC	Time
01 Diagnostic	OK	14:22
02 Room Air	0.000	14:22
03 Subject Test 1	0.050*	14:22
04 Room Air	0.000	14:25
05 Std. Gas	0.080	14:26
06 Room Air	0.000	14:27
07 Subject Test 2	0.079*	14:28
08 Room Air	0.000	14:29

*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = DIFF TOO GREAT
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: **DIFFERENCE TOO GREAT**

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 14:30

Test	AC	Time
01 Diagnostic	OK	14:31
02 Room Air	0.000	14:32
03 Subject Test 1	0.294	14:32
04 Room Air	0.000	14:35
05 Std. Gas	0.080	14:36
06 Room Air	0.000	14:37
07 Subject Test 2	0.290	14:38
08 Room Air	0.000	14:39
09 Reported AC	0.290	14:38

Difference OK
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = CMS TEST
Dr. Lic. = ND/DIS821456
Lot No = 29614080A2
Cyl No = 6
Expiration Date = 12/05/2016
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

CMS TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = TOXL 8164.13.00 06/09
03/24/2016 14:42

Test	AC	Time
01 Room Air	0.000	14:43
02 Std. Gas	0.079	14:43
03 Room Air	0.000	14:44
04 Std. Gas	0.081	14:44
05 Room Air	0.000	14:44
06 Std. Gas	0.080	14:45
07 Room Air	0.000	14:45
08 Std. Gas	0.080	14:46
09 Room Air	0.000	14:46
10 Std. Gas	0.080	14:47
11 Room Air	0.000	14:47
12 Std. Gas	0.080	14:47
13 Room Air	0.000	14:48
14 Std. Gas	0.080	14:48
15 Room Air	0.000	14:49
16 Std. Gas	0.080	14:49
17 Room Air	0.000	14:50
18 Std. Gas	0.080	14:50
19 Room Air	0.000	14:51
20 Std. Gas	0.081	14:51
21 Room Air	0.000	14:52
22 Std. Gas	0.080	14:52
23 Room Air	0.000	14:52
24 Std. Gas	0.079	14:53
25 Room Air	0.000	14:53
26 Std. Gas	0.081	14:54
27 Room Air	0.000	14:54
28 Std. Gas	0.081	14:55
29 Room Air	0.000	14:55
30 Std. Gas	0.081	14:55
31 Room Air	0.000	14:56
32 Std. Gas	0.081	14:56
33 Room Air	0.000	14:57
34 Std. Gas	0.080	14:57
35 Room Air	0.000	14:58
36 Std. Gas	0.080	14:58
37 Room Air	0.000	14:59
38 Std. Gas	0.080	14:59
39 Room Air	0.000	15:00
40 Std. Gas	0.081	15:00
41 Room Air	0.000	15:00
42 Cal Check Stats:		
43 Average	0.0803	
44 Std Dev	0.0006	
45 Rel Std Dev	0.80%	

STABILITY TEST
Charles E. Ehr
Pg 1 of 2

Operator Signature
CHARLES EDER

Remarks:

LOT: 29614080A2

Exp: 12/5/2016

LYL: 006

Form 106-I8000

STABILITY TEST
Charles Eder
pg 2 of 2