

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004198

Location: TOXL

- | | Check When Done |
|---|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u> </u> Rep AC <u>0.021</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5134</u> Lot No. <u> </u> Rep AC <u>0.294</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3846</u> Lot No. <u> </u> Rep AC <u>0.10AC + acetone</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>15410080A1</u>	Cyl No. <u>12</u>	Exp. Date <u>07/01/2012</u>
Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.080</u>	Test 8 <u>0.079</u>
Test 3 <u>0.080</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

L. Tank pressure, Level 3 Func D Sub G P: 455 psi Reg 450 psi

M. Remarks/Maintenance record: Annual inspection

01 Apr 2011
Date

Delo Kashner
Field Inspector's Signature

1 Apr 2011
Date

Mary Jean
Reviewed by

80-004198
p. 121 of 129
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004198
Location = TOXL 8164.13.00 06/09
04/01/2011 09:31

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:31
02 Std. Sol.	0.019	09:32
03 Room Air	0.000	09:33
04 Std. Sol.	0.019	09:34
05 Room Air	0.000	09:34
06 Std. Sol.	0.019	09:35
07 Room Air	0.000	09:35

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.021AC

Form 106-I8000

80-004198
P.123 of 129
KTM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004198
Location = TOXL 8164.13.00 06/09
04/01/2011 09:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:40
02 Std. Sol.	0.293	09:42
03 Room Air	0.000	09:42
04 Std. Sol.	0.295	09:43
05 Room Air	0.000	09:43
06 Std. Sol.	0.295	09:44
07 Room Air	0.000	09:45

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.294AC

Form 106-I8000

80-004198
P.124 of 129
VFM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004198
Location = TOXL 8164.13.00 06/09
04/01/2011 09:50

Test	AC	Time
01 Diagnostic	OK	09:51
02 Room Air	RFI*	09:51
03 Room Air	0.000	09:52

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 15410080A1
Cyl No = 12
Expiration Date = 07/01/2012
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-004198
P. 126 of 129
KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004198
Location = TOXL 8164.13.00 06/09
04/01/2011 09:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:18
02 Std. Gas	0.079	09:19
03 Room Air	0.000	09:19
04 Std. Gas	0.079	09:20
05 Room Air	0.000	09:20
06 Std. Gas	0.080	09:20
07 Room Air	0.000	09:21

Lot No = 15410080A1
Cyl No = 12
Exp Date = 07/01/2012
County = 08 Oper No. = 777777

Deb Keshur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004198
P.127 of 129
VM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004198
Location = TOXL 8164.13.00 06/09
04/01/2011 09:23

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:24
02 Std. Gas	0.079	09:24
03 Room Air	0.000	09:25
04 Std. Gas	0.080	09:25
05 Room Air	0.000	09:25
06 Std. Gas	0.079	09:26
07 Room Air	0.000	09:26

Lot No = 15410080A1
Cyl No = 12
Exp Date = 07/01/2012
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC

Form 106-I8000

80-004198
p. 128 of 129
KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004198
Location = TOXL 8164.13.00 06/09
04/01/2011 09:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:27
02 Std. Gas	0.079	09:27
03 Room Air	0.000	09:28
04 Std. Gas	0.079	09:28
05 Room Air	0.000	09:29
06 Std. Gas	0.079	09:29
07 Room Air	0.000	09:30

Lot No = 15410080A1
Cyl No = 12
Exp Date = 07/01/2012
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004198
P.129 of 129
RM