



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (5-2011)

Serial Number <i>80-004193</i>	Instrument Location <i>Hamstead - Statyman Co.</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>12-02-11 12-08-11</i> <sup>DK</sup>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>02 Apr 12</i>



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004193  
Location = JAME      8164.13.00 06/09  
12/02/2011      13:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:31
02 Std. Gas	0.078	13:31
03 Room Air	0.000	13:32
04 Std. Gas	0.080	13:32
05 Room Air	0.000	13:33
06 Std. Gas	0.079	13:33
07 Room Air	0.000	13:34

Lot No = 05911080A1  
Cyl No = 21  
Exp Date = 02/01/2013  
County = 47      Oper No. = 100414

  
Operator Signature  
JOSH RUDE

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

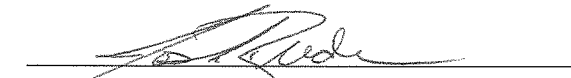
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004193  
Location = JAME      8164.13.00 06/09  
12/02/2011      13:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:35
02 Std. Gas	INT*	13:35
03 Room Air	0.000	13:35

\*Invalid Test  
Interferent Detected

Lot No = 05911080A1  
Cyl No = 21  
Exp Date = 02/01/2013  
County = 47      Oper No. = 100414

  
Operator Signature  
JOSH RUDE

Remarks:

Form 106-I8000

05 Dec 2011

On this morning, I contacted Josh Rude to ask him why he had run an Interferent Detected instead of an RFI test for the 80-004193 installation. Josh wasn't sure why it happened, but he will run another RFI and mail it to me for a complete install.

A handwritten signature in cursive script that reads "Deb Kashur".

Deb Kashur

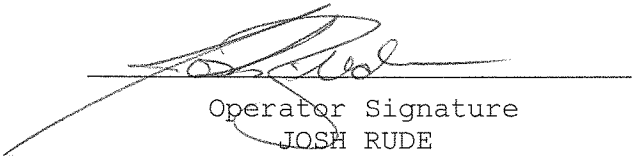
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004193  
Location = JAME      8164.13.00 06/09  
12/08/2011      13:41

Test	AC	Time
01 Diagnostic	OK	13:42
02 Room Air	RFI*	13:42
03 Room Air	RFI*	13:42

\*Invalid Test  
Inhibited - RFI

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
Operator Signature  
JOSH RUDE

Remarks:

Form 106-I8000