

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004942

Location: TOXL

- | | Check When Done |
|---|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) Sim SN <u>DR5132</u> Lot No. <u>12210</u> Rep AC <u>0.020</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) Sim SN <u>DR3803</u> Lot No. <u>12050</u> Rep AC <u>0.300</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) Sim SN <u>DR3846</u> Lot No. <u> </u> Rep AC <u>0.100 AC + acetone</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.) | |

| | | |
|---------------------------|---------------------|-----------------------------|
| Lot No. <u>2341108041</u> | Cyl No. <u>15</u> | Exp. Date <u>10/01/2013</u> |
| Test 1 <u>0.080</u> | Test 4 <u>0.081</u> | Test 7 <u>0.081</u> |
| Test 2 <u>0.081</u> | Test 5 <u>0.080</u> | Test 8 <u>0.081</u> |
| Test 3 <u>0.080</u> | Test 6 <u>0.081</u> | Test 9 <u>0.081</u> |
| Average <u>0.081</u> | | |

L. Tank pressure, Level 3 Func D Sub G P: 673 psi Reg 670 psi

M. Remarks/Maintenance record: Annual inspection

02 Apr 2013
Date

02 APR 13
Date

Deb Kashur
Field Inspector's Signature

Charles E. Edr
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = TOXL 8164.13.00 06/09
04/02/2013 15:32

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 23411080A1
Standard Cyl #? 15
Standard Expiration? 10/01/2013
Oper No? 777777

Flow Cal. Date: 08/21/2012
Slope 638
Intercept -487461

IR Calibration Date: 08/21/2012
 3um 9um

0th Coef(*100): -14539 -16931
1st Coef(*100): 257281 132094
2nd Coef(*100): 2714 1236
H2O adj(mg/l*10k): 530 436

***** Printer Test End *****

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = TOXL 8164.13.00 06/09
04/02/2013 15:41

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 15:42 |
| 02 Std. Sol. | 0.021 | 15:42 |
| 03 Room Air | 0.000 | 15:43 |
| 04 Std. Sol. | 0.021 | 15:44 |
| 05 Room Air | 0.000 | 15:44 |
| 06 Std. Sol. | 0.021 | 15:45 |
| 07 Room Air | 0.000 | 15:45 |

08 Sim Temp = 34.0°C

Simul Ser No = DR5132

Std Sol No = 12210

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = TOXL 8164.13.00 06/09
04/02/2013 15:46

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 15:47 |
| 02 Std. Sol. | 0.295 | 15:47 |
| 03 Room Air | 0.000 | 15:48 |
| 04 Std. Sol. | 0.296 | 15:49 |
| 05 Room Air | 0.000 | 15:49 |
| 06 Std. Sol. | 0.296 | 15:50 |
| 07 Room Air | 0.000 | 15:51 |

08 Sim Temp = 34.0°C

Simul Ser No = DR3803

Std Sol No = 12050

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity check 0300AE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = TOXL 8164.13.00 06/09
04/02/2013 15:54

| Test | AC | Time |
|------------------|-------|-------|
| 01 Room Air | 0.000 | 15:55 |
| 02 *Subject Test | INT* | 15:55 |
| 03 Room Air | 0.000 | 15:56 |

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 15
Expiration Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100 AC + acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = TOXL 8164.13.00 06/09
04/02/2013 15:57

| Test | AC | Time |
|---------------|-------|-------|
| 01 Diagnostic | OK | 15:57 |
| 02 Room Air | RFI* | 15:57 |
| 03 Room Air | 0.000 | 15:58 |

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 15
Expiration Date = 10/01/2013
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = TOXL 8164.13.00 06/09
04/02/2013 15:37

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 15:38 |
| 02 Std. Gas | 0.080 | 15:38 |
| 03 Room Air | 0.000 | 15:38 |
| 04 Std. Gas | 0.081 | 15:39 |
| 05 Room Air | 0.000 | 15:39 |
| 06 Std. Gas | 0.080 | 15:40 |
| 07 Room Air | 0.000 | 15:40 |

Lot No = 23411080A1

Cyl No = 15

Exp Date = 10/01/2013

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature

DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = TOXL 8164.13.00 06/09
04/02/2013 15:59

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 15:59 |
| 02 Std. Gas | 0.081 | 16:00 |
| 03 Room Air | 0.000 | 16:00 |
| 04 Std. Gas | 0.080 | 16:00 |
| 05 Room Air | 0.000 | 16:01 |
| 06 Std. Gas | 0.081 | 16:01 |
| 07 Room Air | 0.000 | 16:02 |

Lot No = 23411080A1
Cyl No = 15
Exp Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = TOXL 8164.13.00 06/09
04/02/2013 16:04

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 16:04 |
| 02 Std. Gas | 0.081 | 16:05 |
| 03 Room Air | 0.000 | 16:05 |
| 04 Std. Gas | 0.081 | 16:06 |
| 05 Room Air | 0.000 | 16:06 |
| 06 Std. Gas | 0.081 | 16:06 |
| 07 Room Air | 0.000 | 16:07 |

Lot No = 23411080A1
Cyl No = 15
Exp Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC