

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003066

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3804 Lot No. 201501A Rep AC 0.015 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5191 Lot No. 13113 Rep AC 0.400 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. ICS #2 Solution 0.10 AC + 0.05% ACE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 14214080A6 Cyl No. 072 Exp. Date 7/5/16
Test 1 0.079 Test 4 0.079 Test 7 0.080
Test 2 0.079 Test 5 0.080 Test 8 0.079
Test 3 0.080 Test 6 0.080 Test 9 0.081
Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 738 psi Reg 750 psi

K. Remarks/Maintenance record: REPLACED SIMULATOR RETURN O-RING,

ANNUAL INSPECTION

4.2.15
Date

02 Apr 15
Date

Charles E. Edler
Field Inspector's Signature

Deb Kashur
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
04/02/2015 15:46

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 14214080A6
Standard Cyl #? 72
Standard Expiration? 07/05/2016
Oper No? 666666

Flow Cal. Date: 07/29/2010
Slope 806
Intercept -895033

IR Calibration Date: 07/29/2010

	3um	9um
0th Coef(*100):	-14075	-5808
1st Coef(*100):	272113	138204
2nd Coef(*100):	1637	986
H2O adj(mg/l*10k):	289	148

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks: PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
04/02/2015 15:32

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:32
02 Std. Sol.	0.016	15:33
03 Room Air	0.000	15:34
04 Std. Sol.	0.016	15:35
05 Room Air	0.000	15:35
06 Std. Sol.	0.016	15:36
07 Room Air	0.000	15:36

08 Sim Temp = 34.0°C

Simul Ser No = DR3804

Std Sol No = 201501A

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

0.015 AC (Sim. Sol.)
Low AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
04/02/2015 15:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:41
02 Std. Sol.	0.396	15:42
03 Room Air	0.000	15:42
04 Std. Sol.	0.395	15:43
05 Room Air	0.000	15:44
06 Std. Sol.	0.395	15:44
07 Room Air	0.000	15:45

08 Sim Temp = 34.0°C

Simul Ser No = DR5191
Std Sol No = 13113
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: 0.400 AC (Sim. Sol.)
 HIGH AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
04/02/2015 16:09

Test	AC	Time
01 Room Air	0.000	16:10
02 *Subject Test	INT*	16:10
03 Room Air	0.000	16:11

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 14214080A6
Cyl No = 72
Expiration Date = 07/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

INTERFERENT CHECK
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
04/02/2015 16:06

Test	AC	Time
01 Diagnostic	OK	16:07
02 Room Air	0.000	16:07
03 *Subject Test	RFI*	16:08
04 Room Air	0.000	16:08

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 14214080A6
Cyl No = 72
Expiration Date = 07/05/2016
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
04/02/2015 15:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:48
02 Std. Gas	0.079	15:48
03 Room Air	0.000	15:49
04 Std. Gas	0.079	15:49
05 Room Air	0.000	15:49
06 Std. Gas	0.080	15:50
07 Room Air	0.000	15:50

Lot No = 14214080A6

Cyl No = 72

Exp Date = 07/05/2016

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

0.080AC (ESS)
CALIBRATION CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
04/02/2015 16:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:13
02 Std. Gas	0.079	16:13
03 Room Air	0.000	16:14
04 Std. Gas	0.080	16:14
05 Room Air	0.000	16:14
06 Std. Gas	0.080	16:15
07 Room Air	0.000	16:15

Lot No = 14214080A6
Cyl No = 72
Exp Date = 07/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: 0.080 AC (E.G.S.)
CALIBRATION CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
04/02/2015 16:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:16
02 Std. Gas	0.080	16:16
03 Room Air	0.000	16:17
04 Std. Gas	0.079	16:17
05 Room Air	0.000	16:18
06 Std. Gas	0.081	16:18
07 Room Air	0.000	16:19

Lot No = 14214080A6
Cyl No = 72
Exp Date = 07/05/2016
County = 08 Oper No. = 666666

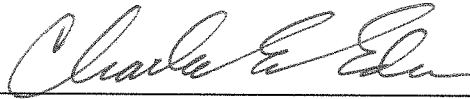

Operator Signature
CHARLES EDER

Remarks: 0.080 AC (E.G.S.)

CALIBRATION CHECK

Form 106-I8000