



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (5-2011)

Serial Number <i>80-003057</i>	Instrument Location <i>Hett - Adams County SO</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): *NONE*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>J Collins - 4100</i>	Date <i>03/01/2012</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kasher</i>	Date <i>03 Aprs 2012</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = HETT 8164.13.00 06/09
03/01/2012 08:23

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 23411080A1
Standard Cyl #? 17
Standard Expiration? 10/01/2013
Oper No? 020533

Flow Cal. Date: 11/29/2010
Slope 653
Intercept -334316

IR Calibration Date: 06/14/2011
 3um 9um

0th Coef(*100): -6449 357
1st Coef(*100): 246728 129732
2nd Coef(*100): 5256 2318
H2O adj(mg/l*10k): 576 529

***** Printer Test End *****



Operator Signature
TRAVIS COLLINS

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = HETT 8164.13.00 06/09
03/01/2012 08:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:26
02 Std. Gas	0.081	08:26
03 Room Air	0.000	08:27
04 Std. Gas	0.081	08:27
05 Room Air	0.000	08:27
06 Std. Gas	0.081	08:28
07 Room Air	0.000	08:28

Lot No = 23411080A1
Cyl No = 17
Exp Date = 10/01/2013
County = 01 Oper No. = 020533



Operator Signature
TRAVIS COLLINS

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = HETT 8164.13.00 06/09
03/01/2012 08:29

Test	AC	Time
01 Diagnostic	OK	08:31
02 Room Air	RFI*	08:31
03 Room Air	RFI*	08:31

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST T
Sub DOB = 03/01/2000
Sub Sex = Male Weight = 100
Test = OTH Cit = 00000000
Dr. Lic. = ND/0000000
Lot No = 23411080A1
Cyl No = 17
Expiration Date = 10/01/2013
County = 01 Oper No. = 020533

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
TRAVIS COLLINS

Remarks:

Form 106-I8000

