



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59281 (1-2010)

Serial Number: <u>80-004189</u>	Instrument Location: <u>Williams Co LEC</u>
Reason for Install/Repair:	
<input type="checkbox"/> Install After Lab Repair/Inspection	
<input checked="" type="checkbox"/> Other (Specify) <u>Annual Replacement</u>	

### Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: \_\_\_\_\_

<u>Samuel Husell</u>	<u>04 Apr 11</u>
Field Inspector Signature	Date
<u>Deb Kashur</u>	<u>13 Apr 11</u>
Reviewed By	Date

80-004189  
P. 125 of 128  
RM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer  
North Dakota Model 8000 SN 80-004189  
Location = ~~TOXILWILL DK~~ 8164.13.00 06/09  
04/04/2011 13 April 17:58

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

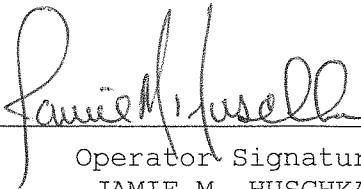
Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On Memfull? Yes  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 15410080A1  
Standard Cyl #? 6  
Standard Expiration? 07/01/2012  
Oper No? 100406

Flow Cal. Date: 02/02/2011  
Slope 628  
Intercept -452608

IR Calibration Date: 02/02/2011  
3um 9um

-----  
0th Coef(\*100): -22075 -26801  
1st Coef(\*100): 257433 132480  
2nd Coef(\*100): 2132 1061  
H2O adj(mg/l\*10k): 437 430

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Operator Signature  
JAMIE M. HUSCHKA

Remarks:

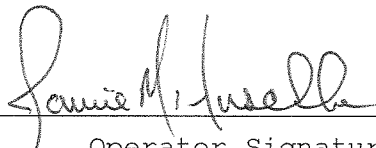
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004189  
Location = ~~TOXE WILL~~      8164.13.00 06/09  
04/04/2011      DK  
   13 APR 11      17:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:00
02 Std. Gas	0.080	18:00
03 Room Air	0.000	18:01
04 Std. Gas	0.079	18:01
05 Room Air	0.000	18:02
06 Std. Gas	0.079	18:02
07 Room Air	0.000	18:03

Lot No = 15410080A1  
Cyl No = 6  
Exp Date = 07/01/2012  
County = 53      Oper No. = 100406

  
\_\_\_\_\_  
Operator Signature  
JAMIE M. HUSCHKA

Remarks:

Form 106-I8000

80-004189  
P. 127 of 128  
FM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

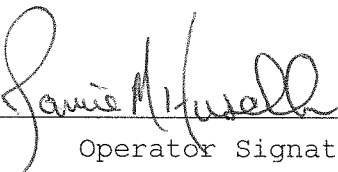
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004189  
Location = ~~FOX~~ WILL      8164.13.00 06/09  
04/04/2011      DK 13 APR 11      18:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	18:04
02 Room Air	0.000	18:04

\*Invalid Test  
Inhibited - RFI

Lot No = 15410080A1  
Cyl No = 6  
Exp Date = 07/01/2012  
County = 53      Oper No. = 100406

  
Operator Signature  
JAMIE M. HUSCHKA

Remarks:

Form 106-I8000

80-004189  
p. 128 of 128  
JMH