

**INTOXILYZER 8000 INSPECTION**

Serial No.: 80-003065

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5144 Lot No. — Rep AC 0.021 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR3847 Lot No. — Rep AC 0.294 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR3452 Lot No. — Rep AC 0.10AC + Acetone ✓
- J. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. <u>15410080A1</u>	Cyl No. <u>12</u>	Exp. Date <u>7-1-12</u>
Test 1 <u>0.080</u>	Test 4 <u>0.081</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.080</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.079</u>	Test 9 <u>0.080</u>
Average <u>0.079</u>		

L. Tank pressure, Level 3 Func D Sub G P: 260 psi Reg 255 psi

M. Remarks/Maintenance record: Annual Inspection

4-7-11  
Date

Deb Kashur  
Field Inspector's Signature

07 Apr 2011  
Date

Deb Kashur  
Reviewed by

80-003065  
P. 83 of 91  
FM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003065  
Location = TOXL                    8164.13.00 06/09  
04/07/2011                            10:26

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            15410080A1  
Standard Cyl #?            12  
Standard Expiration?        07/01/2012  
Oper No?                    555555

Flow Cal. Date:            04/08/2008  
Slope                        694  
Intercept                   -539075

IR Calibration Date:        06/05/2008  
                                    3um                            9um

-----  
0th Coef(\*100):            -9383                        -9235  
1st Coef(\*100):            288128                       138520  
2nd Coef(\*100):            957                            781  
H2O adj(mg/l\*10k):        391                            239

\*\*\*\*\* Printer Test End \*\*\*\*\*

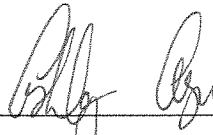  
\_\_\_\_\_  
Operator Signature  
ASHLEY AYD

Remarks: *Print test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003065  
Location = TOXL      8164.13.00 06/09  
04/07/2011      10:28

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:28
02 Std. Sol.	0.020	10:29
03 Room Air	0.000	10:29
04 Std. Sol.	0.021	10:30
05 Room Air	0.000	10:31
06 Std. Sol.	0.021	10:31
07 Room Air	0.000	10:32


08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 08

Oper No. = 555555

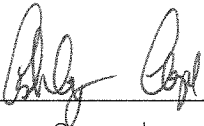

---

Operator Signature  
ASHLEY AYD

Remarks: *Low AC 0.021 AC*

Form 106-I8000

*80-003065  
p. 85 of 91 vta*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003065  
Location = TOXL      8164.13.00 06/09  
04/07/2011      10:33

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:34
02 Std. Sol.	0.295	10:34
03 Room Air	0.000	10:35
04 Std. Sol.	0.295	10:36
05 Room Air	0.000	10:36
06 Std. Sol.	0.296	10:37
07 Room Air	0.000	10:38


08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 08

Oper No. = 555555

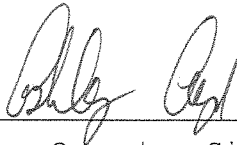

Operator Signature  
ASHLEY AYD

Remarks: *Linearity test 0.294 AC*

Form 106-I8000

80-003065  
P. 86-491 *vm*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003065  
Location = TOXL                    8164.13.00 06/09  
04/07/2011                            10:39

Test	AC	Time
01 Room Air	0.000	10:40
02 *Subject Test	INT*	10:40
03 Room Air	0.000	10:41

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 15410080A1  
Cyl No = 12  
Expiration Date = 07/01/2012  
County = 08                            Oper No. = 555555

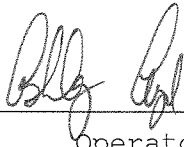

Operator Signature  
ASHLEY AYD

Remarks: *Interferent 0.10 AC + Acetone*

Form 106-I8000

80-003065  
p. 87 of 91  
VM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003065  
Location = TOXL                    8164.13.00 06/09  
04/07/2011                            10:42

Test	AC	Time
01 Diagnostic	OK	10:43
02 Room Air	RFI*	10:43
03 Room Air	0.000	10:44

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 15410080A1  
Cyl No = 12  
Expiration Date = 07/01/2012  
County = 08                            Oper No. = 555555

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

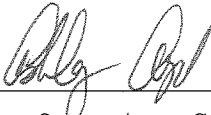

Operator Signature  
ASHLEY AYD

Remarks: RFI

Form 106-I8000

80-003065  
P-88491  
JFM


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003065  
Location = TOXL      8164.13.00 06/09  
04/07/2011      10:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:45
02 Std. Gas	0.080	10:45
03 Room Air	0.000	10:46
04 Std. Gas	0.079	10:46
05 Room Air	0.000	10:47
06 Std. Gas	0.079	10:47
07 Room Air	0.000	10:48

Lot No = 15410080A1  
Cyl No = 12  
Exp Date = 07/01/2012  
County = 08      Oper No. = 555555

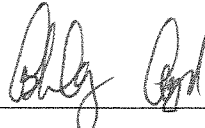

Operator Signature  
ASHLEY AYD

Remarks: Calibration check 0.080 AC

Form 106-I8000

80-003065  
P. 89 of 91  
KAM


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003065  
Location = TOXL                    8164.13.00 06/09  
04/07/2011                            10:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:49
02 Std. Gas	0.081	10:49
03 Room Air	0.000	10:50
04 Std. Gas	0.080	10:50
05 Room Air	0.000	10:51
06 Std. Gas	0.079	10:51
07 Room Air	0.000	10:52

Lot No = 15410080A1  
Cyl No = 12  
Exp Date = 07/01/2012  
County = 08                            Oper No. = 555555

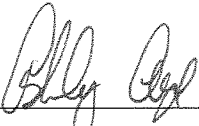  
\_\_\_\_\_  
Operator Signature  
ASHLEY AYD

Remarks: Calibration check 0.080 AC

Form 106-I8000

80-003065  
P. 90491  
FM



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003065  
Location = TOXL      8164.13.00 06/09  
04/07/2011      10:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:53
02 Std. Gas	0.080	10:53
03 Room Air	0.000	10:54
04 Std. Gas	0.079	10:54
05 Room Air	0.000	10:55
06 Std. Gas	0.080	10:55
07 Room Air	0.000	10:56

Lot No = 15410080A1  
Cyl No = 12  
Exp Date = 07/01/2012  
County = 08      Oper No. = 555555



Operator Signature  
ASHLEY AYD

Remarks: Calibration check 0.080 AC

Form 106-I8000

80-003065  
P. 91/91  
SM