



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (5-2011)

Serial Number <b>80-004193</b>	Instrument Location <b>FYPD</b>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): **NONE**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date <b>04-09-13</b>
Reviewed By (Crime Laboratory Use Only) <b>Deb Kashur</b>	Date <b>16 Apr 2013</b>



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004193  
Location = FYPD      8164.13.00 06/09  
04/09/2013      05:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	05:25
02 Std. Gas	0.080	05:26
03 Room Air	0.000	05:26
04 Std. Gas	0.080	05:26
05 Room Air	0.000	05:27
06 Std. Gas	0.080	05:27
07 Room Air	0.000	05:28

Lot No = 16512080A2

Cyl No = 25

Exp Date = 08/01/2014

County = 61

Oper No. = 010606



Operator Signature  
CHAD L. HARMON

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004193  
Location = FYPD                8164.13.00 06/09  
04/09/2013                        05:29

Test	AC	Time
01 Diagnostic	OK	05:32
02 Room Air	0.000	05:33
03 *Subject Test	RFI*	05:33
04 <del>Room Air</del>	0.000	05:34

\*Invalid Test  
Inhibited - RFI

Sub Name = KIRBY, AMBER N  
Sub DOB = 08/03/1984  
Sub Sex = Female                    Weight = 190  
Test = OTH                            Cit = 0000  
Dr. Lic. = ND/0000000  
Lot No = 16512080A2  
Cyl No = 25  
Expiration Date = 08/01/2014  
County = 43                            Oper No. = 010606

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHAD L. HARMON

Remarks: