



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

Serial Number <i>80-004193</i>	Instrument Location <i>FYPD</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): *NONE*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>04-09-13</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashur</i>	Date <i>16 Apr 2013</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *[Signature]* Certified Date: *08 Nov. 16*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = FYPD 8164.13.00 06/09
04/09/2013 05:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	05:25
02 Std. Gas	0.080	05:26
03 Room Air	0.000	05:26
04 Std. Gas	0.080	05:26
05 Room Air	0.000	05:27
06 Std. Gas	0.080	05:27
07 Room Air	0.000	05:28

Lot No = 16512080A2
Cyl No = 25
Exp Date = 08/01/2014
County = 61 Oper No. = 010606

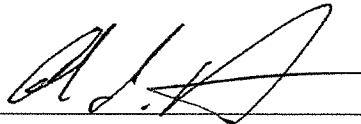

Operator Signature
CHAD L. HARMON

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = FYPD 8164.13.00 06/09
04/09/2013 05:29

Test	AC	Time
01 Diagnostic	OK	05:32
02 Room Air	0.000	05:33
03 *Subject Test	RFI*	05:33
04 Room Air	0.000	05:34

*Invalid Test
Inhibited - RFI

Sub Name = KIRBY, AMBER N
Sub DOB = 08/03/1984
Sub Sex = Female Weight = 190
Test = OTH Cit = 0000
Dr. Lic. = ND/0000000
Lot No = 16512080A2
Cyl No = 25
Expiration Date = 08/01/2014
County = 43 Oper No. = 010606

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHAD L. HARMON

Remarks: