

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004189

Location: TOXL

- Check When Done
- A. Is the warm-up time less than 20 minutes? ✓
 - B. Is a three-pronged grounded outlet used? ✓
 - C. Is the breath tube heated? ✓
 - D. Is the diagnostic check complete? ✓
 - E. Is the time, date, and year correct (re-set if necessary)? ✓
 - F. Print test. (Attach test record.) ✓
 - G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5132 Lot No. 12210 Rep AC 0.020 ✓
 - H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN 12050^{DR} Lot No. 12050 Rep AC 0.300
DR 3803 ✓
 - I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR3846 Lot No. — Rep AC 0.100 + acetone ✓
 - J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
 - K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. <u>23411080A1</u>	Cyl No. <u>15</u>	Exp. Date <u>10/01/13</u>
Test 1 <u>0.080</u>	Test 4 <u>0.081</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.079</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

L. Tank pressure, Level 3 Func D Sub G P: 540 psi Reg 550 psi

M. Remarks/Maintenance record: Annual inspection, replaced breath hose o-ring

10 Apr 2013
Date
10 APR 13
Date

Deb Kashner
Field Inspector's Signature
Charles E. ...
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004189
Location = WILL 8164.13.00 06/09
04/10/2013 14:45

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:45
02 Std. Sol.	0.020	14:46
03 Room Air	0.000	14:47
04 Std. Sol.	0.021	14:47
05 Room Air	0.000	14:48
06 Std. Sol.	0.020	14:49
07 Room Air	0.000	14:49

08 Sim Temp = 34.0°C

Simul Ser No = DR5132

Std Sol No = 12210

County = 08

Oper No. = 777777

Debo Kashur

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004189
Location = WILL 8164.13.00 06/09
04/10/2013 14:50

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:51
02 Std. Sol.	0.299	14:51
03 Room Air	0.000	14:52
04 Std. Sol.	0.298	14:53
05 Room Air	0.000	14:53
06 Std. Sol.	0.298	14:54
07 Room Air	0.000	14:54

08 Sim Temp = 34.0°C

Simul Ser No = DR3803

Std Sol No = 12050

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004189
Location = TOXL 8164.13.00 06/09
04/10/2013 16:02

Test	AC	Time
01 Room Air	0.000	16:03
02 *Subject Test	INT*	16:03
03 Room Air	0.000	16:04

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 15
Expiration Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100 Ac + acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004189
Location = TOXL 8164.13.00 06/09
04/10/2013 16:07

Test	AC	Time
01 Diagnostic	OK	16:08
02 Room Air	RFI*	16:08
03 Room Air	0.000	16:08

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 15
Expiration Date = 10/01/2013
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004189
Location = TOXL 8164.13.00 06/09
04/10/2013 16:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:10
02 Std. Gas	0.080	16:11
03 Room Air	0.000	16:11
04 Std. Gas	0.080	16:12
05 Room Air	0.000	16:12
06 Std. Gas	0.080	16:12
07 Room Air	0.000	16:13

Lot No = 23411080A1

Cyl No = 15

Exp Date = 10/01/2013

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004189
Location = TOXL 8164.13.00 06/09
04/10/2013 16:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:18
02 Std. Gas	0.081	16:18
03 Room Air	0.000	16:19
04 Std. Gas	0.079	16:19
05 Room Air	0.000	16:19
06 Std. Gas	0.080	16:20
07 Room Air	0.000	16:20

Lot No = 23411080A1
Cyl No = 15
Exp Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004189
Location = TOXL 8164.13.00 06/09
04/10/2013 16:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:28
02 Std. Gas	0.080	16:28
03 Room Air	0.000	16:29
04 Std. Gas	0.080	16:29
05 Room Air	0.000	16:30
06 Std. Gas	0.080	16:30
07 Room Air	0.000	16:31

Lot No = 23411080A1
Cyl No = 15
Exp Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC