



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (03-2014)

Serial Number 80-004950	Instrument Location TIOGA P.D.
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): NONE
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date 4/11/16
Reviewed By (Crime Laboratory Use Only) Deb Shenauer	Date 13 Apr 2016

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TIOG 8164.13.00 06/09
04/11/2016 15:09

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 13314080A2
Standard Cyl #? 49
Standard Expiration? 06/05/2016
Oper No? 020632

Flow Cal. Date: 01/18/2010
Slope 774
Intercept -908904

IR Calibration Date: 12/07/2015
 3um 9um

0th Coef(*100): -19691 -21281
1st Coef(*100): 279961 143809
2nd Coef(*100): 1836 1123
H2O adj(mg/l*10k): 321 330

***** Printer Test End *****



Operator Signature
SHANNON DUISEN

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TIOG 8164.13.00 06/09
04/11/2016 15:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:16
02 Std. Gas	0.080	15:16
03 Room Air	0.000	15:17
04 Std. Gas	0.080	15:17
05 Room Air	0.000	15:18
06 Std. Gas	0.080	15:18
07 Room Air	0.000	15:19

Lot No = 13314080A2
Cyl No = 49
Exp Date = 06/05/2016
County = 53 Oper No. = 020632



Operator Signature
SHANNON DUISEN

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TIOG 8164.13.00 06/09
04/11/2016 15:20

Test	AC	Time
01 Room Air	0.000	15:22
02 *Subject Test	RFI*	15:22
03 Room Air	0.000	15:22

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST T
Sub DOB = 01/01/1981
Sub Sex = Female Weight = 200
Test = DUI Cit = 0000000
Dr. Lic. = ND/123456789
Lot No = 13314080A2
Cyl No = 49
Expiration Date = 06/05/2016
County = 53 Oper No. = 020632



Operator Signature
SHANNON DUISEN

Remarks: