

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006506

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5114 Lot No. 201505B Rep AC 0.030 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3068 Lot No. 14102 Rep AC 0.400 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. ICS 4 Solution 0.10AC+0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 13314080A3 Cyl No. 028 Exp. Date 6/5/2016

Test 1 <u>0.081</u>	Test 4 <u>0.082</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.082</u>	Test 8 <u>0.082^{CER}1</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.082</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 675 psi Reg 700 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

4.11.16
Date

11 Apr 2016
Date

[Signature]
Field Inspector's Signature

Deb Shanover
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006506
Location = TOXL 8164.13.00 06/09
04/11/2016 15:29

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 13314080A3
Standard Cyl #? 28
Standard Expiration? 06/05/2016
Oper No? 666666

Flow Cal. Date: 08/20/2015
Slope 657
Intercept -627881

IR Calibration Date: 08/20/2015
 3um 9um

0th Coef(*100): -22470 -18919
1st Coef(*100): 266505 136109
2nd Coef(*100): 3702 1644
H2O adj(mg/l*10k): 709 510

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006506
Location = TOXL 8164.13.00 06/09
04/11/2016 15:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:41
02 Std. Sol.	0.028	15:41
03 Room Air	0.000	15:42
04 Std. Sol.	0.028	15:43
05 Room Air	0.000	15:43
06 Std. Sol.	0.028	15:44
07 Room Air	0.000	15:45

08 Sim Temp = 34.0°C

Simul Ser No = DR5114
Std Sol No = 201505B
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

LOW AC
0.030 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006506
Location = TOXL 8164.13.00 06/09
04/11/2016 15:46

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:47
02 Std. Sol.	0.396	15:48
03 Room Air	0.000	15:48
04 Std. Sol.	0.397	15:49
05 Room Air	0.000	15:50
06 Std. Sol.	0.399	15:50
07 Room Air	0.000	15:51

08 Sim Temp = 34.0°C

Simul Ser No = MP3068

Std Sol No = 14102

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC
0.400AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006506
Location = TOXL 8164.13.00 06/09
04/11/2016 15:53

Test	AC	Time
01 Room Air	0.000	15:54
02 *Subject Test	INT*	15:54
03 Room Air	0.000	15:55

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 28
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

INTERFERENT CHECK
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006506
Location = TOXL 8164.13.00 06/09
04/11/2016 15:55

Test	AC	Time
01 Diagnostic	OK	15:56
02 Room Air	0.000	15:57
03 *Subject Test	RFI*	15:57
04 Room Air	0.000	15:57

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 28
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006506
Location = TOXL 8164.13.00 06/09
04/11/2016 15:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:00
02 Std. Gas	0.081	16:00
03 Room Air	0.000	16:01
04 Std. Gas	0.081	16:01
05 Room Air	0.000	16:02
06 Std. Gas	0.081	16:02
07 Room Air	0.000	16:02

Lot No = 13314080A3
Cyl No = 28
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006506
Location = TOXL 8164.13.00 06/09
04/11/2016 16:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:03
02 Std. Gas	0.082	16:04
03 Room Air	0.000	16:04
04 Std. Gas	0.082	16:05
05 Room Air	0.000	16:05
06 Std. Gas	0.081	16:06
07 Room Air	0.000	16:06

Lot No = 13314080A3
Cyl No = 28
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006506
Location = TOXL 8164.13.00 06/09
04/11/2016 16:06

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:07
02 Std. Gas	0.081	16:07
03 Room Air	0.000	16:08
04 Std. Gas	0.081	16:08
05 Room Air	0.000	16:09
06 Std. Gas	0.082	16:09
07 Room Air	0.000	16:10

Lot No = 13314080A3
Cyl No = 28
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
~~0.400 AC~~ CEE 0.080 AC