

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006490

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5132 Lot No. 201507F Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3841 Lot No. 201603A Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5144 Lot No. FCS4 Solution 0.10Ac + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 1424080A6 Cyl No. 51 Exp. Date 07/05/2016

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 596 psi Reg ~580 psi

K. Remarks/Maintenance record: Annual inspection

12 Apr 2016
Date

Deb Shanaver
Field Inspector's Signature

12 APR 16
Date

Charles E. Edler
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TOXL 8164.13.00 06/09
04/12/2016 14:59

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 14214080A6
Standard Cyl #? 51
Standard Expiration? 07/05/2016
Oper No? 777777

Flow Cal. Date: 08/18/2015
Slope 675
Intercept -635799

IR Calibration Date: 08/18/2015


	3um	9um
0th Coef(*100):	-18073	-15729
1st Coef(*100):	265337	133887
2nd Coef(*100):	3965	1560
H2O adj(mg/l*10k):	767	457

***** Printer Test End *****



Operator Signature
DEB SHANAVER

Remarks:

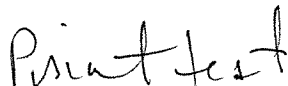

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TOXL 8164.13.00 06/09
04/12/2016 14:52

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:53
02 Std. Sol.	0.019	14:54
03 Room Air	0.000	14:54
04 Std. Sol.	0.020	14:55
05 Room Air	0.000	14:56
06 Std. Sol.	0.020	14:56
07 Room Air	0.000	14:57

08 Sim Temp = 34.0°C

Simul Ser No = DR5132

Std Sol No = 201507F

County = 08

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TOXL 8164.13.00 06/09
04/12/2016 14:46

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:47
02 Std. Sol.	0.303	14:48
03 Room Air	0.000	14:48
04 Std. Sol.	0.304	14:49
05 Room Air	0.000	14:50
06 Std. Sol.	0.304	14:50
07 Room Air	0.000	14:51

08 Sim Temp = 34.0°C

Simul Ser No = DR3841

Std Sol No = 201603A

County = 08

Oper No. = 777777

D.b Shanaver

Operator Signature
DEB SHANAVER

Remarks:

High AC ^{DS}
~~0.0~~ 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TOXL 8164.13.00 06/09
04/12/2016 14:59

Test	AC	Time
01 Room Air	0.000	15:00
02 *Subject Test	INT*	15:00
03 Room Air	0.000	15:01

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 51
Expiration Date = 07/05/2016
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Interferent detected 0.10Ac + 0.05% acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TOXL 8164.13.00 06/09
04/12/2016 15:01

Test	AC	Time
01 Diagnostic	OK	15:02
02 Room Air	RFI*	15:02
03 Room Air	0.000	15:03

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 51
Expiration Date = 07/05/2016
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB SHANAVER

Remarks:

RFI check

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TOXL 8164.13.00 06/09
04/12/2016 15:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:04
02 Std. Gas	0.081	15:04
03 Room Air	0.000	15:05
04 Std. Gas	0.080	15:05
05 Room Air	0.000	15:06
06 Std. Gas	0.080	15:06
07 Room Air	0.000	15:07

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TOXL 8164.13.00 06/09
04/12/2016 15:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:07
02 Std. Gas	0.081	15:08
03 Room Air	0.000	15:08
04 Std. Gas	0.081	15:09
05 Room Air	0.000	15:09
06 Std. Gas	0.081	15:10
07 Room Air	0.000	15:10

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TOXL 8164.13.00 06/09
04/12/2016 15:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:11
02 Std. Gas	0.081	15:11
03 Room Air	0.000	15:12
04 Std. Gas	0.081	15:12
05 Room Air	0.000	15:13
06 Std. Gas	0.081	15:13
07 Room Air	0.000	15:14

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777


Operator Signature
DEB SHANAVER

Remarks:
Calibration check 0.080AC