

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006493

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3374 Lot No. 14240 Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3841 Lot No. 201603A Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5144 Lot No. ICS 4 Solution 0.10AC + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 14214080A6 Cyl No. 51 Exp. Date 07/05/2016

Test 1 0.080 Test 4 0.080 Test 7 0.080

Test 2 0.080 Test 5 0.080 Test 8 0.080

Test 3 0.080 Test 6 0.080 Test 9 0.080

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 664 psi Reg 2660 psi

K. Remarks/Maintenance record: Annual inspection

12 Apr 16
Date

12 APR. 16
Date

Deb Shanaver
Field Inspector's Signature

Charles E. Edr
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = TOXL 8164.13.00 06/09
04/12/2016 11:31

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 14214080A6
Standard Cyl #? 51
Standard Expiration? 07/05/2016
Oper No? 777777

Flow Cal. Date: 02/12/2016
Slope 691
Intercept -663384

IR Calibration Date: 02/12/2016

	3um	9um
0th Coef(*100):	-36676	-26473
1st Coef(*100):	258504	130468
2nd Coef(*100):	6151	2344
H2O adj(mg/l*10k):	821	589

***** Printer Test End *****



Operator Signature
DEB SHANAVER

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = TOXL 8164.13.00 06/09
04/12/2016 11:02

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:03
02 Std. Sol.	0.017	11:03
03 Room Air	0.000	11:04
04 Std. Sol.	0.017	11:05
05 Room Air	0.000	11:05
06 Std. Sol.	0.018	11:06
07 Room Air	0.000	11:06

08 Sim Temp = 34.0°C

Simul Ser No = DR3374
Std Sol No = 14240
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Low AC 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = TOXL 8164.13.00 06/09
04/12/2016 11:12

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:12
02 Std. Sol.	0.306	11:13
03 Room Air	0.000	11:14
04 Std. Sol.	0.307	11:15
05 Room Air	0.000	11:15
06 Std. Sol.	0.308	11:16
07 Room Air	0.000	11:17


08 Sim Temp = 34.0°C

Simul Ser No = DR3841

Std Sol No = 201603A

County = 08

Oper No. = 777777


Operator Signature
DEB SHANAVER

Remarks:

High AC 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = TOXL 8164.13.00 06/09
04/12/2016 11:28

Test	AC	Time
01 Room Air	0.000	11:28
02 *Subject Test	INT*	11:29
03 Room Air	0.000	11:29

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 51
Expiration Date = 07/05/2016
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Interferent detected 0.10AC + 0.05% acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = TOXL 8164.13.00 06/09
04/12/2016 11:32

Test	AC	Time
01 Diagnostic	OK	11:33
02 Room Air	RFI*	11:33
03 Room Air	0.000	11:33

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 51
Expiration Date = 07/05/2016
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB SHANAVER

Remarks:

RFI test


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = TOXL 8164.13.00 06/09
04/12/2016 11:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:34
02 Std. Gas	0.080	11:34
03 Room Air	0.000	11:35
04 Std. Gas	0.080	11:35
05 Room Air	0.000	11:36
06 Std. Gas	0.080	11:36
07 Room Air	0.000	11:37

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = TOXL 8164.13.00 06/09
04/12/2016 11:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:42
02 Std. Gas	0.080	11:43
03 Room Air	0.000	11:43
04 Std. Gas	0.080	11:43
05 Room Air	0.000	11:44
06 Std. Gas	0.080	11:44
07 Room Air	0.000	11:45

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = TOXL 8164.13.00 06/09
04/12/2016 11:46

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:47
02 Std. Gas	0.080	11:47
03 Room Air	0.000	11:48
04 Std. Gas	0.080	11:48
05 Room Air	0.000	11:49
06 Std. Gas	0.080	11:49
07 Room Air	0.000	11:50

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-I8000