

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006497

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3374 Lot No. 14240 Rep AC 0.020 ✓
- F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3841 Lot No. 201603A Rep AC 0.300 ✓
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5144 Lot No. ICS4 Solution 0.10AC + 0.05% acetone ✓
- H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 14214080A6 Cyl No. 51 Exp. Date 07/05/2016

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.082</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.082</u>	Test 9 <u>0.082</u>

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 623 psi Reg 2600 psi

K. Remarks/Maintenance record: Annual inspection

12 Apr 2016
Date

12 APR. 16
Date

Deb Shanaver
Field Inspector's Signature

Charles E. Egan
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = TOXL 8164.13.00 06/09
04/12/2016 14:14

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 14214080A6
Standard Cyl #? 51
Standard Expiration? 07/05/2016
Oper No? 777777

Flow Cal. Date: 08/19/2015
Slope 657
Intercept -642041

IR Calibration Date: 08/19/2015


	3um	9um
0th Coef(*100):	-16255	-17993
1st Coef(*100):	263339	136695
2nd Coef(*100):	5198	1666
H2O adj(mg/l*10k):	766	522

***** Printer Test End *****



Operator Signature
DEB SHANAVER

Remarks:

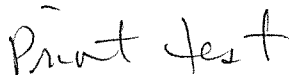

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = TOXL 8164.13.00 06/09
04/12/2016 14:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:01
02 Std. Sol.	0.016	14:02
03 Room Air	0.000	14:03
04 Std. Sol.	0.017	14:03
05 Room Air	0.000	14:04
06 Std. Sol.	0.017	14:05
07 Room Air	0.000	14:06

08 Sim Temp = 34.0°C

Simul Ser No = DR3374
Std Sol No = 14240
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Low AC 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = TOXL 8164.13.00 06/09
04/12/2016 14:07

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:07
02 Std. Sol.	0.308	14:08
03 Room Air	0.000	14:09
04 Std. Sol.	0.303	14:10
05 Room Air	0.000	14:11
06 Std. Sol.	0.302	14:11
07 Room Air	0.000	14:12

08 Sim Temp = 34.0°C

Simul Ser No = DR3841
Std Sol No = 201603A
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

High AC 0.300AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = TOXL 8164.13.00 06/09
04/12/2016 14:14

Test	AC	Time
01 Room Air	0.000	14:15
02 *Subject Test	INT*	14:15
03 Room Air	0.000	14:16

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 51
Expiration Date = 07/05/2016
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

*Interferent detected 0.10AE + 0.05%
acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = TOXL 8164.13.00 06/09
04/12/2016 14:16

Test	AC	Time
01 Diagnostic	OK	14:17
02 Room Air	RFI*	14:17
03 Room Air	0.000	14:17

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 51
Expiration Date = 07/05/2016
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = TOXL 8164.13.00 06/09
04/12/2016 14:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:19
02 Std. Gas	0.081	14:19
03 Room Air	0.000	14:20
04 Std. Gas	0.082	14:20
05 Room Air	0.000	14:21
06 Std. Gas	0.081	14:21
07 Room Air	0.000	14:22

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-18000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = TOXL 8164.13.00 06/09
04/12/2016 14:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:23
02 Std. Gas	0.081	14:23
03 Room Air	0.000	14:24
04 Std. Gas	0.082	14:24
05 Room Air	0.000	14:25
06 Std. Gas	0.082	14:25
07 Room Air	0.000	14:26

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = TOXL 8164.13.00 06/09
04/12/2016 14:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:29
02 Std. Gas	0.082	14:29
03 Room Air	0.000	14:30
04 Std. Gas	0.081	14:30
05 Room Air	0.000	14:31
06 Std. Gas	0.082	14:31
07 Room Air	0.000	14:32

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC