

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006489

Location: TOXL

- | | Check When Done |
|---|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u>201507F</u> Rep AC <u>0.620</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR 3841</u> Lot No. <u>201603A</u> Rep AC <u>0.300</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>DR 5144</u> Lot No. <u>ICS4</u> Solution <u>0.10AC + 0.05% acetone</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. 14214080A6 Cyl No. 51 Exp. Date 07/05/2016

Test 1 0.080 Test 4 0.081 Test 7 0.081

Test 2 0.081 Test 5 0.081 Test 8 0.081

Test 3 0.080 Test 6 0.080 Test 9 0.081

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 564 psi Reg ~550 psi

K. Remarks/Maintenance record: Annual inspection

13 Apr 2016
Date

Deb Shanaver
Field Inspector's Signature

13 APR. 16
Date

Charles E. Ed
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = TOXL 8164.13.00 06/09
04/13/2016 08:51

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 14214080A6
Standard Cyl #? 51
Standard Expiration? 07/05/2016
Oper No? 777777

Flow Cal. Date: 08/18/2015
Slope 663
Intercept -631787

IR Calibration Date: 08/18/2015

	3um	9um
0th Coef(*100):	-24626	-16551
1st Coef(*100):	269218	130789
2nd Coef(*100):	3390	1656
H2O adj(mg/l*10k):	742	460

***** Printer Test End *****



Operator Signature
DEB SHANAVER

Remarks:

Print Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = TOXL 8164.13.00 06/09
04/13/2016 08:38

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:38
02 Std. Sol.	0.021	08:39
03 Room Air	0.000	08:40
04 Std. Sol.	0.021	08:41
05 Room Air	0.000	08:41
06 Std. Sol.	0.021	08:42
07 Room Air	0.000	08:43

08 Sim Temp = 34.0°C

Simul Ser No = DR5132
Std Sol No = 201507F
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Low AC 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = TOXL 8164.13.00 06/09
04/13/2016 08:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:45
02 Std. Sol.	0.305	08:46
03 Room Air	0.000	08:46
04 Std. Sol.	0.306	08:47
05 Room Air	0.000	08:48
06 Std. Sol.	0.306	08:49
07 Room Air	0.000	08:49

08 Sim Temp = 34.0°C

Simul Ser No = DR3841
Std Sol No = 201603A
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

High AC 0.300 Ac

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = TOXL 8164.13.00 06/09
04/13/2016 08:51

Test	AC	Time
01 Room Air	0.000	08:52
02 *Subject Test	INT*	08:52
03 Room Air	0.000	08:53

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 51
Expiration Date = 07/05/2016
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Interferent detected 0.10AC + 0.05% acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = TOXL 8164.13.00 06/09
04/13/2016 08:53

Test	AC	Time
01 Diagnostic	OK	08:54
02 Room Air	RFI*	08:54
03 Room Air	0.000	08:55

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 51
Expiration Date = 07/05/2016
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB SHANAVER

Remarks:

RFI check

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = TOXL 8164.13.00 06/09
04/13/2016 08:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:56
02 Std. Gas	0.080	08:56
03 Room Air	0.000	08:57
04 Std. Gas	0.081	08:57
05 Room Air	0.000	08:58
06 Std. Gas	0.080	08:58
07 Room Air	0.000	08:59

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:
Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = TOXL 8164.13.00 06/09
04/13/2016 08:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:59
02 Std. Gas	0.081	09:00
03 Room Air	0.000	09:00
04 Std. Gas	0.081	09:01
05 Room Air	0.000	09:01
06 Std. Gas	0.080	09:02
07 Room Air	0.000	09:02

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = TOXL 8164.13.00 06/09
04/13/2016 09:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:03
02 Std. Gas	0.081	09:04
03 Room Air	0.000	09:04
04 Std. Gas	0.081	09:05
05 Room Air	0.000	09:05
06 Std. Gas	0.081	09:06
07 Room Air	0.000	09:06

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777

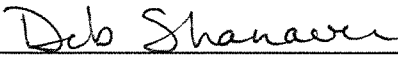

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC