

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-002667

Location: TOX L

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN MP3066 Lot No. 15160 Rep AC 0.020 ✓
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN MP3068 Lot No. 14102 Rep AC 0.400 ✓
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 13314080A3 Cyl No. 028 Exp. Date 6/5/2016

Test 1 0.078 Test 4 0.078 Test 7 0.078

Test 2 0.078 Test 5 0.079 Test 8 0.079

Test 3 0.079 Test 6 0.079 Test 9 0.078

Average 0.078

J. Tank pressure, Level 3 Func D Sub G P: 169 psi Reg ~175 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED SIMULATOR

RETURN O-RING

4.14.2016  
Date

Charles E. Ed  
Field Inspector's Signature

14 Apr 16  
Date

Deb Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-002667  
Location = TOXL                8164.13.00 06/09  
04/14/2016                      14:21

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            13314080A3  
Standard Cyl #?            28  
Standard Expiration?       06/05/2016  
Oper No?                    666666

Flow Cal. Date:            09/10/2007  
Slope                      693  
Intercept                  -570251

IR Calibration Date:       05/11/2015

	3um	9um
0th Coef(*100):	-33166	-22555
1st Coef(*100):	276016	138896
2nd Coef(*100):	666	778
H2O adj(mg/l*10k):	371	188

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002667  
Location = TOXL      8164.13.00 06/09  
04/14/2016      14:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:23
02 Std. Sol.	0.017	14:24
03 Room Air	0.000	14:25
04 Std. Sol.	0.018	14:25
05 Room Air	0.000	14:26
06 Std. Sol.	0.017	14:27
07 Room Air	0.000	14:27

08 Sim Temp = 34.0°C

Simul Ser No = MP3066  
Std Sol No = 15160  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:      *Low AC*  
                    *0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002667  
Location = TOXL      8164.13.00 06/09  
04/14/2016      14:28

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:29
02 Std. Sol.	0.402	14:30
03 Room Air	0.000	14:30
04 Std. Sol.	0.402	14:31
05 Room Air	0.000	14:31
06 Std. Sol.	0.401	14:32
07 Room Air	0.000	14:33

08 Sim Temp = 34.0°C

Simul Ser No = MP3068  
Std Sol No = 14102  
County = 08      Oper No. = 666666

  
Operator Signature  
CHARLES EDER

Remarks: HIGH AC  
0.400 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002667  
Location = TOXL      8164.13.00 06/09  
04/14/2016      14:34

Test	AC	Time
01 Room Air	0.000	14:35
02 *Subject Test	INT*	14:35
03 Room Air	0.000	14:36

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 13314080A3  
Cyl No = 28  
Expiration Date = 06/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002667  
Location = TOXL      8164.13.00 06/09  
04/14/2016      14:36

Test	AC	Time
01 Diagnostic	OK	14:37
02 Room Air	RFI*	14:38
03 Room Air	0.000	14:38

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 13314080A3  
Cyl No = 28  
Expiration Date = 06/05/2016  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-002667  
Location = TOXL                    8164.13.00 06/09  
04/14/2016                            14:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:39
02 Std. Gas	0.078	14:39
03 Room Air	0.000	14:40
04 Std. Gas	0.078	14:40
05 Room Air	0.000	14:40
06 Std. Gas	0.079	14:41
07 Room Air	0.000	14:41

Lot No = 13314080A3  
Cyl No = 28  
Exp Date = 06/05/2016  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002667  
Location = TOXL      8164.13.00 06/09  
04/14/2016      14:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:44
02 Std. Gas	0.078	14:44
03 Room Air	0.000	14:45
04 Std. Gas	0.079	14:45
05 Room Air	0.000	14:46
06 Std. Gas	0.079	14:46
07 Room Air	0.000	14:46

Lot No = 13314080A3  
Cyl No = 28  
Exp Date = 06/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-002667  
Location = TOXL                    8164.13.00 06/09  
04/14/2016                            14:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:47
02 Std. Gas	0.078	14:48
03 Room Air	0.000	14:48
04 Std. Gas	0.079	14:48
05 Room Air	0.000	14:49
06 Std. Gas	0.078	14:49
07 Room Air	0.000	14:50

Lot No = 13314080A3  
Cyl No = 28  
Exp Date = 06/05/2016  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000