

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003058

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN MP3066 Lot No. 15160 Rep AC 0.020 ✓
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN MP3068 Lot No. 14102 Rep AC 0.400 ✓
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. ICS4 Solution 0.10% AC + 0.05% ACETONE ✓
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 13314080A3 Cyl No. 028 Exp. Date 6/5/2016

Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.080</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.080</u>	Test 6 <u>0.081</u>	Test 9 <u>0.080</u>

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 247 psi Reg ~250 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED SIMULATOR RETURN O-RING, REPLACED BREATH HOSE O-RING

4.14.2016  
Date  
14 Apr 2016  
Date

Charles E. Ed  
Field Inspector's Signature  
Db Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003058  
Location = TOXL                    8164.13.00 06/09  
04/14/2016    11:08

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:                    Enabled  
Start Test Sequence:                DABACABA  
Display Prelim Rslt?                Yes  
Display Third Digit?                Yes  
Inhib Printer(Y/N)?                No  
Display Volume?                    No  
Disable On Memfull?                Yes  
# of Print Copies?                  1  
Select Std (D/W/I)?                Dry  
Standard Value?                    0.080  
Standard Lot #?                    13314080A3  
Standard Cyl #?                    28  
Standard Expiration?                06/05/2016  
Oper No?                            666666

Flow Cal. Date:                    04/07/2008  
Slope                                698  
Intercept                            -678970

IR Calibration Date:                04/15/2015

	3um	9um
0th Coef(*100):	-18934	-10185
1st Coef(*100):	272557	131040
2nd Coef(*100):	2331	1317
H2O adj(mg/l*10k):	333	280

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:                    PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003058  
Location = TOXL                    8164.13.00 06/09  
04/14/2016                            11:11

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:11
02 Std. Sol.	0.019	11:12
03 Room Air	0.000	11:12
04 Std. Sol.	0.019	11:13
05 Room Air	0.000	11:14
06 Std. Sol.	0.019	11:14
07 Room Air	0.000	11:15

08 Sim Temp = 34.0°C

Simul Ser No = MP3066

Std Sol No = 15160

County = 08

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Low AC 0.020 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.13.00 06/09  
04/14/2016      11:21

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:21
02 Std. Sol.	0.396	11:22
03 Room Air	0.000	11:23
04 Std. Sol.	0.396	11:23
05 Room Air	0.000	11:24
06 Std. Sol.	0.395	11:24
07 Room Air	0.000	11:25

08 Sim Temp = 34.0°C

Simul Ser No = MP3068

Std Sol No = 14102

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC

0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003058  
Location = TOXL                8164.13.00 06/09  
04/14/2016                            12:23

Test	AC	Time
01 Room Air	0.000	12:24
02 *Subject Test	INT*	12:24
03 Room Air	0.000	12:25

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 13314080A3  
Cyl No = 28  
Expiration Date = 06/05/2016  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003058  
Location = TOXL                8164.13.00 06/09  
04/14/2016                        12:25

Test	AC	Time
01 Diagnostic	OK	12:26
02 Room Air	0.000	12:27
03 *Subject Test	RFI*	12:27
04 Room Air	0.000	12:27

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 13314080A3  
Cyl No = 28  
Expiration Date = 06/05/2016  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.13.00 06/09  
04/14/2016      12:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:29
02 Std. Gas	0.079	12:29
03 Room Air	0.000	12:30
04 Std. Gas	0.080	12:30
05 Room Air	0.000	12:30
06 Std. Gas	0.080	12:31
07 Room Air	0.000	12:31

Lot No = 13314080A3  
Cyl No = 28  
Exp Date = 06/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.13.00 06/09  
04/14/2016      12:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:32
02 Std. Gas	0.079	12:33
03 Room Air	0.000	12:33
04 Std. Gas	0.079	12:33
05 Room Air	0.000	12:34
06 Std. Gas	0.081	12:34
07 Room Air	0.000	12:35

Lot No = 13314080A3  
Cyl No = 28  
Exp Date = 06/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003058  
Location = TOXL                 8164.13.00 06/09  
04/14/2016                        12:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:36
02 Std. Gas	0.079	12:36
03 Room Air	0.000	12:36
04 Std. Gas	0.079	12:37
05 Room Air	0.000	12:37
06 Std. Gas	0.080	12:38
07 Room Air	0.000	12:38

Lot No = 13314080A3  
Cyl No = 28  
Exp Date = 06/05/2016  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000