

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004206

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3066 Lot No. 15160 Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3068 Lot No. 14102 Rep AC 0.400 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 13314080A3 Cyl No. 028 Exp. Date 6/5/2016

Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.078</u>	Test 8 <u>0.079</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.079</u>

Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 131 psi Reg ~125 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED SIMULATOR

RETURN O-RING
4.14.2016
Date
14 Apr 16
Date

Charles E. Eden
Field Inspector's Signature
Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
04/14/2016 15:46

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 13314080A3
Standard Cyl #? 28
Standard Expiration? 06/05/2016
Oper No? 666666

Flow Cal. Date: 06/10/2009
Slope 690
Intercept -645883

IR Calibration Date: 04/09/2015

	3um	9um
0th Coef(*100):	-24157	-27270
1st Coef(*100):	262661	128882
2nd Coef(*100):	2137	1256
H2O adj(mg/l*10k):	491	640

***** Printer Test End *****

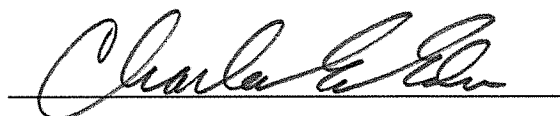

Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
04/14/2016 15:49

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:50
02 Std. Sol.	0.017	15:51
03 Room Air	0.000	15:51
04 Std. Sol.	0.018	15:52
05 Room Air	0.000	15:53
06 Std. Sol.	0.017	15:53
07 Room Air	0.000	15:54

08 Sim Temp = 34.0°C

Simul Ser No = MP3066
Std Sol No = 15160
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
04/14/2016 16:00

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:00
02 Std. Sol.	0.402	16:01
03 Room Air	0.000	16:02
04 Std. Sol.	0.400	16:02
05 Room Air	0.000	16:03
06 Std. Sol.	0.400	16:04
07 Room Air	0.000	16:04

08 Sim Temp = 34.0°C

Simul Ser No = MP3068
Std Sol No = 14102
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: HIGH AC
0.400 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
04/14/2016 16:07

Test	AC	Time
01 Room Air	0.000	16:07
02 *Subject Test	INT*	16:08
03 Room Air	0.000	16:08

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 28
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
04/14/2016 16:09

Test	AC	Time
01 Diagnostic	OK	16:09
02 Room Air	RFI*	16:10
03 Room Air	0.000	16:10

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 28
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
04/14/2016 16:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:11
02 Std. Gas	0.079	16:11
03 Room Air	0.000	16:12
04 Std. Gas	0.079	16:12
05 Room Air	0.000	16:12
06 Std. Gas	0.080	16:13
07 Room Air	0.000	16:13

Lot No = 13314080A3
Cyl No = 28
Exp Date = 06/05/2016
County = 08 Oper No. = 666666


Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
04/14/2016 16:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:14
02 Std. Gas	0.079	16:15
03 Room Air	0.000	16:15
04 Std. Gas	0.078	16:15
05 Room Air	0.000	16:16
06 Std. Gas	0.080	16:16
07 Room Air	0.000	16:17

Lot No = 13314080A3
Cyl No = 28
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
04/14/2016 16:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:18
02 Std. Gas	0.079	16:18
03 Room Air	0.000	16:19
04 Std. Gas	0.079	16:19
05 Room Air	0.000	16:19
06 Std. Gas	0.079	16:20
07 Room Air	0.000	16:20

Lot No = 13314080A3
Cyl No = 28
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC