

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004196

Location: TOXL

- | | Check When Done |
|---|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>MP3066</u> Lot No. <u>15160</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>MP3068</u> Lot No. <u>14102</u> Rep AC <u>0.400</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>DR5135</u> Lot No. <u>ICS 4</u> Solution <u>0.10AC+0.05% ACETONE</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. 13314080A3 Cyl No. 028 Exp. Date 6/5/2016

Test 1 0.079 Test 4 0.080 Test 7 0.079

Test 2 0.079 Test 5 0.079 Test 8 0.078

Test 3 0.079 Test 6 0.079 Test 9 0.079

Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 110 psi Reg ~100 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED SIMULATOR

RETURN O-RING

4-16-14-2016

Date

15 Apr 16

Date

Charles E. Eide
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
04/14/2016 16:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:26
02 Std. Sol.	0.019	16:27
03 Room Air	0.000	16:27
04 Std. Sol.	0.019	16:28
05 Room Air	0.000	16:28
06 Std. Sol.	0.020	16:29
07 Room Air	0.000	16:30

08 Sim Temp = 34.0°C

Simul Ser No = MP3066
Std Sol No = 15160
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
04/14/2016 16:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:36
02 Std. Sol.	0.407	16:37
03 Room Air	0.000	16:37
04 Std. Sol.	0.410	16:38
05 Room Air	0.000	16:39
06 Std. Sol.	0.409	16:39
07 Room Air	0.000	16:40

08 Sim Temp = 34.0°C

Simul Ser No = MP3068
Std Sol No = 14102
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: HIGH AC
 0.400 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
04/14/2016 16:41

Test	AC	Time
01 Room Air	0.000	16:42
02 *Subject Test	INT*	16:43
03 Room Air	0.000	16:43

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 28
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
04/14/2016 16:43

Test	AC	Time
01 Diagnostic	OK	16:44
02 Room Air	RFI*	16:44
03 Room Air	0.000	16:45

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 28
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
04/14/2016 16:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:46
02 Std. Gas	0.079	16:46
03 Room Air	0.000	16:47
04 Std. Gas	0.079	16:47
05 Room Air	0.000	16:47
06 Std. Gas	0.079	16:48
07 Room Air	0.000	16:48

Lot No = 13314080A3
Cyl No = 28
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
04/14/2016 16:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:50
02 Std. Gas	0.080	16:50
03 Room Air	0.000	16:51
04 Std. Gas	0.079	16:51
05 Room Air	0.000	16:52
06 Std. Gas	0.079	16:52
07 Room Air	0.000	16:53

Lot No = 13314080A3
Cyl No = 28
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
04/14/2016 16:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:53
02 Std. Gas	0.079	16:54
03 Room Air	0.000	16:54
04 Std. Gas	0.078	16:55
05 Room Air	0.000	16:55
06 Std. Gas	0.079	16:55
07 Room Air	0.000	16:56

Lot No = 13314080A3
Cyl No = 28
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000