



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (1-2010)

Serial Number: <u>80-004945</u>	Instrument Location: <u>ASHL</u>
Reason for Install/Repair:	
<input type="checkbox"/> Install After Lab Repair/Inspection	
<input checked="" type="checkbox"/> Other (Specify) <u>NEW INSTALLATION OF DEVICE</u>	

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: NONE

[Signature] MC30
 Field Inspector Signature

4/16/2010
 Date

Deb Kashur
 Reviewed By

07-June 2010
 Date

80-004945
 P. 34437
[Signature]

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = ASHL 8164.13.00 06/09
04/16/2010 16:35

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 709352
Standard Cyl #? 32
Standard Expiration? 12/23/2011
Oper No? 100631

Flow Cal. Date: 01/18/2010
Slope 709
Intercept -721404

IR Calibration Date: 01/18/2010

	3um	9um
0th Coef(*100):	-31758	-31251
1st Coef(*100):	257820	138163
2nd Coef(*100):	2248	1218
H2O adj(mg/l*10k):	753	539

***** Printer Test End *****



Operator Signature
GERARD R. ZLOTKOWSKI

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = ASHL 8164.13.00 06/09
04/16/2010 16:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:41
02 Std. Gas	0.081	16:42
03 Room Air	0.000	16:42
04 Std. Gas	0.079	16:43
05 Room Air	0.000	16:43
06 Std. Gas	0.080	16:44
07 Room Air	0.000	16:44

Lot No = 709352
Cyl No = 32
Exp Date = 12/23/2011
County = 26 Oper No. = 100631



Operator Signature
GERARD R. ZLOTKOWSKI

Remarks:

Form 106-I8000

80-004945
p. 36 of 37
15M

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = ASHL 8164.13.00 06/09
04/16/2010 16:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	16:55
02 Room Air	RFI*	16:56

*Invalid Test
Inhibited - RFI

Lot No = 709352
Cyl No = 32
Exp Date = 12/23/2011
County = 26 Oper No. = 100631



Operator Signature
GERARD R. ZLOTKOWSKI

Remarks:

Form 106-I8000

80-004945
P. 37 of 37