

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004949

Location: TOX LAB

- |   | Check When Done |
|---|-----------------|
| A. Is the warm-up time less than 20 minutes?  | <u>✓</u>        |
| B. Is a three-pronged grounded outlet used?   | <u>✓</u>        |
| C. Is the breath tube heated?   | <u>✓</u>        |
| D. Is the diagnostic check complete?  | <u>✓</u>        |
| E. Is the time, date, and year correct (re-set if necessary)?   | <u>✓</u>        |
| F. Print test. (Attach test record.)  | <u>✓</u>        |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)<br>Sim SN <u>DR5190</u> Lot No. <u>—</u> Rep AC <u>0.019 AC</u>  | <u>✓</u>        |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)<br>Sim SN <u>DR5113</u> Lot No. <u>—</u> Rep AC <u>0.285 AC</u>   | <u>✓</u>        |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.<br>(Attach test record.)<br>Sim SN <u>DR5135</u> Lot No. <u>—</u> Rep AC <u>0.10 AC + Acetone 0.05%</u> | <u>✓</u>        |
| J. RFI check. Run CMS mode. Key radio on first room air.<br>(Attach test record.)   | <u>✓</u>        |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.<br>(Attach test record.)   |                 |

Lot No. <u>659358</u>	Cyl No. <u>39</u>	Exp. Date <u>09/03/2011</u>
Test 1 <u>0.077</u>	Test 4 <u>0.078</u>	Test 7 <u>0.078</u>
Test 2 <u>0.078</u>	Test 5 <u>0.078</u>	Test 8 <u>0.078</u>
Test 3 <u>0.078</u>	Test 6 <u>0.078</u>	Test 9 <u>0.079</u>
Average <u>0.078</u>		

L. Tank pressure, Level 3 Func D Sub G P: 359 psi Reg 400 psi

M. Remarks/Maintenance record: Annual Inspection

16 April 2010  
Date

Mary Platon  
Field Inspector's Signature

19 Apr 2010  
Date

Deb Keshan  
Reviewed by

80-004949  
34/42 km



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004949  
Location = TOXL      8164.13.00 06/09  
04/16/2010      09:48

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:48
02 Std. Sol.	0.015	09:49
03 Room Air	0.000	09:49
04 Std. Sol.	0.015	09:50
05 Room Air	0.000	09:51
06 Std. Sol.	0.015	09:51
07 Room Air	0.000	09:52


08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 0000

County = 08

Oper No. = 107501

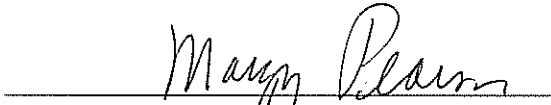

Operator Signature  
MARGY PEARSON

Remarks:

Low AC 0.019 AC

Form 106-I8000

80-004949  
3/6/12 KM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004949  
Location = TOXL      8164.13.00 06/09  
04/16/2010      09:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:24
02 Std. Sol.	0.277	09:26
03 Room Air	0.000	09:26
04 Std. Sol.	0.278	09:27
05 Room Air	0.000	09:27
06 Std. Sol.	0.278	09:28
07 Room Air	0.000	09:29


08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 0000

County = 08

Oper No. = 107501

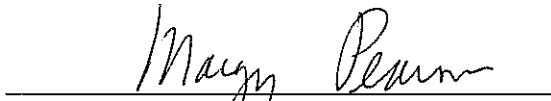

Operator Signature  
MARGY PEARSON

Remarks:

ACA Cal Check  
Linearity 0.285 AC  
Form 106-I8000

80-004949  
37/42 KAL


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004949  
Location = TOXL                    8164.13.00 06/09  
04/16/2010                            08:59

Test	AC	Time
01 Room Air	0.000	09:00
02 *Subject Test	INT*	09:00
03 Room Air	0.000	09:01

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 659358  
Cyl No = 39  
Expiration Date = 09/03/2011  
County = 08                            Oper No. = 107501

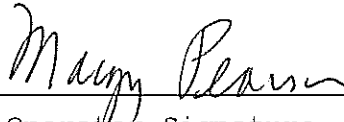

Operator Signature  
MARGY PEARSON

Remarks:

*Alc + Acetone*

Form 106-I8000

80-004949  
38/42 FM




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004949  
Location = TOXL      8164.13.00 06/09  
04/16/2010      10:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:05
02 Std. Gas	0.077	10:05
03 Room Air	0.000	10:06
04 Std. Gas	0.078	10:06
05 Room Air	0.000	10:07
06 Std. Gas	0.078	10:07
07 Room Air	0.000	10:08

Lot No = 659358  
Cyl No = 39  
Exp Date = 09/03/2011  
County = 08      Oper No. = 107501

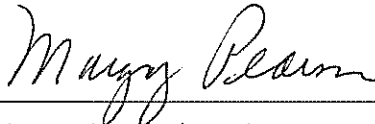

Operator Signature  
MARGY PEARSON

Remarks:

*Cal check*

Form 106-I8000

80-004949  
B  
4/16/12 YFM


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004949  
Location = TOXL      8164.13.00 06/09  
04/16/2010      10:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:09
02 Std. Gas	0.078	10:09
03 Room Air	0.000	10:09
04 Std. Gas	0.078	10:10
05 Room Air	0.000	10:10
06 Std. Gas	0.078	10:11
07 Room Air	0.000	10:11

Lot No = 659358  
Cyl No = 39  
Exp Date = 09/03/2011  
County = 08      Oper No. = 107501

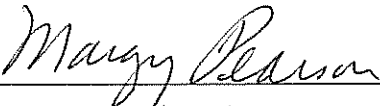  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

*Cal Check*

Form 106-I8000

80-004949  
4/1/12 FR




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004949  
Location = TOXL      8164.13.00 06/09  
04/16/2010      10:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:18
02 Std. Gas	0.078	10:18
03 Room Air	0.000	10:19
04 Std. Gas	0.078	10:19
05 Room Air	0.000	10:20
06 Std. Gas	0.079	10:20
07 Room Air	0.000	10:21

Lot No = 659358  
Cyl No = 39  
Exp Date = 09/03/2011  
County = 08      Oper No. = 107501

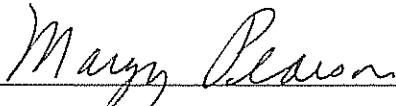  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

*Cal Check*

Form 106-18000

80-004949  
4/2/12 *PM*