

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004950

Location: TOXL

- | | Check When Done |
|---|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5144</u> Lot No. <u>015</u> Rep AC <u>0.019</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5187</u> Lot No. <u> </u> Rep AC <u>0.285</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>G-8766</u> Lot No. <u>✓</u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>709352</u>	Cyl No. <u>38</u>	Exp. Date <u>23 Dec 2011</u>
Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

L. Tank pressure, Level 3 Func D Sub G P: 820 psi Reg 820 psi

M. Remarks/Maintenance record: Annual inspection

16 Apr 2010
Date

Deb Kasher
Field Inspector's Signature

16 Apr 2010
Date

Maggie Pearson
Reviewed by

80-004950
34/42 JEM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.13.00 06/09
04/16/2010 09:22

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 709352
Standard Cyl #? 38
Standard Expiration? 12/23/2011
Oper No? 777777

Flow Cal. Date: 01/18/2010
Slope 774
Intercept -908904

IR Calibration Date: 02/11/2010
 3um 9um

0th Coef(*100): -29762 -38464
1st Coef(*100): 275393 145322
2nd Coef(*100): 383 366
H2O adj(mg/l*10k): 725 627

***** Printer Test End *****

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Print Test

Form 106-18000

80-004950
35/42 JAM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.13.00 06/09
04/16/2010 11:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:16
02 Std. Sol.	0.017	11:19
03 Room Air	0.000	11:20
04 Std. Sol.	0.018	11:21
05 Room Air	0.000	11:21
06 Std. Sol.	0.018	11:22
07 Room Air	0.000	11:22

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.019 AC

Form 106-I8000

80-004950
3/12/12 FK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.13.00 06/09
04/16/2010 10:54

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:55
02 Std. Sol.	0.286	10:56
03 Room Air	0.000	10:56
04 Std. Sol.	0.287	10:57
05 Room Air	0.000	10:58
06 Std. Sol.	0.287	10:58
07 Room Air	0.000	10:59

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.285 AC

Form 106-I8000

80-004950
3/12 AM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.13.00 06/09
04/16/2010 11:09

Test	AC	Time
01 Room Air	0.000	11:09
02 *Subject Test	INT*	11:10
03 Room Air	0.000	11:10

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = 00000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100 alc + acetone

Form 106-18000

80-004950
38/42 FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.13.00 06/09
04/16/2010 11:00

Test	AC	Time
01 Diagnostic	OK	11:01
02 Room Air	RFI*	11:01
03 Room Air	0.000	11:02

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = 00000
Std Sol No = 000
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-004950
3/12/12 KML

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.13.00 06/09
04/16/2010 09:23

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:24
02 Std. Gas	0.079	09:24
03 Room Air	0.000	09:24
04 Std. Gas	0.079	09:25
05 Room Air	0.000	09:25
06 Std. Gas	0.079	09:26
07 Room Air	0.000	09:26

Lot No = 709352
Cyl No = 38
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004950
4/16/12 KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.13.00 06/09
04/16/2010 09:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:29
02 Std. Gas	0.079	09:30
03 Room Air	0.000	09:30
04 Std. Gas	0.079	09:31
05 Room Air	0.000	09:31
06 Std. Gas	0.079	09:32
07 Room Air	0.000	09:32

Lot No = 709352
Cyl No = 38
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

*80-004950
4/1/12 KA*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.13.00 06/09
04/16/2010 09:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:34
02 Std. Gas	0.079	09:34
03 Room Air	0.000	09:34
04 Std. Gas	0.079	09:35
05 Room Air	0.000	09:35
06 Std. Gas	0.079	09:36
07 Room Air	0.000	09:36

Lot No = 709352
Cyl No = 38
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC
Form 106-I8000

80-004950
4/2/12 FAH