

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005939

Location: Cass Co Corr

- | | Check When Done |
|---|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5143</u> Lot No. <u>201501B</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3455</u> Lot No. <u>14270</u> Rep AC <u>0.300</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u>1CS#2</u> Solution <u>0.10AC + 0.05% acetone</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. <u>14214080A6</u>	Cyl No. <u>49</u>	Exp. Date <u>07/05/2016</u>
Test 1 <u>0.083</u>	Test 4 <u>0.082</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.082</u>	Test 8 <u>0.082</u>
Test 3 <u>0.082</u>	Test 6 <u>0.083</u>	Test 9 <u>0.082</u>
Average <u>0.082</u>		

J. Tank pressure, Level 3 Func D Sub G P: 261 psi Reg 260 psi

K. Remarks/Maintenance record: Annual inspection, cleaned breath hose screen, replaced breath hose o-ring

16 Apr 15
Date
17 APR. 15 ✓
Date

Deb Kashner
Field Inspector's Signature
Charles E. [Signature] ✓
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = CASS 8164.13.00 06/09
04/16/2015 07:19

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 14214080A6
Standard Cyl #? 49
Standard Expiration? 07/05/2016
Oper No? 777777

Flow Cal. Date: 06/18/2013
Slope 676
Intercept -672473

IR Calibration Date: 06/18/2013

	3um	9um
0th Coef(*100):	-16377	-17793
1st Coef(*100):	286199	137970
2nd Coef(*100):	3159	1297
H2O adj(mg/l*10k):	522	398

***** Printer Test End *****

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = CASS 8164.13.00 06/09
04/16/2015 07:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:25
02 Std. Sol.	0.020	07:26
03 Room Air	0.000	07:27
04 Std. Sol.	0.020	07:27
05 Room Air	0.000	07:28
06 Std. Sol.	0.020	07:29
07 Room Air	0.000	07:29

08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 201501B

County = 09

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = CASS 8164.13.00 06/09
04/16/2015 07:31

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:31
02 Std. Sol.	0.301	07:32
03 Room Air	0.000	07:33
04 Std. Sol.	0.300	07:33
05 Room Air	0.000	07:34
06 Std. Sol.	0.299	07:35
07 Room Air	0.000	07:35

08 Sim Temp = 34.0°C

Simul Ser No = DR3455

Std Sol No = 14270

County = 09

Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

High AC 0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = CASS 8164.13.00 06/09
04/16/2015 07:38

Test	AC	Time
01 Room Air	0.000	07:39
02 *Subject Test	INT*	07:39
03 Room Air	0.000	07:40

*Invalid Test
Interferent Detected

Sub Name = TEST, NA NA
Sub DOB = 08/13/1980
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 14214080A6
Cyl No = 49
Expiration Date = 07/05/2016
County = 09 Oper No. = 777777

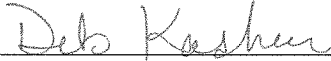

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.10 AC + 0.05 acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = CASS 8164.13.00 06/09
04/16/2015 07:40

Test	AC	Time
01 Diagnostic	OK	07:41
02 Room Air	RFI*	07:41
03 Room Air	0.000	07:42

*Invalid Test
Inhibited - RFI

Sub Name = RFI, NA NA
Sub DOB = 08/13/1980
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 14214080A6
Cyl No = 49
Expiration Date = 07/05/2016
County = 09 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = CASS 8164.13.00 06/09
04/16/2015 07:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:20
02 Std. Gas	0.083	07:20
03 Room Air	0.000	07:21
04 Std. Gas	0.082	07:21
05 Room Air	0.000	07:22
06 Std. Gas	0.082	07:22
07 Room Air	0.000	07:23

Lot No = 14214080A6
Cyl No = 49
Exp Date = 07/05/2016
County = 09

Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = CASS 8164.13.00 06/09
04/16/2015 07:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:43
02 Std. Gas	0.082	07:43
03 Room Air	0.000	07:43
04 Std. Gas	0.082	07:44
05 Room Air	0.000	07:44
06 Std. Gas	0.083	07:45
07 Room Air	0.000	07:45

Lot No = 14214080A6
Cyl No = 49
Exp Date = 07/05/2016
County = 09 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = CASS 8164.13.00 06/09
04/16/2015 07:46

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:46
02 Std. Gas	0.082	07:47
03 Room Air	0.000	07:47
04 Std. Gas	0.082	07:47
05 Room Air	0.000	07:48
06 Std. Gas	0.082	07:48
07 Room Air	0.000	07:49

Lot No = 14214080A6

Cyl No = 49

Exp Date = 07/05/2016

County = 09

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000