

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005944

Location: Wahpeton

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5143 Lot No. 201501B Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3455 Lot No. 14270 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5132 Lot No. 105#2 Solution 0.10 AC + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 33913080A4 Cyl No. 20 Exp. Date 01/01/2016
Test 1 0.082 Test 4 0.083 Test 7 0.082
Test 2 0.082 Test 5 0.082 Test 8 0.082
Test 3 0.082 Test 6 0.082 Test 9 0.083
Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 832 psi Reg 840 psi

K. Remarks/Maintenance record: Annual inspection, checked breath hose screen - good,

16 Apr 15
Date

17 APR 15 ✓
Date

Deb Kashur
Field Inspector's Signature

Charles E. Dan
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
04/16/2015 09:27

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 33913080A4
Standard Cyl #? 20
Standard Expiration? 01/01/2016
Oper No? 777777

Flow Cal. Date: 06/20/2013
Slope 677
Intercept -702620

IR Calibration Date: 06/20/2013
 3um 9um


0th Coef(*100): -20012 -21455
1st Coef(*100): 275397 137470
2nd Coef(*100): 2699 1252
H2O adj(mg/l*10k): 704 563

***** Printer Test End *****



Operator Signature
DEB KASHUR

Remarks:



Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
04/16/2015 09:31

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:32
02 Std. Sol.	0.017	09:32
03 Room Air	0.000	09:33
04 Std. Sol.	0.017	09:34
05 Room Air	0.000	09:34
06 Std. Sol.	0.017	09:35
07 Room Air	0.000	09:35


08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 201501B

County = 39

Oper No. = 777777

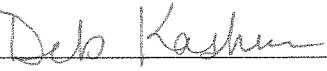

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.020AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
04/16/2015 09:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:58
02 Std. Sol.	0.293	09:59
03 Room Air	0.000	10:00
04 Std. Sol.	0.294	10:00
05 Room Air	0.000	10:01
06 Std. Sol.	0.296	10:02
07 Room Air	0.000	10:02

08 Sim Temp = 34.0°C

Simul Ser No = DR3455
Std Sol No = 14270
County = 39 Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

*High Ac ^{DK}
~~0.0~~ 0.300 AC*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
04/16/2015 09:53

Test	AC	Time
01 Room Air	0.000	09:54
02 *Subject Test	INT*	09:55
03 Room Air	0.000	09:55

*Invalid Test
Interferent Detected

Sub Name = TEST, NA NA
Sub DOB = 08/13/1980
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 33913080A4
Cyl No = 20
Expiration Date = 01/01/2016
County = 39 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.10AC + 0.05% acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
04/16/2015 09:55

Test	AC	Time
01 Diagnostic	OK	09:56
02 Room Air	RFI*	09:56
03 Room Air	0.000	09:57

*Invalid Test
 Inhibited - RFI

Sub Name = RFI, NA NA
Sub DOB = 08/13/1980
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 33913080A4
Cyl No = 20
Expiration Date = 01/01/2016
County = 39 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
04/16/2015 09:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:40
02 Std. Gas	0.082	09:40
03 Room Air	0.000	09:41
04 Std. Gas	0.082	09:41
05 Room Air	0.000	09:42
06 Std. Gas	0.082	09:42
07 Room Air	0.000	09:43

Lot No = 33913080A4
Cyl No = 20
Exp Date = 01/01/2016
County = 39 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
04/16/2015 09:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:47
02 Std. Gas	0.083	09:48
03 Room Air	0.000	09:48
04 Std. Gas	0.082	09:49
05 Room Air	0.000	09:49
06 Std. Gas	0.082	09:49
07 Room Air	0.000	09:50

Lot No = 33913080A4
Cyl No = 20
Exp Date = 01/01/2016
County = 39 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
04/16/2015 09:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:51
02 Std. Gas	0.082	09:51
03 Room Air	0.000	09:52
04 Std. Gas	0.082	09:52
05 Room Air	0.000	09:52
06 Std. Gas	0.083	09:53
07 Room Air	0.000	09:53

Lot No = 33913080A4
Cyl No = 20
Exp Date = 01/01/2016
County = 39 Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000