

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004946

Location: TOXL

- | | Check When Done |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5144</u> Lot No. <u> </u> Rep AC <u>0.019</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5187</u> Lot No. <u> </u> Rep AC <u>0.285</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>G-8766</u> Lot No. <u> </u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>709352</u>	Cyl No. <u>2</u>	Exp. Date <u>23 Dec 2011</u>
Test 1 <u>0.078</u>	Test 4 <u>0.078</u>	Test 7 <u>0.078</u>
Test 2 <u>0.078</u>	Test 5 <u>0.078</u>	Test 8 <u>0.078</u>
Test 3 <u>0.077</u>	Test 6 <u>0.078</u>	Test 9 <u>0.078</u>
Average <u>0.078</u>		

L. Tank pressure, Level 3 Func D Sub G P: 456 psi Reg 470 psi

M. Remarks/Maintenance record: Annual inspection
19 Apr 2010 DK
16 Apr 2010
Date 19 Apr 2010
Date

Deb Kashur
Field Inspector's Signature
Mary Blair
Reviewed by

80-004946
34/42 km

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004946
Location = TOXL 8164.13.00 06/09
04/16/2010 11:34

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 709352
Standard Cyl #? 2
Standard Expiration? 12/23/2011
Oper No? 777777

Flow Cal. Date: 01/18/2010
Slope 717
Intercept -728279

IR Calibration Date: 02/10/2010

	3um	9um
0th Coef(*100):	4321	-2310
1st Coef(*100):	254996	129019
2nd Coef(*100):	1560	1203
H2O adj(mg/l*10k):	410	499

***** Printer Test End *****

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Print test

Form 106-I8000

80-004946
3/5/12 *[initials]*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004946
Location = TOXL 8164.13.00 06/09
04/16/2010 13:13

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:14
02 Std. Sol.	0.022	13:14
03 Room Air	0.000	13:15
04 Std. Sol.	0.022	13:16
05 Room Air	0.000	13:16
06 Std. Sol.	0.022	13:17
07 Room Air	0.000	13:17

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.019 AC

Form 106-I8000

80-004946
3/4/12 JEM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004946
Location = TOXL 8164.13.00 06/09
04/16/2010 11:42

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:43
02 Std. Sol.	0.280	11:44
03 Room Air	0.000	11:44
04 Std. Sol.	0.282	11:45
05 Room Air	0.000	11:46
06 Std. Sol.	0.283	11:46
07 Room Air	0.000	11:47

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.285 AC

Form 106-I8000

80-004946
37/42 KA

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004946
Location = TOXL 8164.13.00 06/09
04/19/2010 09:10

Test	AC	Time
01 Room Air	0.000	09:11
02 *Subject Test	INT*	09:12
03 Room Air	0.000	09:12

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 709352
Cyl No = 2
Expiration Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100 AC + acetone

Form 106-I8000

80-004946
38/42 JEM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004946
Location = TOXL 8164.13.00 06/09
04/19/2010 09:12

Test	AC	Time
01 Diagnostic	OK	09:14
02 Room Air	RFI*	09:14
03 Room Air	0.000	09:14

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 709352
Cyl No = 2
Expiration Date = 12/23/2011
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

*80-004946
3/12 FM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004946
Location = TOXL 8164.13.00 06/09
04/19/2010 09:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:26
02 Std. Gas	0.078	09:26
03 Room Air	0.000	09:27
04 Std. Gas	0.078	09:27
05 Room Air	0.000	09:27
06 Std. Gas	0.077	09:28
07 Room Air	0.000	09:28

Lot No = 709352
Cyl No = 2
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-009946
4/19/12 JFH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004946
Location = TOXL 8164.13.00 06/09
04/19/2010 09:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:29
02 Std. Gas	0.078	09:30
03 Room Air	0.000	09:30
04 Std. Gas	0.078	09:30
05 Room Air	0.000	09:31
06 Std. Gas	0.078	09:31
07 Room Air	0.000	09:32

Lot No = 709352
Cyl No = 2
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004946
2/1/42 FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004946
Location = TOXL 8164.13.00 06/09
04/19/2010 09:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:33
02 Std. Gas	0.078	09:33
03 Room Air	0.000	09:34
04 Std. Gas	0.078	09:34
05 Room Air	0.000	09:35
06 Std. Gas	0.078	09:35
07 Room Air	0.000	09:36

Lot No = 709352
Cyl No = 2
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

*80-004946
4/1/12 YAM*