

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004939

Location: Mandan jail

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u> </u> Rep AC <u>0.021</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5134</u> Lot No. <u> </u> Rep AC <u>0.294</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3846</u> Lot No. <u> </u> Rep AC <u>0.10 AC + acetone</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>659358</u>	Cyl No. <u>40</u>	Exp. Date <u>03 Sept 2011</u>
Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.080</u>
Test 2 <u>0.081</u>	Test 5 <u>0.080</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

L. Tank pressure, Level 3 Func D Sub G P: 581 psi Reg 580 psi

M. Remarks/Maintenance record: Annual inspection

19 Apr 11
Date

Deb Keshur
Field Inspector's Signature

4-19-11
Date

[Signature]
Reviewed by

80-004939
P. 53 of 61
JAN

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = MAND 8164.13.00 06/09
04/19/2011 11:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:35
02 Std. Sol.	0.017	11:36
03 Room Air	0.000	11:37
04 Std. Sol.	0.017	11:38
05 Room Air	0.000	11:38
06 Std. Sol.	0.017	11:39
07 Room Air	0.000	11:39

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 30

Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Low AC 0.021 AC

Form 106-I8000

80-004939
P. 55 of 61
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = MAND 8164.13.00 06/09
04/19/2011 11:40


WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:41
02 Std. Sol.	0.301	11:42
03 Room Air	0.000	11:42
04 Std. Sol.	0.298	11:43
05 Room Air	0.000	11:44
06 Std. Sol.	0.298	11:44
07 Room Air	0.000	11:45

08 Sim Temp = 34.0°C

Simul Ser No = 0
Std Sol No = 0
County = 30

Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.294 AC

Form 106-I8000

80-004939
P. 56 of 61
JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = MAND 8164.13.00 06/09
04/19/2011 12:00

Test	AC	Time
01 Room Air	0.000	12:01
02 *Subject Test	INT*	12:01
03 Room Air	0.000	12:02

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 40
Expiration Date = 09/03/2011
County = 30 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent test 0.10Ac + acetone

Form 106-I8000

80-004939
p 57 of 61
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = MAND 8164.13.00 06/09
04/19/2011 12:02

Test	AC	Time
01 Diagnostic	OK	12:03
02 Room Air	RFI*	12:03
03 Room Air	0.000	12:04

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 40
Expiration Date = 09/03/2011
County = 30 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-004939
P. 58 of 61
YAM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = MAND 8164.13.00 06/09
04/19/2011 11:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:50
02 Std. Gas	0.081	11:50
03 Room Air	0.000	11:50
04 Std. Gas	0.081	11:51
05 Room Air	0.000	11:51
06 Std. Gas	0.081	11:52
07 Room Air	0.000	11:52

Lot No = 659358
Cyl No = 40
Exp Date = 09/03/2011
County = 30 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004939
P. 59 of 61
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = MAND 8164.13.00 06/09
04/19/2011 11:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:53
02 Std. Gas	0.081	11:54
03 Room Air	0.000	11:54
04 Std. Gas	0.080	11:55
05 Room Air	0.000	11:55
06 Std. Gas	0.081	11:55
07 Room Air	0.000	11:56

Lot No = 659358
Cyl No = 40
Exp Date = 09/03/2011
County = 30 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004939
P. 60 of 61
RM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = MAND 8164.13.00 06/09
04/19/2011 11:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:57
02 Std. Gas	0.080	11:57
03 Room Air	0.000	11:58
04 Std. Gas	0.081	11:58
05 Room Air	0.000	11:58
06 Std. Gas	0.081	11:59
07 Room Air	0.000	11:59

Lot No = 659358
Cyl No = 40
Exp Date = 09/03/2011
County = 30 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC

Form 106-I8000

80-004939
P. Lotafel
KM