



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (03-2014)

Serial Number <b>80-00 4194</b>	Instrument Location <b>FYPD (STANDING ROCK)</b>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <b>INSTALL DURING ANNUAL INSPECTION</b>	

### Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P). ✓
  - B. ACA Test (Level 1, Function C). ✓
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test). ✓
- 7. Repair and/or Maintenance Performed (if Any): **NA - INSTALL DURING ANNUAL INSPECTION**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Charles E. Edr</i>	Date <b>4.20.15 ✓</b>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashur</i>	Date <b>28 Apr 2015</b>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Edr*      Certified Date: **08 Nov. 16**




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004194  
Location = FYPD      8164.13.00 06/09  
04/20/2015      15:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:24
02 Std. Gas	0.080	15:25
03 Room Air	0.000	15:25
04 Std. Gas	0.080	15:26
05 Room Air	0.000	15:26
06 Std. Gas	0.080	15:27
07 Room Air	0.000	15:27

Lot No = 33913080A4  
Cyl No = 15  
Exp Date = 01/01/2016  
County = 61      Oper No. = 666666

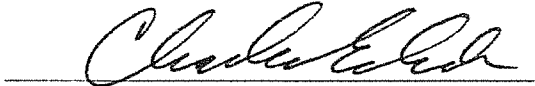

Operator Signature  
CHARLES EDER

Remarks:      *ACA TEST*  
                    *0.080 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004194  
Location = FYPD      8164.13.00 06/09  
04/20/2015      15:28

Test	AC	Time
01 Diagnostic	OK	15:29
02 Room Air	RFI*	15:29
03 Room Air	0.000	15:30

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 33913080A4  
Cyl No = 15  
Expiration Date = 01/01/2016  
County = 61      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK