

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004945

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR377 Lot No. 13270 Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5189 Lot No. 13300 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. JCS#2 Rep AC 0.100AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. 16512080A2 Cyl No. 019 Exp. Date 08/01/2014
Test 1 0.083 Test 4 0.083 Test 7 0.083
Test 2 0.083 Test 5 0.082 Test 8 0.083
Test 3 0.083 Test 6 0.083 Test 9 0.083
Average 0.083

J. Tank pressure, Level 3 Func D Sub G P: 904 psi Reg 900 psi

K. Remarks/Maintenance record: REPLACED SIMULATOR RETURN AND BREATH

HOSE O-RINGS, CLEANED BREATH HOSE SCREEN, ANNUAL INSPECTION

17 APR 2014
Date

Charles E. Ed
Field Inspector's Signature

21 Apr 2014 ✓
Date

Deb Kashner
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.13.00 06/09
04/17/2014 11:19

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 16512080A2
Standard Cyl #? 19
Standard Expiration? 08/01/2014
Oper No? 666666

Flow Cal. Date: 09/26/2012
Slope 622
Intercept -485836

IR Calibration Date: 10/01/2012
 3um 9um

0th Coef(*100): -23823 -20690
1st Coef(*100): 260812 140752
2nd Coef(*100): 3107 1288
H2O adj(mg/l*10k): 695 492

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.13.00 06/09
04/17/2014 11:32

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:32
02 Std. Sol.	0.022	11:33
03 Room Air	0.000	11:34
04 Std. Sol.	0.022	11:34
05 Room Air	0.000	11:35
06 Std. Sol.	0.021	11:36
07 Room Air	0.000	11:36

08 Sim Temp = 34.0°C

Simul Ser No = DR3377

Std Sol No = 13270

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

Low AC

0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.13.00 06/09
04/17/2014 11:54

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:54
02 Std. Sol.	0.304	11:55
03 Room Air	0.000	11:56
04 Std. Sol.	0.305	11:57
05 Room Air	0.000	11:57
06 Std. Sol.	0.304	11:58
07 Room Air	0.000	11:58

08 Sim Temp = 34.0°C

Simul Ser No = DR5189
Std Sol No = 13300
County = 08 Oper No. = 666666

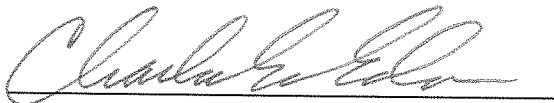

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.13.00 06/09
04/17/2014 12:02

Test	AC	Time
01 Room Air	0.000	12:02
02 *Subject Test	INT*	12:03
03 Room Air	0.000	12:03

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 16512080A2
Cyl No = 19
Expiration Date = 08/01/2014
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *INTERFERENT CHECK*
0.100 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.13.00 06/09
04/17/2014 12:05

Test	AC	Time
01 Diagnostic	OK	12:06
02 Room Air	RFI*	12:06
03 Room Air	0.000	12:06

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 16512080A2
Cyl No = 19
Expiration Date = 08/01/2014
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.13.00 06/09
04/17/2014 12:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:08
02 Std. Gas	0.083	12:09
03 Room Air	0.000	12:09
04 Std. Gas	0.083	12:10
05 Room Air	0.000	12:10
06 Std. Gas	0.083	12:10
07 Room Air	0.000	12:11

Lot No = 16512080A2
Cyl No = 19
Exp Date = 08/01/2014
County = 08 Oper No. = 666666

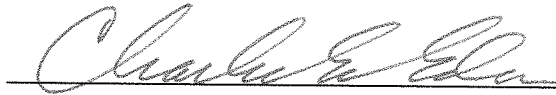

Operator Signature
CHARLES EDER

Remarks: 0.080 AC

CALIBRATION CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.13.00 06/09
04/17/2014 12:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:13
02 Std. Gas	0.083	12:13
03 Room Air	0.000	12:14
04 Std. Gas	0.082	12:14
05 Room Air	0.000	12:15
06 Std. Gas	0.083	12:15
07 Room Air	0.000	12:16

Lot No = 16512080A2
Cyl No = 19
Exp Date = 08/01/2014
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.13.00 06/09
04/17/2014 12:16

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:17
02 Std. Gas	0.083	12:17
03 Room Air	0.000	12:17
04 Std. Gas	0.083	12:18
05 Room Air	0.000	12:18
06 Std. Gas	0.083	12:19
07 Room Air	0.000	12:19

Lot No = 16512080A2
Cyl No = 19
Exp Date = 08/01/2014
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000