



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (03-2014)

Serial Number <b>80-004953</b>	Instrument Location <b>EMMONS COUNTY S.O. (LINT)</b>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <b>INSTALL DURING ANNUAL INSPECTION</b>	

### Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P) ✓
  - B. ACA Test (Level 1, Function C) ✓
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test) ✓
- 7. Repair and/or Maintenance Performed (if Any): INSTALL DURING ANNUAL INSPECTION
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Charles E. Eder</i>	Date <b>4.21.15 ✓</b>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <b>28 Apr 2015</b>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Eder* Certified Date: **29 Nov. 2016**

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004953  
Location = LINT                 8164.13.00 06/09  
04/21/2015                        10:54

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            13314080A3  
Standard Cyl #?            34  
Standard Expiration?       06/05/2016  
Oper No?                    666666

Flow Cal. Date:            01/19/2010  
Slope                        691  
Intercept                   -628068

IR Calibration Date:        04/07/2015  
                              3um                        9um

-----  
0th Coef(\*100):            -18160                    -22754  
1st Coef(\*100):            254326                    129076  
2nd Coef(\*100):            3670                      1866  
H2O adj(mg/l\*10k):        414                        530

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004953  
Location = LINT      8164.13.00 06/09  
04/21/2015      10:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:58
02 Std. Gas	0.080	10:58
03 Room Air	0.000	10:59
04 Std. Gas	0.080	10:59
05 Room Air	0.000	11:00
06 Std. Gas	0.080	11:00
07 Room Air	0.000	11:01

Lot No = 13314080A3  
Cyl No = 34  
Exp Date = 06/05/2016  
County = 15      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:      *ACA TEST*  
                    *0.080 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004953  
Location = LINT      8164.13.00 06/09  
04/21/2015      11:01

Test	AC	Time
01 Diagnostic	OK	11:02
02 Room Air	RFI*	11:02
03 Room Air	0.000	11:03

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 13314080A3  
Cyl No = 34  
Expiration Date = 06/05/2016  
County = 15      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

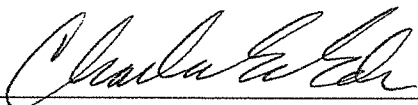

Operator Signature  
CHARLES EDER

Remarks:

*RFI Check*