



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <i>80-005951</i>	Instrument Location <i>Kidder County 50</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>4-27-14</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Keshu</i>	Date <i>22 Apr 2014</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005951  
Location = STEE                    8164.13.00 06/09  
04/22/2014                            12:10

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

Current Instrument Setup

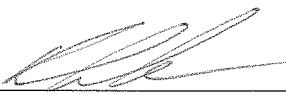
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            16512080A2  
Standard Cyl #?            14  
Standard Expiration?       08/01/2014  
Oper No?                    011107

Flow Cal. Date:            06/21/2013  
Slope                        693  
Intercept                   -721277

IR Calibration Date:        04/08/2014  
                                    3um                            9um

-----  
0th Coef(\*100):            -27997                        -27896  
1st Coef(\*100):            265131                        138103  
2nd Coef(\*100):            2321                            1004  
H2O adj(mg/l\*10k):        795                            708

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Operator Signature  
BRANT WEISENBURGER

Remarks:

*Print Test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005951  
Location = STEE      8164.13.00 06/09  
04/22/2014      12:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:44
02 Std. Gas	0.080	12:44
03 Room Air	0.000	12:44
04 Std. Gas	0.080	12:45
05 Room Air	0.000	12:45
06 Std. Gas	0.080	12:46
07 Room Air	0.000	12:46

Lot No = 16512080A2  
Cyl No = 14  
Exp Date = 08/01/2014  
County = 22      Oper No. = 011107



Operator Signature  
BRANT WEISENBURGER

Remarks:

ACA

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005951  
Location = STEE      8164.13.00 06/09  
04/22/2014      12:53

Test	AC	Time
01 Room Air	RFI*	12:54
02 Room Air	0.000	12:55

\*Invalid Test  
Inhibited - RFI



Operator Signature  
BRANT WEISENBURGER

Remarks:

*RFI*

Form 106-I8000